



Ecstasy Bath Salts Multi-Use Tests DR-4000



Read Before Use

As a precaution, gloves and eye protection should be worn when handling substances and using test units. If a spill occurs, thoroughly rinse affected area with water. Keep out of reach of children. Store the tubes in a dark place, not exposed to light. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

This test consists of one tube. Since the tube contains enough liquid to test many samples, do not dispose of the tube until all desired samples are tested. For additional tests with the same tube, take a new sheet of test paper from the pad to test the next sample.

Test Preparation

1

Remove a sheet of test paper from the pad.



2

Substance

Touch the test paper on the suspected substance.



Traces


Wipe the test paper across the suspected surface.



Identification


3

With **thumb and forefinger**, squeeze the test tube until the ampule breaks.

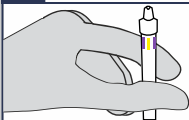

4

Apply a drop to the test paper.

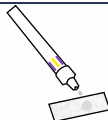

CAUTION:

**The reagent in the tube contains an acid.
Avoid skin contact with the reagent.**

Results



The appearance of a violet color strongly indicates the presence of **Ecstasy/MDMA** or its analogues.
* The color may take up to 3 minutes to develop.



The appearance of a yellow color indicates the presence of Bath Salts such as **MDPV**, **Methylone**, or **Butylone**.



If none of the above colors develop, the above listed drugs are not present.

Disposal



When time comes to dispose of the tube, dilute the remaining reagent with water before placing in the trash.



After diluting the reagent, the tube is safe to dispose of in the trash.

01-19



Manufactured by IDenta Ltd * Made in Israel
www.identa-corp.com * Patented Technology

LOT:

EXP: