



The twiist Pump Button

When enabled, the pump button can be used to deliver a bolus of insulin. This can be helpful if the twiist App is not communicating with the pump or if the iPhone is not connected to Bluetooth. Basal insulin will continue to be delivered if the pump is away from your phone.



If the pump button will be used, the One-Button Bolus feature must be enabled in the twiist App under Therapy Settings.

To Deliver Bolus Insulin Using the One-Button Bolus Feature

- 1 Press the pump button for each unit of bolus insulin you want to deliver. You will hear a beep with each button press.



Each button press represents one unit of insulin.

It is limited by your max bolus setting or 10 units, whichever is lower.

- 2 When you have reached the number of units you would like to bolus, press and hold the pump button until you hear the playback start, then release the pump button.
- 3 Listen and confirm that the pump plays back the number of units you entered.
- 4 If the playback is correct, press and hold the pump button until you hear three ascending tones.
- 5 If the playback is incorrect, stop interacting with the pump. The pump will play a long beep, indicating that the one button bolus has timed out and been canceled.



The Temporary Status Banner in the twiist App Home Screen tracks the progress of the One-Button Bolus delivery. It displays the number of units delivered out of the total units programmed.



For people who do not self-manage their diabetes care, leave the One-Button Bolus feature disabled in the twiist App to avoid accidental bolus delivery.

To Verify Basal Insulin Delivery

- 1 The pump button can be used to verify that basal insulin delivery is in progress, either when the twiist App is not available or if you want an audio confirmation.
- 2 Press and hold the pump button for approximately three seconds.
 - If the pump beeps three ascending tones, delivery is in progress.
 - If the pump sounds a single beep, insulin is not being delivered. Additional action is required to resume insulin delivery. Refer to user guide section Resume Basal Insulin Delivery.

For additional details on the functionality of the Pump Button, please refer to the twiist AID System User Guide.

Customer Care: 1 877-4TWIIST (1-877-489-4478)

The twiist™ Automated Insulin Delivery (AID) System is for people with type 1 diabetes, 6 years of age and above, available with a prescription, only in the U.S.

© 2025 Sequel Med Tech, LLC. All rights reserved. All other trademarks are the property of their respective owners. Visit twiist.com/safety for important safety information.