

Infusion Set and Site Changes

Changing an infusion set can occur with or without a cassette change.




WARNING


Never prime the tubing with it connected to a body.

Changing the infusion set with a cassette change

The infusion set is changed every two to three days during a cassette change.

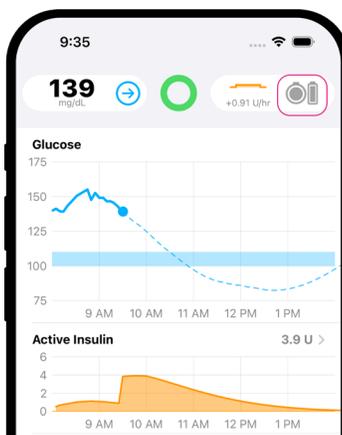


1 Tap **Pump Status/Menu**

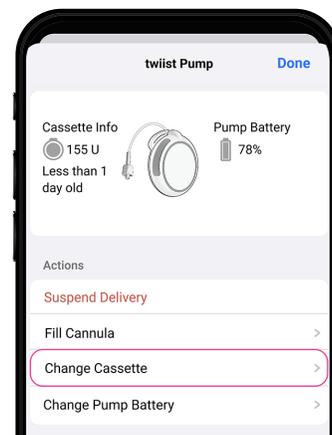


2 Tap **Change Cassette**
3 Follow the instructions in the Twiist App

Changing your infusion set without a cassette change

If you need to change your infusion site for any reason, outside of a cassette change, you may need to fill your cannula. Insulin delivery must be in progress to complete a cannula fill.

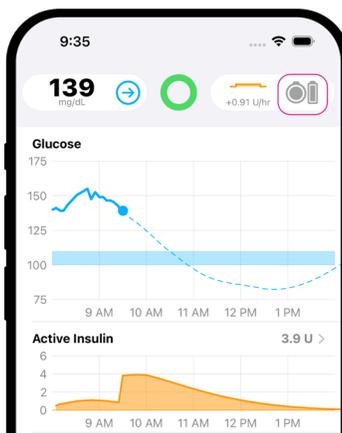

3 Tap **Pump Status/Menu**

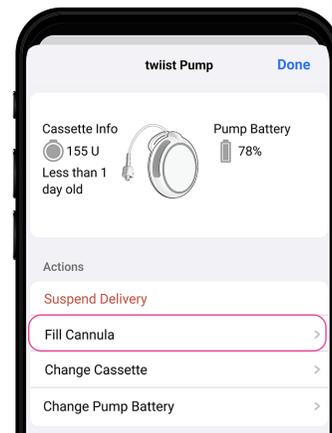


4 Tap **Fill Canula**
5 Follow the instructions in the Twiist App

1 Insert a new infusion set onto body based on manufacturer's instructions
2 Connect tubing

An infusion set can be inserted anywhere insulin can be injected. Insulin absorption can vary from site to site and can be reviewed with your Certified twiist Trainer or your Healthcare Provider (HCP).

Common infusion site locations include:


- Abdomen
- Upper buttocks
- Hips
- Upper arms
- Upper legs

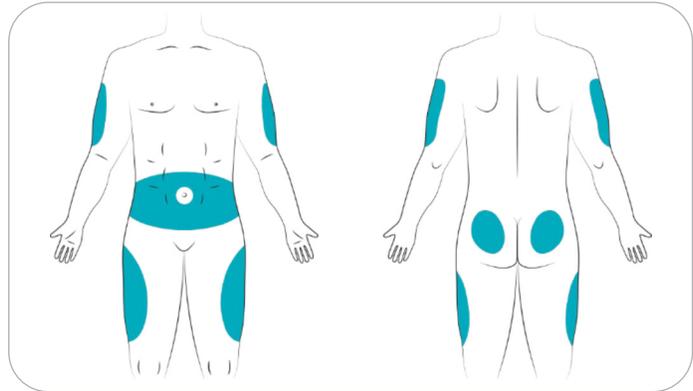

Tips for inserting an infusion set

- 1 Always clean the infusion site area with a prep wipe before inserting.
- 2 Change infusion set every 2-3 days or when the twiist Cassette is empty.
- 3 Insert infusion set at least 2-3 inches away from previous sites.
- 4 Keep the infusion set at least 3 inches away from your CGM site.
- 5 Do not place in areas that may constrict or where the body naturally bends.
- 6 Do not insert into scar tissue, moles, stretch marks, tattoos, or areas with lumps.

Use the "M," "W," or "clock" method to rotate the site location.

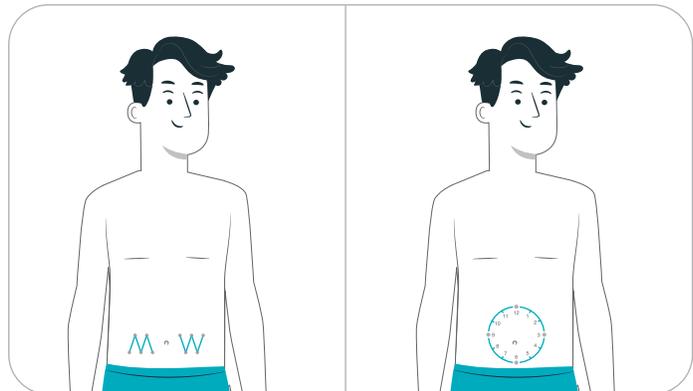

Check your infusion set every day to make sure it's still properly attached. Replace your infusion site if you notice leaks around the site.

Customer Care: 1 877-4TWIIST (1-877-489-4478)

The twiist™ Automated Insulin Delivery (AID) System is for people with type 1 diabetes, 6 years of age and above, available with a prescription, only in the U.S.

© 2025 Sequel Med Tech, LLC. All rights reserved. All other trademarks are the property of their respective owners. Visit twiist.com/safety for important safety information.