

# How to Change a Cassette

The twiist™ AID System ensures continuous, accurate insulin delivery to help manage your blood sugar. Changing the cassette is an important part of using the twiist AID System.


## When to Change a Cassette

A cassette change will need to be completed every 72 hours (3 days) or when the cassette is empty, whichever occurs first.

## What You'll Need

-  Fully charged battery
-  New cassette
-  Alcohol wipes
-  New infusion set
-  Sterile 3 mL syringe with 27-gauge x 3/8 inch needle
-  Humalog or Novolog U-100 insulin

To start the cassette change, tap the  icon on the Home Screen.


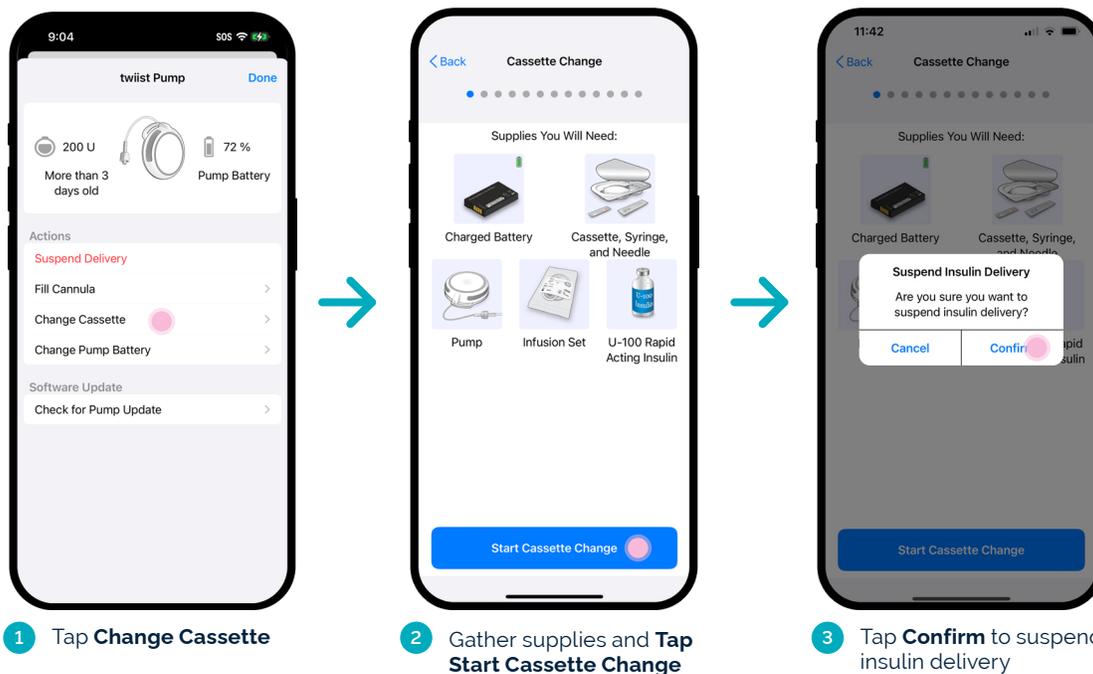


Follow the step-by-step instructions in the twiist App to complete the cassette change, cassette fill, and infusion set change.

Refer to the twiist User Guide as needed for detailed support on the twiist App prompts. Refer to the Infusion Set User Guide for specific infusion set guidance.

**IMPORTANT!**

When filling the cassette with insulin, insert the needle at a 90° angle into the fill port. If you encounter strong resistance, remove and reinsert the needle.

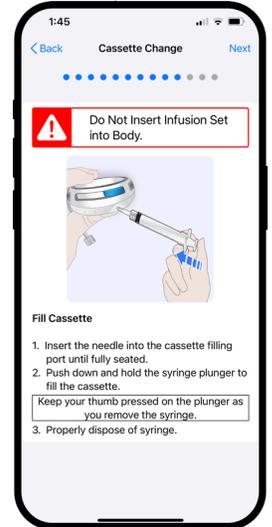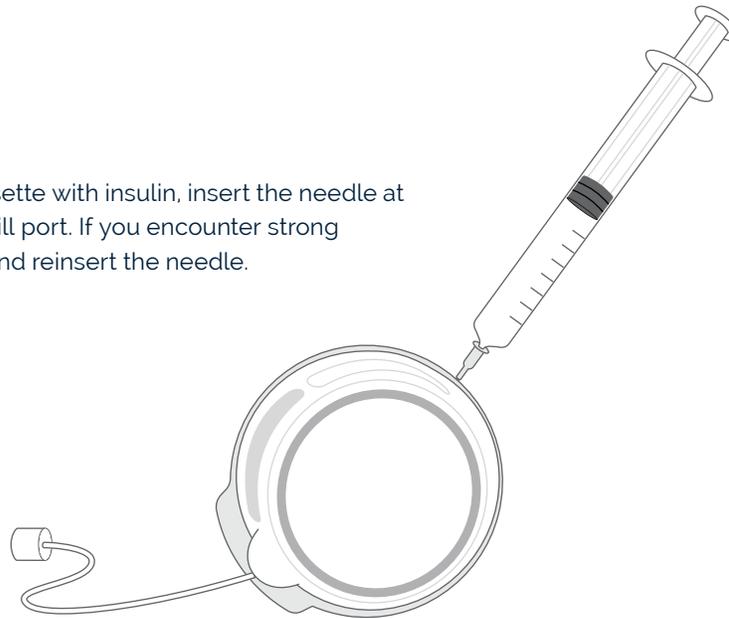

**WARNING**

Do not fill a cassette before you plan to use it.

**WARNING**

Do not fill the cassette with insulin pen devices.

**IMPORTANT!**

Always disconnect from your infusion site before starting a cassette change.

**Customer Care: 1 877-4TWIIST (1-877-489-4478)**

The twist™ Automated Insulin Delivery (AID) System is for people with type 1 diabetes, 6 years of age and above, available with a prescription, only in the U.S.

© 2025 Sequel Med Tech, LLC. All rights reserved. All other trademarks are the property of their respective owners. Visit [twiist.com/safety](https://twiist.com/safety) for important safety information.