



Suspend and Resume Insulin Delivery


There may be times when you will want or need to suspend insulin delivery. You can suspend your insulin delivery through the twiist App, and you can resume delivery either through the app or on the pump itself.

How to Suspend Insulin Delivery

- 1 Tap the **Pump Status** icon  on the Home Screen to open the Pump Menu.
- 2 Tap **Suspend Delivery**.
- 3 Tap **Confirm** to suspend insulin delivery.

The pump will play four descending tones.

The pump sound will not be played when Quiet Mode is enabled.

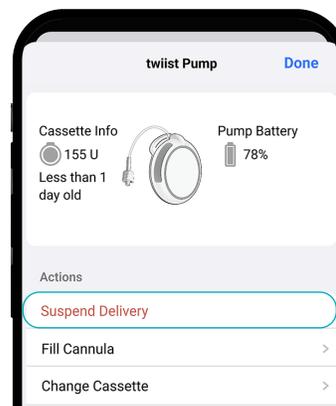


Resume Insulin Delivery from the twiist App

- 1 Tap the **Temporary Status** Banner or tap the Insulin Suspended icon.
- 2 Tap **Resume Delivery**.

The Home Screen will display a Communicating with Pump message and the pump will play three ascending tones.

The pump sound will not be played when Quiet Mode is enabled.

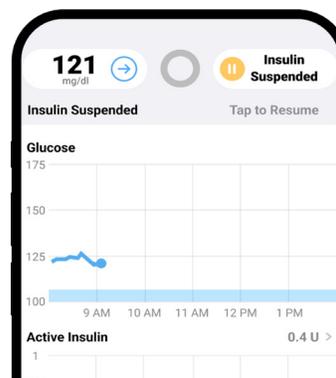

Resume Insulin Delivery from the twiist Pump

Press and hold the pump button for approximately three seconds, until the pump plays three ascending tones.

If the pump sound is a beep, insulin is not being delivered. Additional action is required to resume insulin delivery.

Refer to user guide section *Resume Basal Insulin Delivery*.



Customer Care: 1 877-4TWIIST (1-877-489-4478)

The twiist™ Automated Insulin Delivery (AID) System is for people with type 1 diabetes, 6 years of age and above, available with a prescription, only in the U.S.

© 2025 Sequel Med Tech, LLC. All rights reserved. All other trademarks are the property of their respective owners. Visit twiist.com/safety for important safety information.

TRNG-00369 Ver. 3.0