




twiist AID System: iPhone and Pump Alert Settings

The twiist Automated Insulin Delivery (AID) System uses iPhone sounds, vibrations, and messages to help keep you safe. These will appear on your iPhone and on your twiist Pump (sound only). Two main settings control how your iPhone and twiist Pump notify you: iPhone settings and twiist App settings.

iPhone Settings

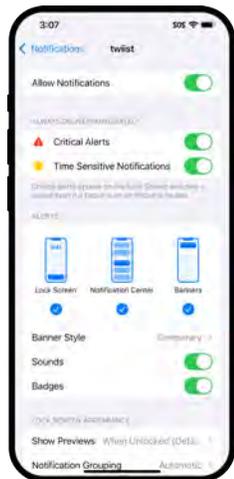


Notification ON

Allow Notifications and Critical Alerts for the twiist App in your iPhone settings.

You can customize how you want to receive notifications.

If lock screen and notification center are turned off, notifications will be received but disappear quickly.

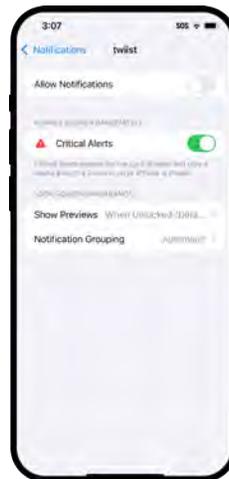

Notification OFF

It is not recommended to disable "Critical Alerts".

If you choose to disable "Allow Notifications", non-critical alerts will not make a sound or appear on the screen.

Always keep "Critical Alerts" toggled on for safety-related notifications.

As a safety feature, urgent low glucose (less than 55 mg/dL) will alarm on the pump regardless of phone settings. Other CGM alerts will not sound on the pump but certain twiist alerts will sound on the pump when it is out of communication with the iPhone.



Focus Features, including **Do Not Disturb** may not play sounds, vibrate, or display notifications on the screen. **Silent Mode** will not play sounds, but will vibrate (if haptics are enabled), and allows notifications to appear on the screen. Critical alerts bypass both **Silent Mode** and **Do Not Disturb** (and other focus features) on your iPhone.

iPhone Vibration

Your iPhone has settings that can be adjusted to allow vibration notifications to be on.

In iPhone settings:

- Select **Sounds & Haptics**
- Select **Haptics**
- Select **Always Play**

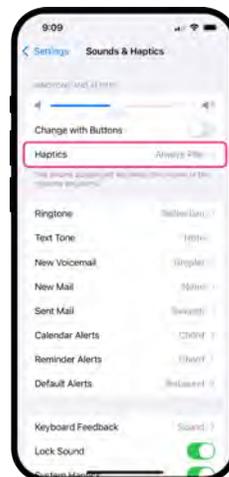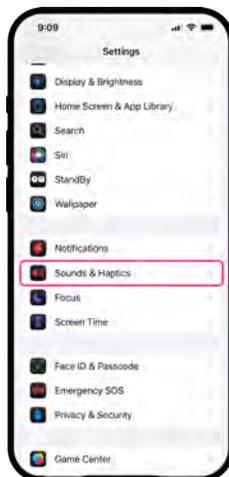

Screen appearance may vary across different iOS versions, including older and future versions

twiist App Alert Settings



There are two categories of app-based settings: **twiist-specific** and **CGM-specific**. Some alerts for app-based settings can be customized within the twiist App.

Adjustable twiist-specific alerts

Quiet Mode
Can be disabled.

Low Insulin Alert #2
Can be disabled.
Value can be adjusted.

Phone Out of Range
Can be disabled.
Value can be adjusted.

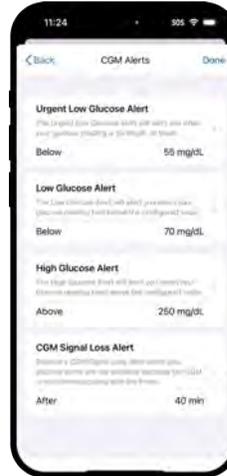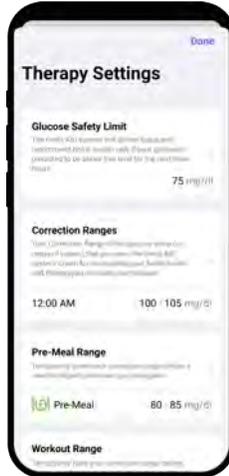

Adjustable CGM-specific alerts

Low Glucose Alert
Can be disabled.
Value can be adjusted.

High Glucose Alert
Can be disabled.
Value can be adjusted.

CGM Signal Loss Alert
Can be disabled.

Low Insulin Alert #1 cannot be disabled. Value cannot be adjusted. Value is set to 10 U.
Urgent Low Glucose Alert cannot be disabled. Value cannot be adjusted. Value is set at 55 mg/dL.

Quiet Mode Quiet Mode will not silence Alerts, Urgent Alerts, CGM Alerts, or Alarms.

CGM alerts have an override focus

When enabled, this feature allows a sound to be played when your iPhone is muted, or when the Focus feature is on.

When disabled, this feature prevents a sound from being played when your iPhone is muted, or when the Focus feature is on.

Override Focus



Turn on if you want this alert to always play a sound and appear on the lock screen even if your phone is muted or Focus is on.

Customer Care: 1 877-4TWIIST (1-877-489-4478)

The twiist™ Automated Insulin Delivery (AID) System is for people with type 1 diabetes, 6 years of age and above, available with a prescription, only in the U.S.

© 2025 Sequel Med Tech, LLC. All rights reserved. All other trademarks are the property of their respective owners. Medical disclaimer pending, to be provided by Sequel once approved.

Visit twiist.com/safety for important safety information.