



How to Change the Pump Battery


There are two ways to change the twiist Pump battery: **as part of a cassette change** or **separate from a cassette change**.

Option 1: Changing a pump battery as part of a cassette change


- 1 Open the **Pump Menu** by tapping the Pump Status icon  on the Home Screen.
- 2 Tap **Change Cassette** and follow the step-by-step instructions in the twiist App to complete the cassette change, including guidance to change the pump battery.
- 3 Insert a fully charged pump battery each time a cassette change is performed.



 A fully charged pump battery should power the pump for three days (72 hrs).

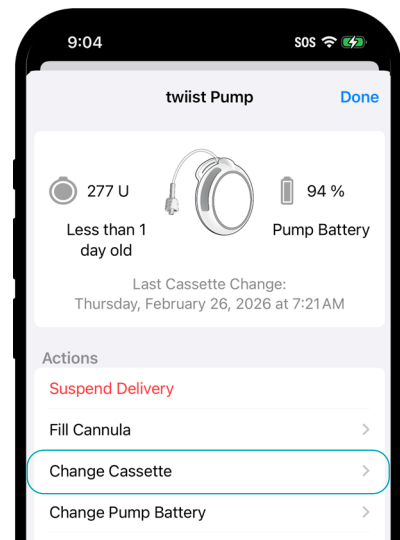
There are times you may need to change your pump battery before a cassette change.



Option 2: Changing a pump battery separate from a cassette change

- 1 Open the **Pump Menu** by tapping the Pump Status icon  on the Home Screen.
- 2 Tap **Change Pump Battery**.
- 3 Follow the on-screen instructions.

 If your battery is low, the pump battery portion of the Pump Status icon will be displayed in red  and a Pump Battery Low alert is displayed.

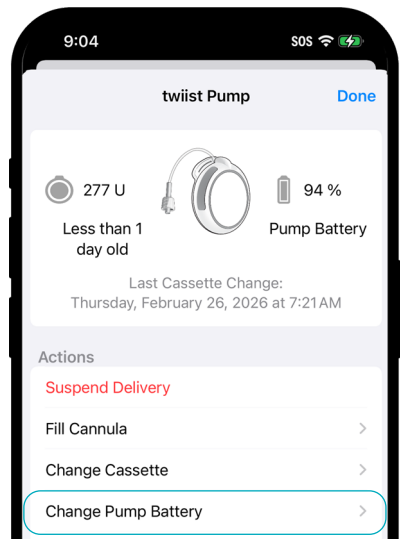

Customer Care: 1 877-4TWIIST (1-877-489-4478)

The twiist™ Automated Insulin Delivery (AID) System is for people with type 1 diabetes, 6 years of age and above, available with a prescription, only in the U.S.

© 2026 Sequel Med Tech, LLC. All rights reserved. All other trademarks are the property of their respective owners. Visit twiist.com/safety for important safety information.