




Pre-Meal Preset

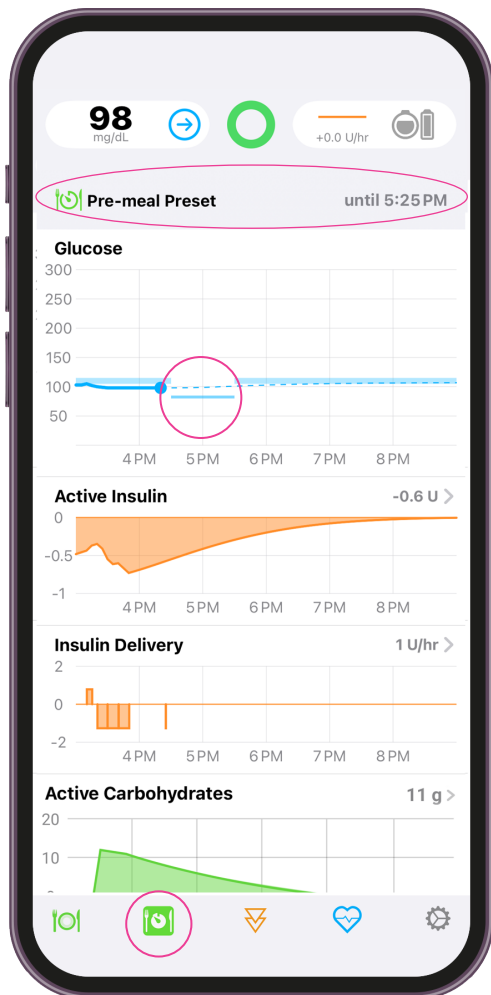

Circled sections highlight a lowered correction range, Temporary Status banner, and active Pre-Meal Preset.



The **Pre-Meal Preset**  can be used to temporarily lower your Correction Range before a meal. This can help reduce blood sugar spikes after eating.

When the **Pre-Meal Preset**  is active, the button color will be inverted, a Temporary Status banner will appear, and the Glucose Chart will update to show the change in the Correction Range.

The **Pre-Meal Preset** stays on for up to one hour, unless canceled or carbs are entered.

To cancel an active **Pre-Meal Preset**, select the  button.



Pre-Meal Preset is unavailable when Loop is off. The button will be grayed out and unable to be selected.

Customer Care: 1 877-4TWIIST (1-877-489-4478)

The twiist™ Automated Insulin Delivery (AID) System is for people with type 1 diabetes, 6 years of age and above, available with a prescription, only in the U.S.

© 2026 Sequel Med Tech, LLC. All rights reserved. All other trademarks are the property of their respective owners. Visit twiist.com/safety for important safety information.