


Workout Preset

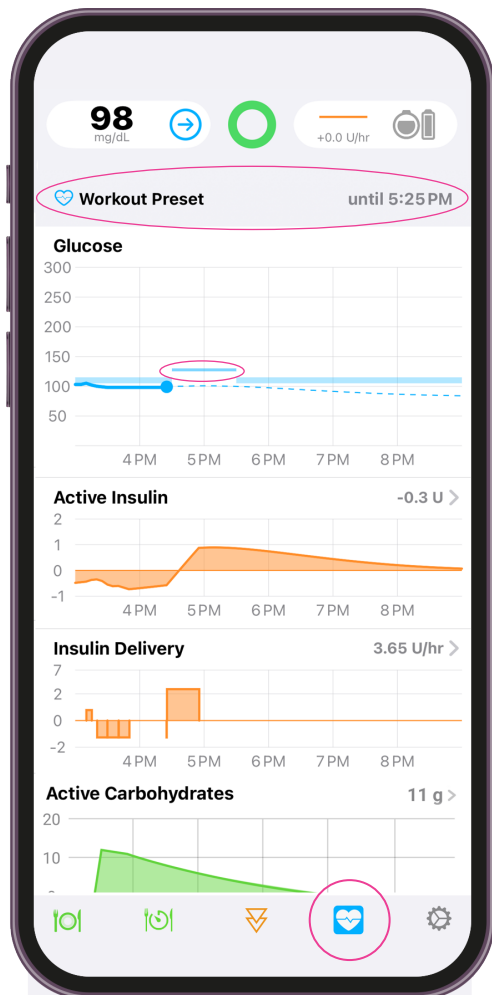

Circled sections highlight a raised correction range, Temporary Status banner, and active Workout Preset.



The **Workout Preset**  can be used to temporarily raise your Correction Range for activities.

When the **Workout Preset**  is active, the button color will be inverted, a Temporary Status banner will appear, and the Glucose Chart will update to show the change in the Correction Range.

The **Workout Preset** can be in effect for one of the following time options:

- one hour
- two hours, or
- until turned off

The twiist App will display a **Workout is Still Active** notice after 24 hours: *Workout has been turned on for more than 24 hours. Make sure you still want it enabled, or turn it off in the app.*

To cancel an active **Workout Preset**, select the  button.



Workout Preset is unavailable when Loop is off. When Loop is off the Workout Preset button is used to manually set a Temporary Basal.

Customer Care: 1 877-4TWIIST (1-877-489-4478)

The twiist™ Automated Insulin Delivery (AID) System is for people with type 1 diabetes, 6 years of age and above, available with a prescription, only in the U.S.

© 2026 Sequel Med Tech, LLC. All rights reserved. All other trademarks are the property of their respective owners. Visit twiist.com/safety for important safety information.