

How to Change a Cassette

The twiist™ AID System delivers insulin to help manage your blood sugar. Changing the cassette is an important part of using the twiist AID System.

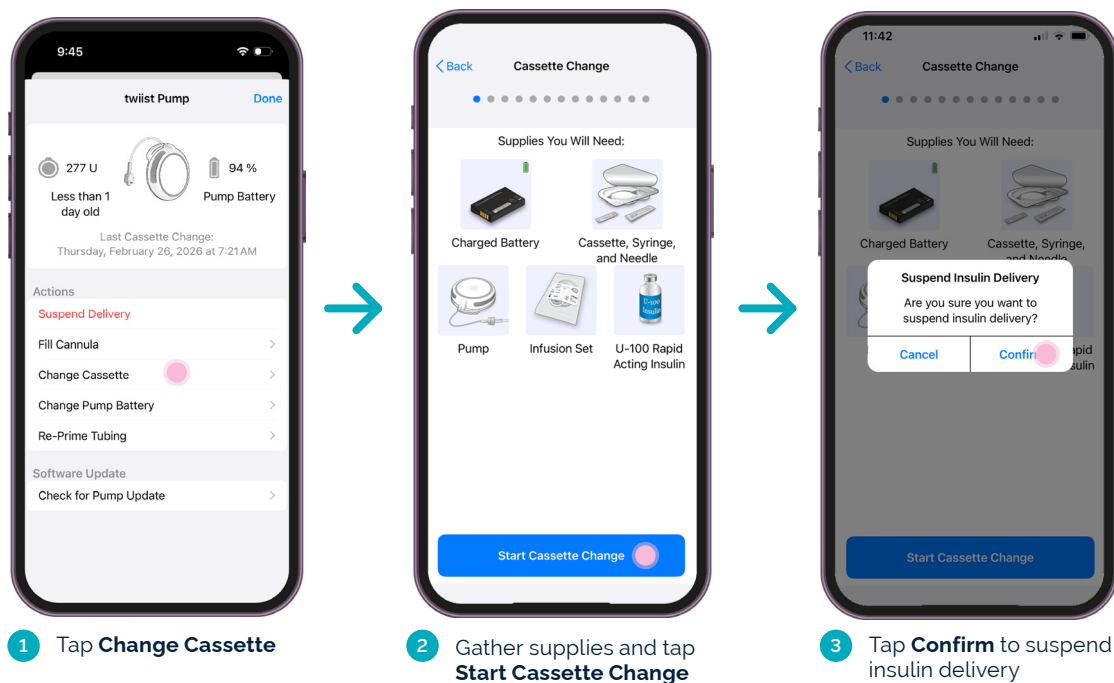
When to Change a Cassette

A cassette change will need to be completed every 72 hours (3 days) or when the cassette is empty, whichever occurs first.

What You'll Need

- | | | | |
|---|-----------------------|---|--|
|  | Fully charged battery |  | New infusion set |
|  | New cassette |  | Sterile 3 mL syringe with 27-gauge x 3/8 inch needle |
|  | Alcohol wipes |  | Humalog or NovoLog U-100 insulin |

To start the cassette change, tap the Pump Status icon  on the Home Screen.

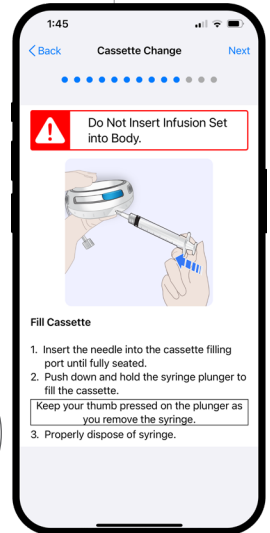
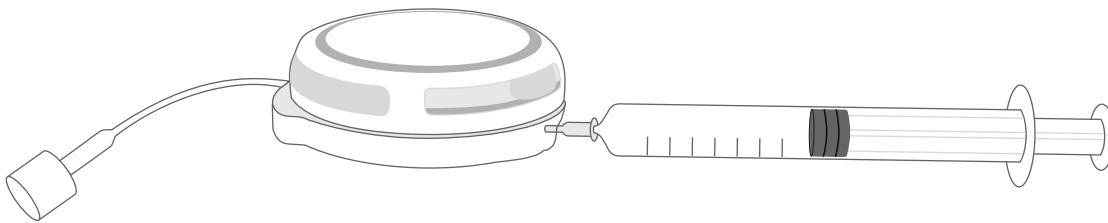


Follow the step-by-step instructions in the twiist App to complete the cassette change, cassette fill, and infusion set change.

Refer to the twiist User Guide as needed for detailed support on the twiist App prompts.
Refer to the Infusion Set User Guide for specific infusion set guidance.

IMPORTANT !

When filling the cassette with insulin, gently insert the needle parallel to the bottom surface of the cassette and straight into the filling port. Avoid inserting the needle at an angle. If you encounter strong resistance, remove and reinsert the needle.


WARNING

- Do not fill a cassette ahead of performing the cassette change process.
- Do not fill the cassette with insulin pen devices.

IMPORTANT !

- Always disconnect from your infusion site before starting a cassette change.
- Ensure you draw the amount of insulin equal to the Cassette Fill Volume selected in the twiist App, between 100-300 units. Avoid overfilling the syringe as it may result in damage to the cassette and/or pump..
- Add insulin to your cassette only one time during the cassette fill process. Do not add insulin to a previously filled cassette as it increases the risk of overfilling the cassette and damaging your pump.

Customer Care: 1 877-4TWIIST (1-877-489-4478)

The twiist™ Automated Insulin Delivery (AID) System is for people with type 1 diabetes, 6 years of age and above, available with a prescription, only in the U.S.

© 2026 Sequel Med Tech, LLC. All rights reserved. All other trademarks are the property of their respective owners. Visit twiist.com/safety for important safety information.