



Suspend and Resume Insulin Delivery


There may be times when you want or need to suspend insulin delivery. You can suspend and resume insulin delivery through the twiist™ App with just a few taps.

How to Suspend Insulin Delivery

- 1 Tap the **Pump Status** icon  on the Home Screen to open the Pump Menu.
- 2 Tap **Suspend Delivery**.
- 3 Tap **Confirm** to suspend insulin delivery.

The pump will play four descending tones.

The pump sound will not be played when Quiet Mode is enabled.

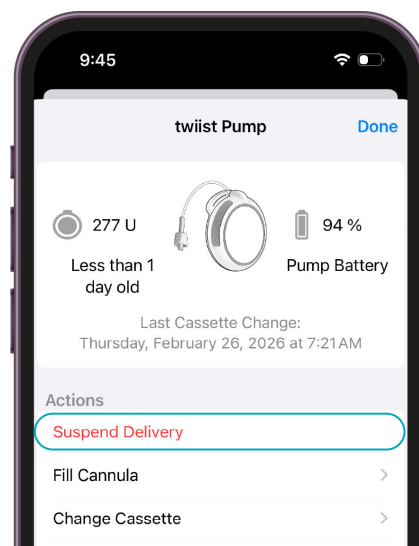


Resume Insulin Delivery from the twiist App

- 1 Tap the **Temporary Status** Banner or tap the Insulin Suspended icon.
- 2 Tap **Resume Delivery**.

The Home Screen will display a Communicating with Pump message and the pump will play three ascending tones.

The pump sound will not be played when Quiet Mode is enabled.

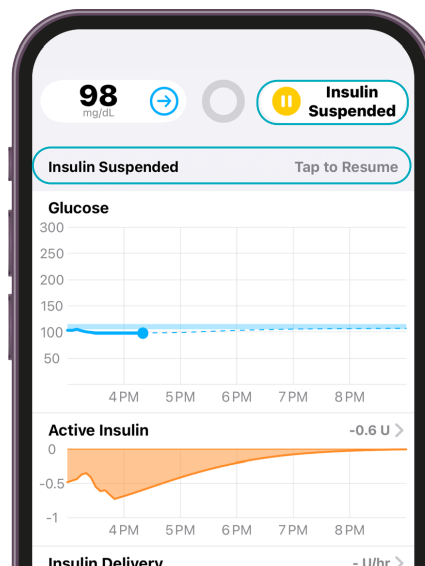

Customer Care: 1 877-4TWIIST (1-877-489-4478)

The twiist™ Automated Insulin Delivery (AID) System is for people with type 1 diabetes, 6 years of age and above, available with a prescription, only in the U.S.

© 2026 Sequel Med Tech, LLC. All rights reserved. All other trademarks are the property of their respective owners. Visit twiist.com/safety for important safety information.