DE-500 DERMATOSCOPE - Instructions For Use. Intended Use: Medical microscope for skin lesion diagnosis via contact/non-contact examination. Connects to smartphones/tablets for image capture by trained medical staff on patients with skin lesions. The smartphone or tablet used with this device is not a medical device and is not part of the medical device. **Economic Operators:** Manufacturer: Shenzhen Iboolo Optics Co., Ltd. (B1-406 Mingyou Purchasing Center, Xixiang St., Bao'an District, Shenzhen, Guangdong, 518102 China). SRN: CN-MF-000029373. Authorised Representative (EAR): Kingsmead Service B.V. (Zonnehof 36, 2632 BE, Nootdorp, Netherlands, office@kingsmead-service.com). SRN: NL-AR-000002066. Importer: Melatech ApS (Trekronergade 126G, 2500 Copenhagen, Denmark).

Contraindications: None known. **Warnings:** Use only 5V ≤30W chargers. Do not disassemble/damage battery. Avoid direct eye exposure to LED. Potential skin irritation/allergic reaction in individuals sensitive to Aluminum, Glass. **Precautions:** Read IFU before use. Charge fully before first use. Clean contact plate before each patient. Avoid dropping, excessive force, liquid ingress. Discontinue use if malfunction/damage. **Potential Risks:** Rare mild skin irritation. Misdiagnosis with incorrect use/training. Cross-contamination from improper cleaning. **Overview:** Device components include: Universal Lens Mount, On/Off Button, Battery/Charging Indicator. Features etched millimetre ruler. **Contents:** Dermatoscope, universal/metal mounts, USB-C cable, case, cloth, this manual. **Technical Data:** Materials: Aluminum, Glass. Magnification: 10X. Distortion: ≤5%. LEDs: 15 (10 polarized, 5 non-polarized). Resolution: 300/250 LP/MM (Axis/Edge). Battery: 230mAh Li-ion. Charging: 5V USB. No IP Rating. UDI-DI: 06923299000329. The UDI-DI is provided on the device label. **Operation:** Connect to smart device via mount. Press the red buttons for on. Press again to turn it off. It will turn off automatically after 3 minutes. Use ruler for size estimation. **Cleaning & Maintenance:** Contact plate: Clean before each use with lint-free cloth. Disinfect after each patient with 70% ethyl alcohol. Body/lens: Clean with soft, slightly damp cloth. No user servicing beyond cleaning/charging. **Lifespan:** Expected lifespan min. 2 years with normal use/charging. End-of-life indicated by: inability to hold charge, significant battery degradation, physical damage, or persistent malfunction. **Disposal:** Dispose of device at e-waste facility. Battery: If designed to be user-removable and it is safe to do so, remove and recycle according to local battery disposal regulations. Otherwise, dispose of the entire device at an e-waste facility. **Report Serious Incidents:** Report any serious incident to the Manufacturer (via Importer/EAR) AND the Danish Medicines Agency (Lægemiddelstyrelsen). Importer: Melatech ApS, Trekronergade 126G, 2500 Copenhagen, Denmark, support@melatech.io.

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