




Avive[®] Portable Power Bank



Get to know the Avive Portable Power Bank

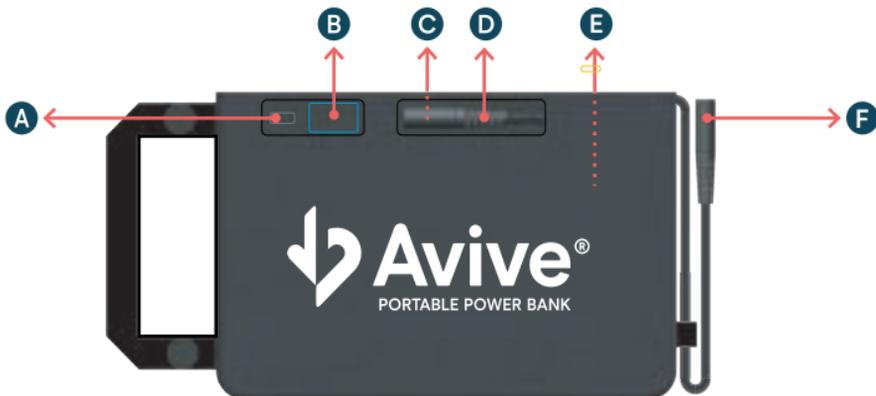

- A Power Bank button** turns on the Display Screen on the Avive Power Bank.
- B Power Bank display screen** shows the charge level for the Power Bank. Will never show the battery percentage of the AED.
- C USB-C port** used to charge the Power Bank and for the Power Bank cable.
- D Power Bank cable** for connecting the Avive Power Bank to the Avive Connect AED®.
- E Avive Power Bank** for extending the standby time for the Avive Connect AED®.
- F AED charging cable** for connecting the Avive Connect AED® to the Avive Power Bank.

Set up the Avive Portable Power Bank

Let's get started!



Scan for info!


Step 1: Set up the Avive Portable Power Bank

A. Verify the Power Bank's battery level by pressing the Power Bank Button (see page 1 for location). If the battery is not fully charged, follow the instructions on page 5 to charge the Power Bank.

B. Fully Charge the Avive Connect AED® before connecting it to the Power Bank. Use an Avive USB Charging Cable, an Avive USB Power Adapter, and a compatible wall outlet to charge.



C. Connect the AED USB Charging cable on the Avive Portable Power Bank to the Avive Connect AED®. →

Important: The Power Bank cable **MUST** be plugged into the Avive Portable Power Bank for the Power Bank to function as intended.

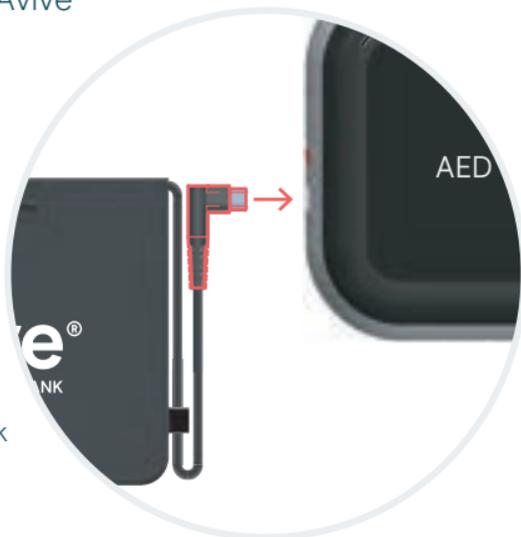


Step 2: Turn on Power Bank Mode

A. If using the Avive Connect AED®, navigate to Settings > Power on the device screen to view device power options.

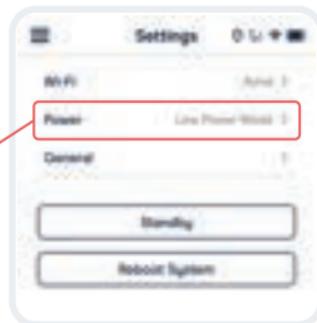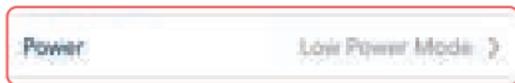

B. Toggle “Power Bank Mode” **ON**.

NOTE: This will automatically configure your Avive Connect AED® to maximize Power Case standby time.

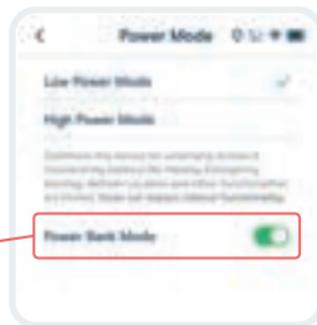

Done!

The Avive Portable Power Bank will automatically turn on to extend the standby time of your Avive Connect AED®.

Recharging the Avive Portable Power Bank

A. When it's time to recharge the Avive Portable Power Bank, first unplug the Power Bank Cable from the USB-C port on the Power Bank. →

B. Connect the USB-C end of a compatible charging cable to the USB-C port on the Power Bank. Connect the other end of the cable to a compatible adapter and wall outlet and let charge for 24 hrs or until the battery level reaches 100%.



← **C.** Once the Power Bank is fully charged, unplug the cable and reconnect the Power Bank Cable into the USB-C port on the Power Bank.

Important: The Power Bank Cable **MUST** be plugged into the Avive Portable Power Bank so that the Power Bank can charge your device.



Operating & storage conditions

Temperature 32°F – 95°F (0°C - 35°C)

Humidity 5 - 95% non-condensing

Altitude < 9,842 ft. (3,000 m)

Water & Ingress Do not expose the Avive Power Case to any liquids.



Keep away from liquids



Do not drop



Do not expose to extreme temperatures



Keep away from direct sunlight



Do not disassemble



Keep away from fire

Avive AED and Power Bank standby time: Up to 5 years

USB-C output: 5V/3A

Battery rated capacity: 42000mAh (5V/3A)

USB-C input: 5V/2A

MADE IN CHINA





Avive Solutions, Inc.
185 Valley Drive Brisbane, CA, 94005 USA
www.avive.life | 1 (800) 489-4428
Patents: www.avive.life/patents

© 2024 Avive Solutions, Inc. All Rights Reserved.