




# Avive® Training Cartridge Installation Guide

CAUTION: This Training Cartridge is for training only and can not deliver therapy. It is recommended that the Avive AED™ is stored with the Avive Pad Cartridge.

## 1 Training Resources


Installing a Training Cartridge switches the Avive AED to Training Mode, which allows a user to practice using the AED in a training environment. The Avive AED provides audio instructions as if it were being used in an emergency, however, the device will not actually analyze a heart rhythm or deliver defibrillation therapy. For more information, visit [www.avive.life/AviveAED/Training](http://www.avive.life/AviveAED/Training)

## 2 Remove the Pad Cartridge

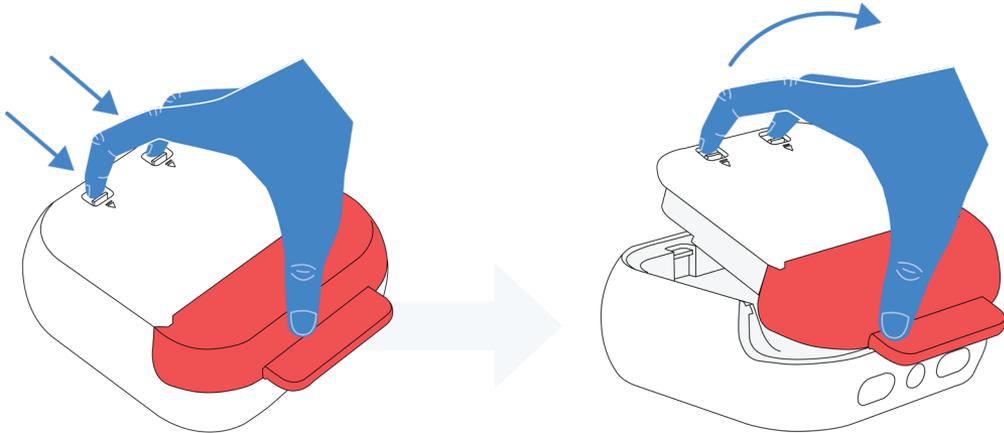

1. Firmly pull back and lift up on both snaps on the back of the device to release the cartridge.

### 3 Install a Training Cartridge

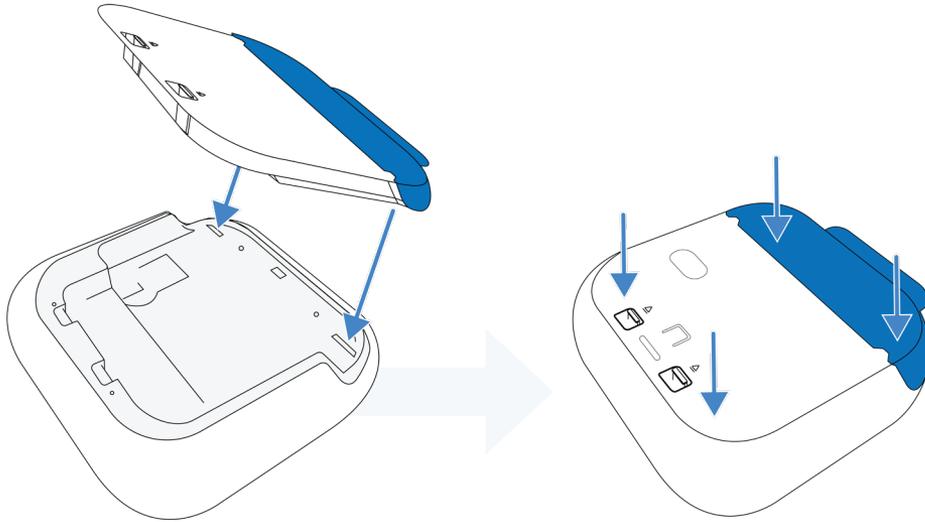

**1.** Install a Training Cartridge by first aligning the cartridge with the back of the AED. Push the cartridge firmly into the recess on the back of the AED.

**2. IMPORTANT:** Press down firmly on the corners of the cartridge. Make sure the cartridge is fully inserted around all sides of the AED.

Once training is complete, install an Avive Pad Cartridge so the device is ready for emergency use.



avive.life  
(800) 489-4428

Avive Solutions, Inc.  
185 Valley Drive  
Brisbane, CA 94005 USA

©2023 Avive Solutions, Inc. All rights reserved. The Heart and Arrow Logo, Heart and Arrow Logo AED, Avive, Avive AED, Avive Connect AED, REALConnect, REALConnect Platform, QuickRescue, Intelligent Response, Rapid Incident Reporting, LIFESaver, Avive 4 Minute City, Incident Data Transfer, AllRescue Technology, EverCharge Battery, are trademarks of Avive Solutions, Inc. Other marks are the property of their respective owners.