


Alto Holter Monitor

What do the lights mean?

Only Green


Green light will flash twice every 2 seconds to indicate the device is receiving a good signal and the test is underway.

After a few minutes, the green light will flash once every 2 seconds when recording starts. It will continue to flash for the test duration.

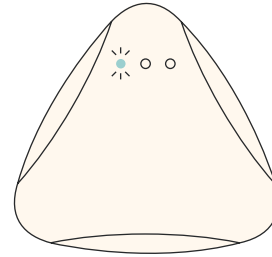


Only Yellow

Yellow flashing light means the battery is low. The device will still record normally in this mode.

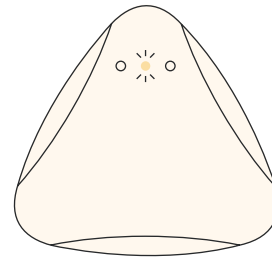


Green / Yellow / Red

A series of green, yellow, and red will flash when you double-tap the device to record a symptomatic event.

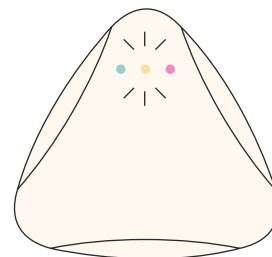

Only Red

When the red light is permanently on, please contact your doctor/clinic for support.

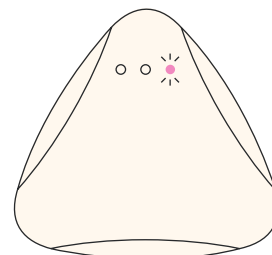

Need help?

Contact your local support team by visiting cardioscan.co/contact