


Electrode & Device Replacement

5 Step Process

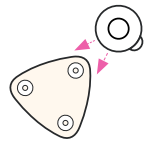


1. Remove Device

Gently detach the device.



2. Remove Used Electrodes

Peel off and dispose old electrodes.

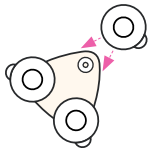


3. Attach New Electrodes

Connect new electrodes to the device.



4. Peel Backing

Remove the protective backing once attached.

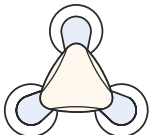

5. Reapply Device

Place and press onto your chest in the same area.




Scan the QR code & watch:
'Using the device during a test'.

What do the lights mean?


Green Light

1 flash every 2 seconds: **Recording.**

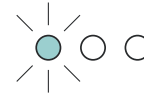

Red Light


One flash every 2 seconds notifies that recording period has finished.

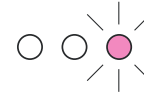

Permanently on notifies of device error.


Green | Yellow

One flash every 2 seconds notifies that device battery level is low.

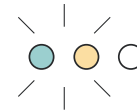


Green | Yellow | Red

When you double-tap to record an event.



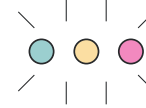


Alternative Electrode Positions

Male

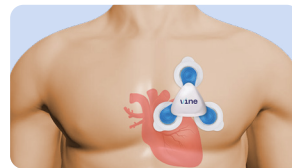

Female

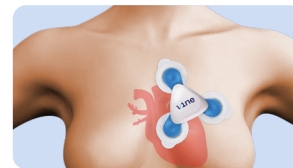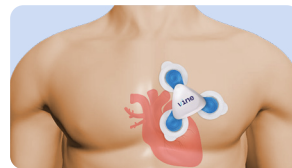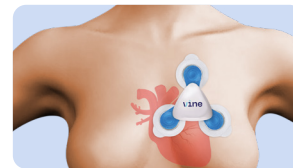

Replace the electrodes yourself if itching or skin irritation occurs. They can be replaced individually or all three at once.

Replace electrodes after showering when the skin is dry. At the end of the recording, detach the Holter by pulling it off the snap connectors.

← Fold



Vine Holter Test Patient Diary

Patient name

Patient ID (if applicable)

Date of Birth

Gender

Male

Female

Pacemaker

Yes

No

Hospital

Specialty

Referring Dr.

Duration

Date & Time Fitted

Date & Time Removed

Technician

AU_A082

