



A BREAKTHROUGH WOUND CARE SOLUTION THAT STARTS AND ENDS WITH THE PATIENT.

WHAT IS AURIX?

Aurix[®] is a biodynamic hemogel that harnesses a patient's innate healing mechanisms in order to jumpstart the natural healing process within the wound bed.

HOW IT WORKS

The Aurix gel is derived from a patient's own platelets and plasma. Once applied, Aurix goes to work, stimulating the formation of tissue to fill in wound defects and help reduce wound volume.

INDICATION

Aurix is a point-of-care solution that seeks to deliver better outcomes to a wide array of patients. It is the only therapy of its kind that is FDA-cleared for use on ulcers and wounds of all types.

Aurix is indicated for:

- ✓ Exuding wounds: venous leg ulcers, pressure injuries, diabetic foot ulcers
- ✓ The management of mechanically or surgically debrided wounds.

REIMBURSEMENT

AURIX is reimbursed under the National Coverage Determination (NCD) issued by CMS on April 13, 2021, which states: *The Centers for Medicare & Medicaid Services (CMS) will cover autologous platelet-rich plasma (PRP) for the treatment of chronic non-healing diabetic wounds under section 1862(a)(1)(A) of the Social Security Act (the Act) for a duration of 20 weeks, when prepared by devices whose FDA cleared indications include the management of exuding cutaneous wounds, such as diabetic ulcers. Coverage of autologous PRP for the treatment of chronic non-healing diabetic wounds beyond 20 weeks will be determined by local Medicare Administrative Contractors (MACs). For more detailed reimbursement information including covered ICD-10 codes please see the AURIX Reimbursement Guide.*

IMPORTANT SAFETY INFORMATION

Aurix is produced with the use of bovine thrombin. Aurix is contraindicated in patients sensitive to components and/or materials of bovine origin.


TREATMENT PROCESS

The Aurix System[®] provides the safe and rapid preparation of biodynamic hemogel from a small sample of a patient's own blood. Product preparation and application can be completed in five minutes after blood draw, and breaks down into four easy steps.

- 1


DRAW IT

Take a small amount of the patient's blood.


- 2


SPIN IT

Use the Aurix System centrifuge to separate the platelets (1 minute).


- 3


MIX IT

Mix with additives supplied in the Aurix Reagent Kit (such as ascorbic acid) to produce a golden biodynamic gel.


- 4

APPLY IT

Apply over the entire wound bed and cover with a clear dressing.




AVERAGE PREPARATION TIME: 5 MINUTES*

**after blood draw*

The Aurix System is produced and distributed by Nuo Therapeutics, a biomedical company that pioneers treatment solutions in regenerative therapies. **For more clinical or patient information about Aurix, visit: www.AurixSystems.com**

Nuo Therapeutics, Inc.
 8285 El Rio Street, Suite 190, Houston, TX 77054
Phone 1.833.298.6633 | **Email** info@nuot.com
Website www.nuot.com

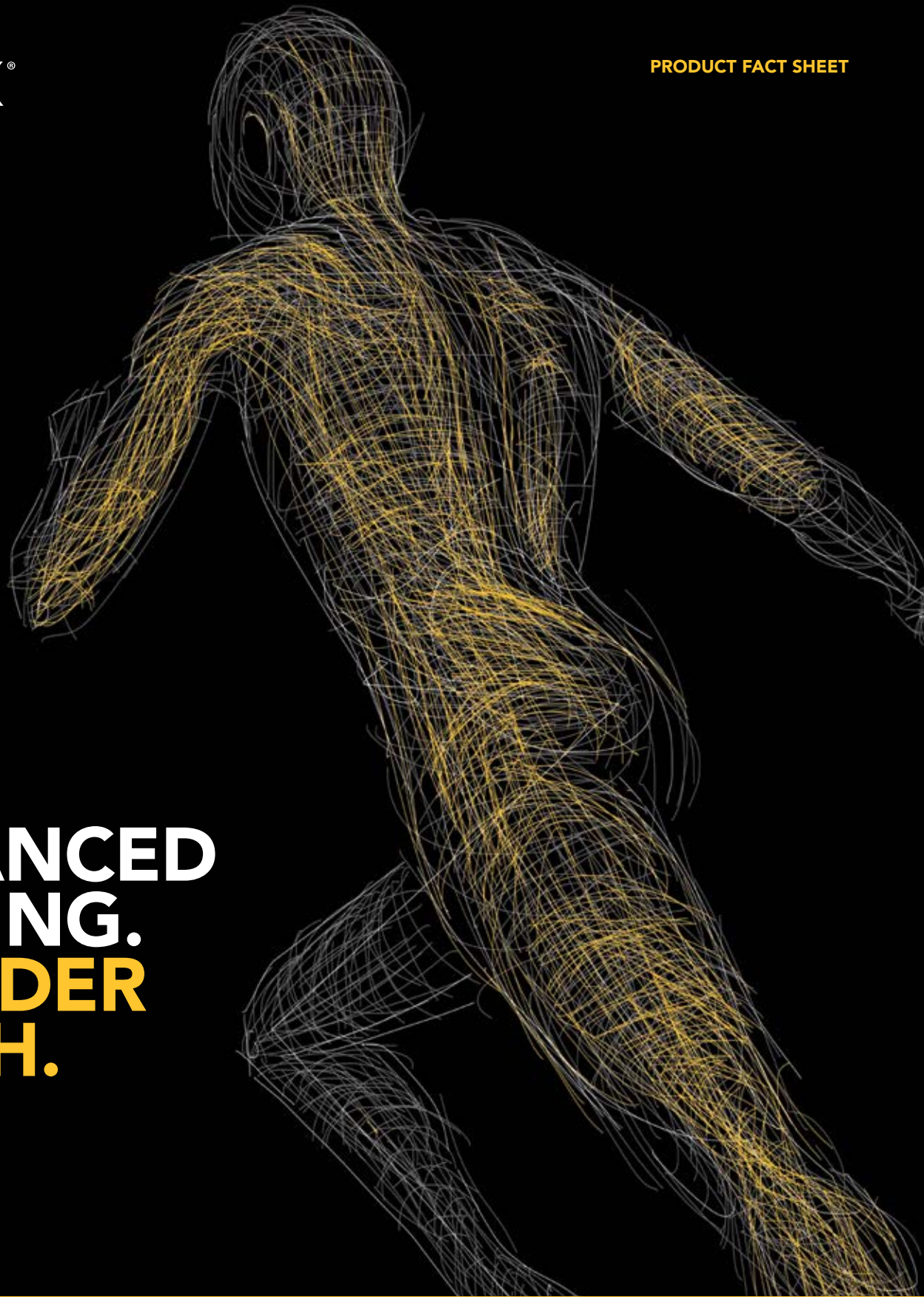

ENHANCED HEALING. BROADER REACH.



AURIX: HEAL ON.

Aurix[®] is a breakthrough wound care solution that starts and ends with the patient.

By harnessing the body's innate healing mechanisms, this biodynamic hematology helps providers change the trajectory of healing for a broader range of patients. Because of its individualized nature, Aurix offers providers more flexibility at all stages of healing.