
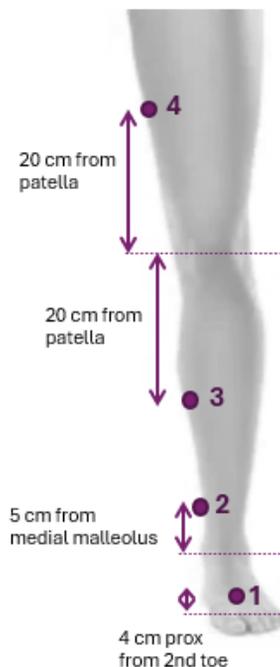


Measurement protocol for lymphedema in leg - LymphScanner

Subject's name or id _____ Date _____

MEASUREMENT INSTRUCTIONS

- Select a reference site on volar forearm
- Measure the reference site with the LymphScanner and record the reading from the device's display to the table below
- Use the same site on the forearm as a reference for both legs
- Measure sites 1 – 3
- Record readings of all sites from the device's display to the table below
- Measure corresponding sites of the other leg and record reading to the table

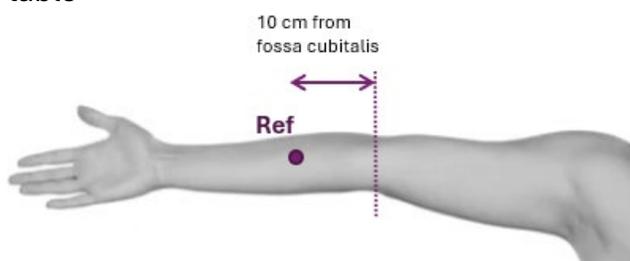

Site	Forearm (Ref)	Left leg (PWC)	Ratio = Left leg (PWC)/ Forearm (Ref)	Right leg (PWC)	Ratio = Right leg (PWC)/ Forearm (Ref)
1					
2					
3					
4					

INTERPRETATION

Ratio is the quotient of the measurement values of the leg and the forearm.
The LymphScanner calculates the ratio automatically when used in Scan mode.

Ratio ≥ 1.35 indicates lymphedema.

Example Forearm: 33 %

Leg: 48 %

Ratio: 48 % / 33 % = 1.45