Lansinoh®

EN

SOOTHIES® COOLING GEL PADS

Gel pads soothe on contact and protect sensitive nipple area to support breastfeeding success for nursing women.

Contraindications: There are no known contraindications.

Caution: Use only as directed. Keep out of reach of children. For external use only. Should sensitivity or irritation occur discontinue use and seek medical advice. Total usage of gel pad therapy should not exceed 3 weeks. If condition persists, discontinue use and seek the advice of a breastfeeding specialist or healthcare provider.

Directions:

- Open pack & remove gel pad from reusable tray

- Gently place the gel side of the pad directly over nipple to clean dry skin

- Before breastfeeding or pumping, remove gel pad. Inspect breast for residue and gently cleanse if necessary

- After breastfeeding or pumping, reapply gel pad over nipple

- May be used for up to 72 hours or until fully saturated. If pad becomes spongy or less tacky, it is likely saturated and should be replaced

- In between use, store gel pads on reusable tray, gel side down and return to the outer pouch

- For additional cooling relief, store gel pads in refrigerator. For sustained cooling effect, apply until the product warms to body temperature. Remove, allow to cool and reapply.

- Do not rinse gel pads with water

- Do not use pads with nipple creams or other topical remedies

If gel pad appears milky or cloudy, dispose of the pad and apply a new one.

This product is a medical device and should be disposed as a medical product according to the local procedures and regulations.

Serious events experienced from using this product should be reported to the manufacturer and to the local competent Authority.

Store at ambient temperature

The Soothies Gel pads are intended for single patient use. Re-use outside of the maximum permitted timeframe or between individuals may lead to increased risk of infection or cross contamination and the physical properties of the device may no longer be optimal for intended use.

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