

Enroll Patient's Holter Procedure

BeatBox User Guide – myPatch®sl

BeatBox User Guide – myPatch®sl

Enroll Patient's Holter Procedure

Log in to BeatBox and snap on a Holter device

1. Go to beatbox.cloud and log in to BeatBox.
2. Check that the Holter device is plugged into your computer:
 - attach the Holter device to the docking station, then insert the docking station cable into the USB port on your computer.
 - You will see blue flashing light coming from device to indicate connection to computer

Entering the patient details

1. Click **Enroll**.
2. Select the patient's lab then click Create patient.

Add/confirm patient demographics and contact details

3. Add the patient's demographic details (**First name**, **Last name**, **Patient ID**, **Gender** and **Date of birth** are mandatory).
4. Add patient's contact details, click **Next**.

Add patient Insurance details

5. Add the patient's Insurance card details
6. Click Next.

Add procedure details

7. "Procedure type" will remain **Holter**
8. "Choose device" will remain **myPatch®sl**
9. Select **Hookup date** and **Hookup time**
10. "Electrode" will remain Standard
11. In the Channels field you will indicate how many channels you wish to record.

Note: If you do not use the extra lead wire then you will select Channels 1 and 2. If you do use the extra lead wire then you will select Channels 1, 2 and 3

12. Select the sample rate and the recording duration.

- 1-9 days: Sample Rate of 256
 - 10-14 days: Sample Rate of 128
-

13. Select **Tests ordered by Physician**.

Add patient medical details

14. Click to select the indications that the patient presented with.

Note: Use the Search option at the top of the **Indications** list to either narrow your search or find a specific indication.

15. Select Ordering Clinician (if relevant).

Note: If you can't find the Ordering Clinician in the list, click **Add**. Once you have saved their details they will also be added to the selection list.

16. Click Save.

The patient's Holter procedure is added to BeatBox.

You can either enroll the patient on the Holter device now, or later via your Worklist (see [Enroll patient on Holter device](#)).