

View and Download Holter Procedure

BeatBox User Guide


BeatBox User Guide

View and Download Holter Procedure


Select patient

1. Go to beatbox.cloud and log in
 2. Search for patient/procedure in worklist
 3. Select Review/Report under actions column to be taken to reporting module
 - (a) Review for non-doctor level users
 - (b) Report for Doctor level
-

4. To download the Analyzed report, select 'Open in a new tab'

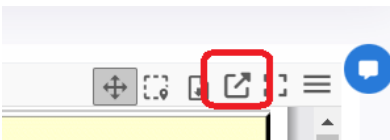

- (a) To download a finalized report, follow same steps or select 3 dots in worklist under actions column and select to download report

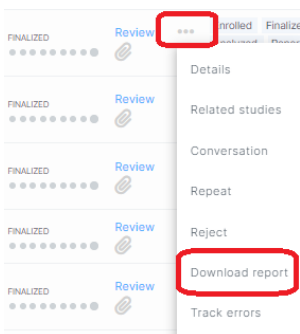

-
5. Report will open in new tab in PDF format.
 6. Download/Print/Save as PDF
 7. Close tab and return to worklist

Interpret/Finalize a Holter Procedure

8. Go to beatbox.cloud and log in

9. Search for patient/procedure in worklist

10. Select Report under actions column to be taken to reporting module

11. The procedure's PDF report is displayed to the right of the *Report procedure* page. You can:

- (b) scroll to view all pages and zoom in or out as required
- (c) download or print the report
- (d) choose a two-page view
- (e) view a list of all pages and select the relevant page to view

12. Enter notes/comments into Conclusion field

Demo Conclusion Testing

DOB: 01/01/2000 Age: 24 yrs Gender: O Patient ID: 123

Holter

Rec. duration

3 days

Recorded

11/07/2023 08:33:00 AM

Analyzed

11/07/2023 03:48:19 PM

Status

Finalized

Finalized

12/11/2023 11:43:13 AM

Indications

R00.2: Palpitations

Medications

Not entered

Conclusion

1 Day Holter.
The average heart rate was 84. The minimum heart rate was 61. The maximum heart rate was 141.
Pauses greater than 2.00 seconds were 0.
Ventricular ectopy was 0, with 0 Isolated VE, 0 V-Pairs and 0 V-Runs.
Supraventricular ectopy was 0, with 0 Isolated SVE, 0 SV-Pairs 0 SV-Runs.
Atrial Fib Burden was 0.00%.
Patient pressed the Event Button 0 times.
CCT: ME

Assigned to User Testing

Support docs

Details

Related studies

13. Once finished, click Confirm button on bottom right hand of screen

Note: A procedure must be in either 'Analyzed' or 'Reviewed' status before it can be reported.

Note: You must have the relevant permissions to finalize a procedure. If you do have these permissions, the Confirm button will be disabled.

The PDF update process begins, and a front page is added to the final report with your signature. Once the report has been successfully updated, the procedure's status changes to 'Finalized' and a 'Finalized' workflow tag is added to the procedure. The procedure's finalized date is now displayed in your Worklist.