


## myPatch®sl Electrode Replacement

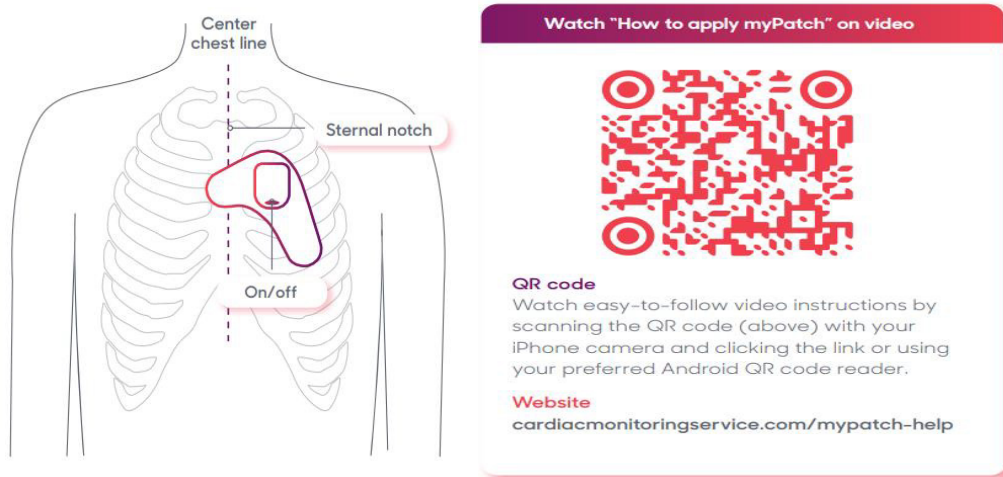

**Important: DO NOT press the power button. Please keep the device powered on.**

### Safely Remove Electrode

- Soak electrode with warm water on a paper towel for 1-2 minutes to help loosen adhesive.
- From the bottom, gently peel the electrode up and away from the skin, slowly.

### Prepare the skin

- Remove any chest hair where myPatch®sl will be placed.
- Gently clean the area, removing any remnants of glue, using the provided alcohol pads or with a mild soap and water
- Allow the area to air dry before placing electrode on.

### Prepare the device

- Remove the electrode Patch from the pouch (keep pouch)
- Secure the myPatch®sl recorder onto the four metal snaps on the electrode patch.

### Apply myPatch®sl Electrode (Use above diagram for reference)

- Peel the plastic cover off the back of the myPatch®sl electrode and place it on the body as shown above. Placing 2-3 fingers below sternal notch
- Smooth out all edges of the electrode, ensuring there are no creases or folds.

### Record time & date

- You must mark the time & date you placed the electrode in your symptom diary.