

# myPatch®-sl Holter Charging Instructions



**MAKE SURE** the myPatch®-sl Holter Recorder is fully charged. Connect the myPatch®-sl Recorder to the myPatch®-sl USB Transfer Cable. Do not force the myPatch®-sl ECG Recorder onto the connection of the cable, it will only snap on in one direction. Then connect the USB-end of the myPatch®-sl USB Transfer Cable to a powered on computer or provided wall charger.

The myPatch®-sl Recorder will automatically sense the cable and go into charge mode. The LED will flash **BLUE**, once it is fully charged the light will be solid **BLUE**. Allow recharge of the myPatch®-sl Recorder before use, approximately 1 ½ hours. You can charge your myPatch®-sl Recorder after you've uploaded a patient test, keep it connected to fully charge, once you see the solid **BLUE** light then you can store your myPatch®-sl Recorder until you have another patient to test, you will still charge your recorder but it won't take as long to reach the full charge.



**NOTE:** Once myPatch®-sl Recorder is fully charged, the charging will stop and it will go to trickle mode. This means myPatch®-sl is no longer being charged. If you have left myPatch®-sl charging over 24 hours, please remove myPatch®-sl from the charge and plug myPatch®-sl back into the charging source. myPatch®-sl may need to charge for another 10 minutes or so. Make sure you see a solid blue light prior to hooking up the patient.

## myPatch®-sl Holter Powering On Guide



**Turn on the myPatch®-sl Recorder** by holding down the power button for 4 seconds, you will then see a blinking **GREEN** light for about 30 seconds. Once the startup is done there will be no lights on the myPatch®-sl Recorder. **Your recording has begun** and will continue recording until the myPatch®-sl Recorder is powered off or at the end of the Recording Duration (set up during Enrollment).

You can choose to have the patient wear the monitor for up to 14 days (when you choose a 2 channel 128 Sample Rate during Enrollment).

### **To Record an Event:**



Tap twice in the middle of the myPatch®-sl ECG Recorder to mark an event during the recording period. A solid **BLUE** light will come on briefly to acknowledge the recording of the event.

## myPatch<sup>®</sup>-sl Light Guide



Blinks <b>BLUE</b>	When myPatch <sup>®</sup> -sl is charging
Solid <b>BLUE</b>	When myPatch <sup>®</sup> -sl is fully charged
Blinks <b>GREEN</b>	Start Up Mode (up to 30 seconds)
Continuously <b>RED</b>	Not Functioning, please contact CMS
Solid <b>YELLOW</b>	Low Battery
Brief <b>BLUE</b>	When recording an Event (2 seconds)
Brief <b>RED</b>	When powering off the myPatch <sup>®</sup> -sl

**\*\*Picture shown with no light illuminated**

## **Disconnect and Uploading Holter Recordings:**

**\*\*Please note; the myPatch-SL Holter monitor will power off automatically once study is completed. To ensure the device is powered off, double tap device as if to record an event. If the blue light flashes, this indicates the device is still powered on. In this case, follow the below instructions. \*\***

**\*\*If you do not see a blue light flash, this indicates the device has already powered itself off. In this case, immediately place the device onto transfer cable. \*\***



To end the recording, hold down the power button on the bottom of the myPatch®-sl Recorder for 4 seconds, you will see a **RED** light come on indicating that you are ending the recording. Then you will remove the myPatch®-sl Recorder from the Electrode and set aside.

The myPatch®-sl electrode is for **single-use and is disposable** and should be handled as normal waste and discarded in accordance to hospital or physician guidelines.



The myPatch®-sl Recorder is ready for processing in your Holter system.

Snap the myPatch®-sl Transfer Cable onto the myPatch®-sl Recorder. Plug the other end of the Transfer Cable into your computer. The myPatch®-sl is ready for processing.



**\*\*See BeatBox Guide – myPatch Uploading Data document for upload instructions\*\***