

Cleaning the myPatch®sl recorder

The myPatch®sl ECG Recorder shall be cleaned properly. For general cleaning, use a soft, lint-free cloth lightly moistened with a mild soap and water solution. Wipe and air dry. Use a clean, lint-free cloth.

- DO NOT use abrasive or highly alkaline cleaners
 - The recorder may be damaged by some common household solvents including Lysol, Pinesol, Isopropanol alcohol.
 - Benzene, gasoline, acetone, or carbon tetrachloride should never be used.
- DO NOT scrape with squeegees, razor blades, brushes or other sharp instruments.
 - The coating is UV-resistant, not mar-resistant

DO NOT clean products in hot sun or at elevated temperatures.

Compatible Cleaners & Cleaning Instructions

The following cleaning agents have been found to be compatible. The manufacturer's recommendations and instructions should be followed.

- Joy, Palmolive liquid, Top Job, VM&P grade Naphtha, Freon T.F., and Windex Ammonia Free
 - SaniCloth HB
 - SaniCloth Plus
 - SaniCloth Prime
 - CaviWipes1
-
- Wash with recommended mild soap and lukewarm water using a clean sponge or a soft lint-free cloth. Or use recommended wipes. Do not leave cleaning products on for periods of time; Rinse well immediately with clean water. Dry thoroughly with a chamois or moist cellulose sponge to prevent water spots.
-
- To decontaminate/disinfect the recorder, we recommend the unit be sprayed with a 50% Isopropyl Alcohol/50% water mix. Immediately rinse with clear, lukewarm water.