

AdaptAir Guide

NE7 Control Unit 

This guide is designed to help you with becoming familiar with the layout of the NE7 control unit, the program guide and safety features.



Tilt Sensor



Manual Programming



CPR Valve




Tilt Sensor

- To ensure the tilt sensor is working correctly, please ensure the cord is plugged in to the side panel of the pump/control unit.
- **Do Not Touch Tilt Sensor** - once the mattress system is on the correct programme, any changes made may cause the tilt-sensor to deactivate.

Care & Layout Guide

NE7 Layout

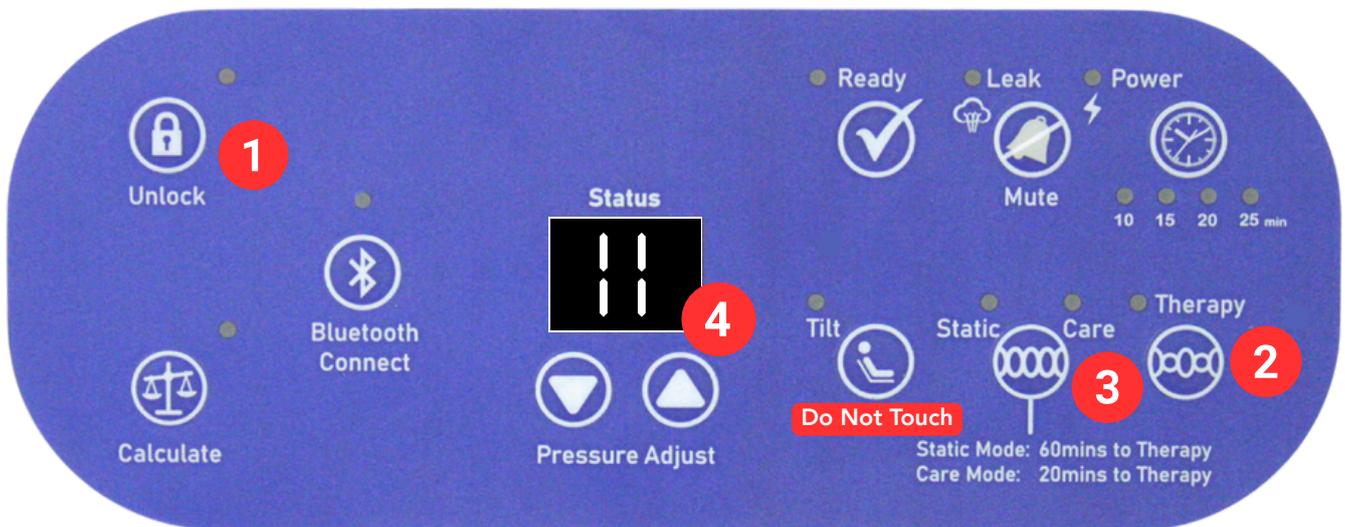

1 Unlock Button

- Press for 1 - 2 seconds, the control panel will be unlocked.

2 Therapy Mode

- After unlock, pressing 'Therapy' will engage air alternation programme.

3 Care/Static Mode

- Press **twice** to engage 'Care' mode.
- Care Mode allows max inflation of all air cells, and recommended for patient care and transfers.

(The pump will automatically return back to 'Therapy' after 20 minutes.)

- Press **once** to engage 'Static' Mode.
- This will cause air-cells to remain **static** for 60 minutes

4 Pressure Adjust

- Pressing '▲' will **increase** air cell pressure mmHg.
- Pressing '▼' will **decrease** air cell pressure mmHg.

Layout & Program Guide

NE7 Control Unit ✧

CPR

If **CPR** is needed whilst patient is using the mattress:

1. Press the **Red Button** located on the side panel of the control unit.
(This will allow the air in the cells to deflate rapidly.)
2. Disconnect air hose entirely by pressing down the **Blue Button** and detach by pulling firmly.
3. Call emergency services immediately.

See image for CPR Safety Attachments



36 Smallwood St,
Underwood QLD 4119

 Studio Care

Studio-Care.com.au
1800 822 830

© Studio Care Pty Ltd 2026

No copying or reproduction permitted without written authorisation.