



BrainCore Certification Program

Video Proficiency Assessment Outline

To complete your BrainCore Certification Program, please submit a recorded video demonstrating the following competencies. This ensures your proficiency with BrainCore's QEEG mapping and neurofeedback software setup.

This outline includes two pages. Be sure to **review both sections**—QEEG Mapping and Neurofeedback Training—for full submission requirements.

You may record this as one continuous video or submit multiple shorter videos for each section. Please ensure your screen recordings are clear and include audio narration when possible.

Please Note: It will be necessary for you to have a 2 participants. You will need one person to serve as the client for you to perform the brain map and training session on, and a second person to record your actions throughout the process. I.E. When locating sites, we need to see the top of the head and your hands as you locate.

QEEG Brain Mapping Proficiency

1

Cap Placement & Prep

- Demonstrate correct cap placement
- Prepare the electrode sites and ear clips properly (ex. gel or paste placement)
- Show ear clip placement and secure fit of the cap

2

Recording the QEEG

- Connect the client and initiate the brain mapping session
- Show real-time signal quality and ensure electrode impedance is within acceptable range
- Demonstrate a brief recording segment

3

Saving & Uploading

Screen record the process of:

- Naming the map file correctly
- Saving it in the appropriate format/location
- Uploading to the patient's file

Neurofeedback Training Proficiency

1

Site Selection

- Demonstrate method used to locate P3/P4
- Demonstrate method used to locate O1/O2

2

Session Setup and Signal Quality

- Demonstrate how to setup on T3/T4
- Demonstrate how you verify sensor signal quality and that frequencies are within an acceptable range

3

Live Training Session

Screen record 1 minute of a live training segment. Ensure the screen clearly shows:

- Client feedback animations/games running
- Frequency spectrum graph and EEG data
- Session timers and metrics

Submission Guidelines

1. Ensure all videos are clearly labeled if submitting separately (ex. cap placement, neurofeedback session, etc).
2. Upload to Google Drive: Place all videos into a single Google Drive folder.
 - a. Set sharing settings to: "Anyone with the link can view"
 - b. Copy the folder link
3. Log into your BrainCore Dashboard and navigate to:
 - a. My Training → Clinical Certification: Part 3 (Assessment / Live Hands-On) → Upload Video
 - b. Paste your Google Drive folder link in the URL field

Once your video proficiency assessment has been reviewed, you will receive an email notification with your results:

- If approved, you'll receive confirmation of your completion, followed by a separate email containing your official BrainCore Technical Certification, which you are welcome to print and frame.
- If not approved, we will email you with specific feedback outlining which portions need to be redone, so you have the opportunity to revise and resubmit those sections.