APPLICATION

viXion connect

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Insert the included USB Type-C cable into the main unit's charging port (A). Charging is complete when the power indicator lights up blue.

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
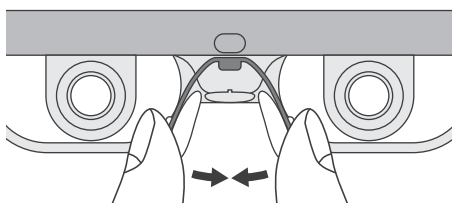
Gently bend the nose pads forward/backward and left/right to adjust the width for a comfortable fit.

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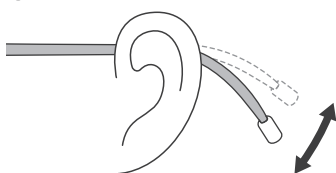
Gradually bend the temples to fit the shape of your ears. Do not bend them excessively, as this may impair the shape-memory properties.

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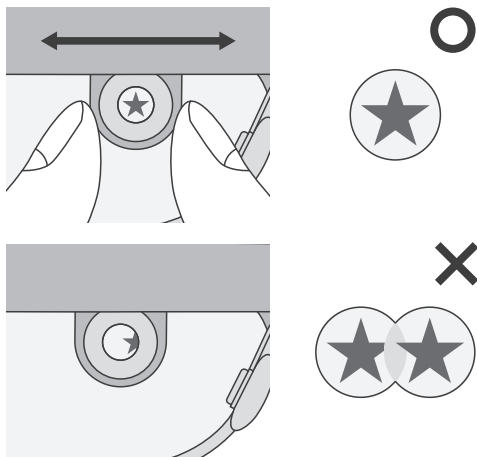Put on ViXion2 Pro and close one eye. While looking at a target object about 1 meter away, slide the focus lens left and right until the target appears centered within the black frame. Adjust the other lens in the same way. When the circular fields of view from both eyes overlap into one, as shown in the illustration on the left, the position adjustment is complete.

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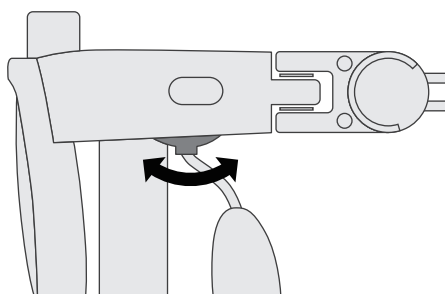
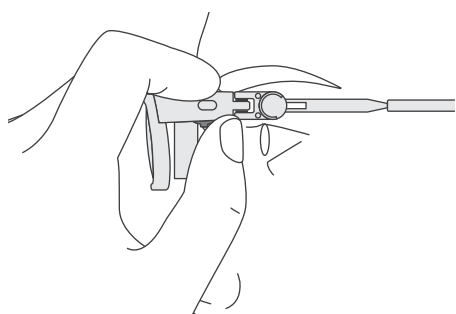
Close one eye and look at a target about 1m away. Keep tilting the lever forward or backward until the image is in clear focus. You will hear a beep while the lever is in use. Repeat for the other eye to complete the adjustment. Releasing the lever activates auto-focus mode and saves your settings for future use.

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

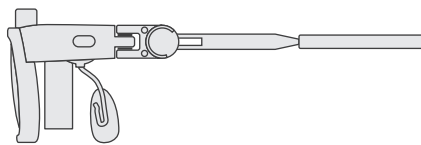
The temple tilt is adjustable up to 30°. This helps reduce strain on the neck and shoulders when looking downward for long periods.

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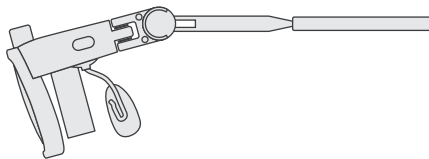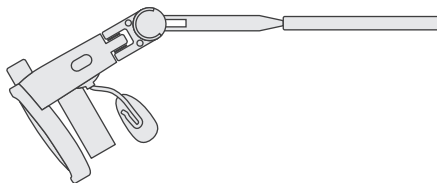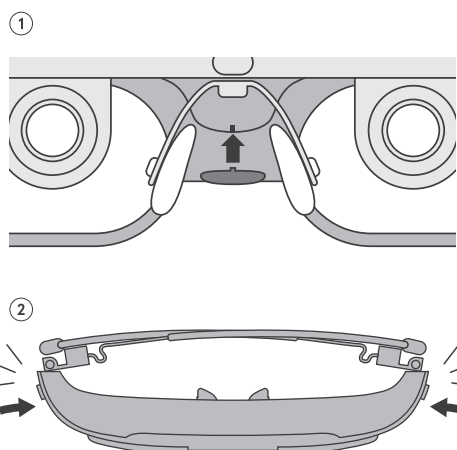
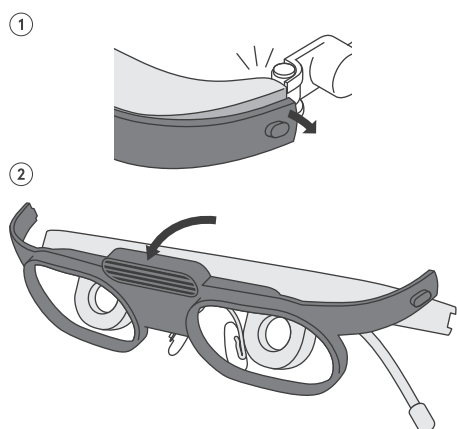
Fold the temples, then pinch the edge of the outer frame and pull it away from the main unit. Repeat on the other side.

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Tilting the outer frame downward, as shown in the illustration, makes it easier to remove.

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Align the protrusion at the bottom of the outer frame with the recess below the nose pads. Once centered, press the top of the outer frame straight into the main unit.

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Press each edge of the outer frame into the main unit until it clicks into place.

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Cautions

- ・ Do not drive a car, ride a bicycle, or operate any vehicle while wearing this product.
- ・ Do not use in situations involving fast movement or sports.
- ・ Do not use the product while it is charging.
- ・ Not waterproof. Do not splash with water or immerse in liquid.
- ・ Avoid using or storing in environments exceeding 50°C (122°F).
- ・ Remove immediately if you notice any abnormality with the product.
- ・ Stop use immediately if you feel unwell.
- ・ This product is not a medical device.

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