

#whsterilization



video.wh.com

Now at
your dealer
or **wh.com**



SPEEDS YOU UP



lisa^{MINI}

Speed up workflow

Lisa Mini speeds up your daily workflow. The device impresses with incredible fast cycles and its flexible usability.

The compact dimensions guarantee an optimal integration into the reprocessing room. Fully equipped, Lisa Mini also offers the peace of mind of a type B sterilizer such as our Lisa, Lara or Lina sterilizers.



Super fast

Instrument reprocessing
with surprisingly short
cycles

.....

Workflow optimisation

Speed up cycle times

Choose whatever you need. With Lisa Mini all load types can be sterilized. Thanks to the different cycles, it is possible to reprocess any instrument, wrapped or unwrapped, with extremely short cycle times. In addition, the sterilizer offers one of the fastest type B cycles in its performance class.

Unbeatable cycle times

Thanks to the optimised cycle profile, wrapped instruments like handpieces can be reprocessed in less than 19 minutes. Unwrapped loads can be reprocessed in less than 10 minutes.

Reprocessing loads



60"

Wipe
disinfection
1 minute



10"

Lubrication
10 seconds



17' 45"

Speed up flexibility

Take advantage of Lisa Mini's flexibility. Thanks to its small dimensions, Lisa Mini can be easily positioned in the reprocessing room. Moreover, it offers the possibility to stack items on top of each other. Thus, Lisa Mini offers a wide range of applications, e.g., as a complete device for the fast reprocessing of instruments, e.g. handpieces, and as an emergency back-up of tabletop sterilizers.



Speed up traceability

The technical expertise makes the difference. The Lisa Mini sterilizer is based on a sophisticated technical concept that offers future-oriented possibilities and significantly simplifies your workflow.

Speed up with Activation Codes

Needs and requirements may change. Prepare yourself now: With the Activation Code system, additional functions are activated to meet your changing needs and future requirements.




Enhanced traceability

- › The documentation is automatically saved via a high-capacity USB stick.
- › The barcode label printer provides additional support on traceability thanks to the opportunity for linking cycle, sterilization package and patient.
- › The cycle report printer ensures adequate paper documentation of the cycle.
- › Wi-Fi and LAN connections offer the possibility to download cycle reports directly to any PC or smart



Speed up daily tasks

Not only small and fast, but also fully equipped. The Lisa Mini touch screen with intuitive menus offers a wide range of options to meet all your needs. The automatic water filling makes the handling convenient. High-quality components and innovative technology ensure a reduced maintenance.

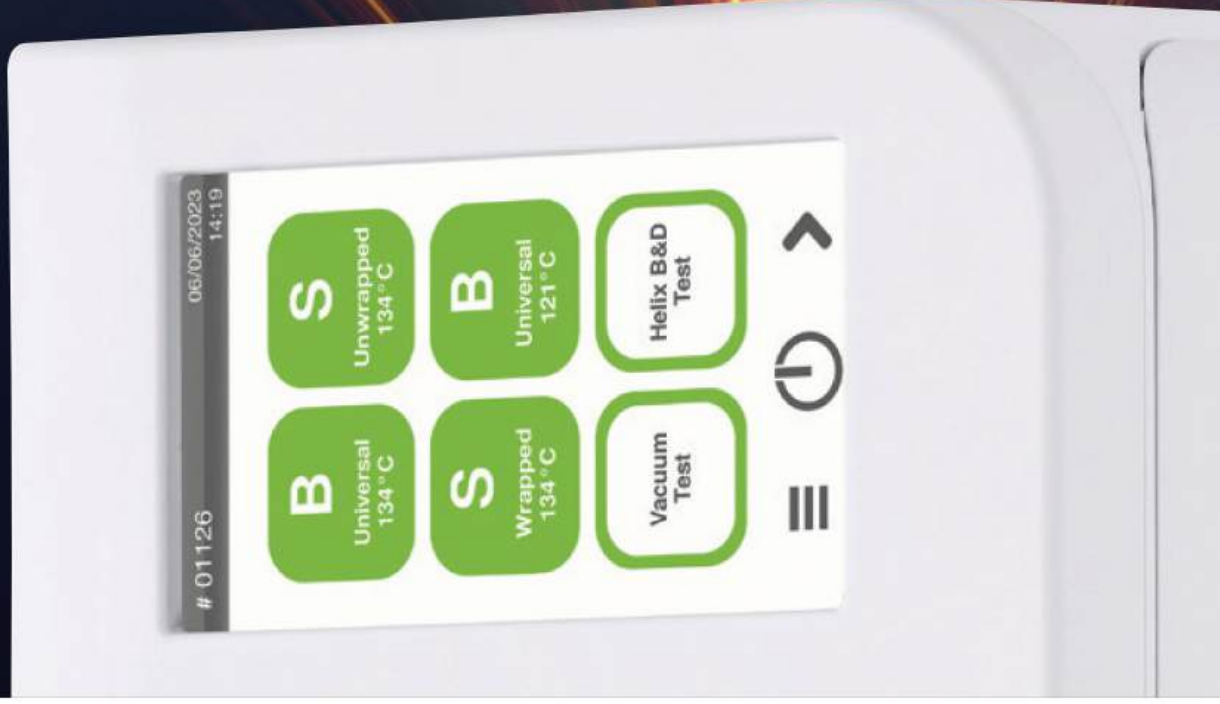

Simple cycle selection

As soon as the Lisa Mini is switched on, the home screen immediately displays the cycle selection.

.....

Risk-free cycle selection

Thanks to the possibility of customising the start screen with only the sterilization cycles used, daily activities are simplified.

.....

User guidance

Instruction is available with the help button in order to assist you during navigation.

Appropriate accessories

1 Barcode scanner

The scanner enables you to scan labels into patient files.

2 Sprint

Print out cycle protocols on thermal paper with 10-year paper guarantee.



W&H AIMS Safety first!

Hygiene is extremely important in dental practice. W&H AIMS is here to help: with a wide product portfolio, fundamental knowledge and all relevant information at a glance.

For your safety and the safety of your patients. **aims.wh.com**



Technical data

Sterilization cycle:		B Universal 134	B Prion 134	B Universal 121	S Wrapped 134	S Unwrapped
Temperature:		134°C	134°C	121°C	134°C	134°C
Holding time (minutes):		4'	18' 30"	20' 30"	4'	3' 30"
Total cycle duration (minutes) including drying:	3I ⁽¹⁾	19' 45"—23' 45"	34' 15"—38' 15"	37' 15"—38' 45"	17' 45"—19' 30"	8' 45"—10' 30"
	5I ⁽¹⁾	21' 45"—29' 15"	36' 15"—43' 45"	38' 15"—45' 15"	18' 45"—22' 45"	9' 45"—13' 45"
	5I ⁽²⁾	23' 45"—31' 15"	38' 15"—45' 45"	39' 45"—46' 45"	20' 45"—24' 45"	10' 15"—14' 15"
Load type:		All types of load ⁽³⁾			Solid items, Simple hollow items (Hollow B), Specific load: dental load, medical load ⁽⁴⁾ and handpieces	
Max. load:		Wrapped and unwrapped items: 1 kg (3 litre)/1.5 kg (5 litre)				Unwrapped items: 1 kg (3 litre)/1.5 kg (5 litre)
		Wrapped and unwrapped porous/textile: 100 g				
Test cycles:		Helix/Bowie and Dick/Vacuum				
Item supplied with 2 aluminium trays, drain hose, tray holder, 8 GB USB stick, door opening tool, factory test, compressed air tube ⁽¹⁾ .						

⁽¹⁾ Compressed air, ⁽²⁾ Vacuum pump, ⁽³⁾ Excluded liquid

⁽⁴⁾ Dental load according to ISO 17665-3; for medical load is intended to load up to grade 7 steam penetration according to ISO 17665-3.



Trade name:	Lisa Mini	
Type:	RIS-303	RIS-305
Chamber size:	3 litres	5 litres
Power supply:	200-240 V AC; 50/60 Hz; 10 A single phase 100-125 V AC; 50/60 Hz; 12 A single phase	200-240 V AC; 50/60 Hz; 10 A single phase
Power consumption:	2-2.4 kW (220 V)/1.2-1.5 kW (110 V)	
Overall dimension (w x h x d):	440 mm x 200 mm x 581 mm	440 mm x 200 mm x 666 mm
Weight:	21,9 kg	24,4 kg
Main/used water tanks:	1.6 l/0.6 l	2.5 l/0.6 l
Working range:	3 to 7 cycles	
Average air consumption:	50-60 (N)l/min (13.2-15.8 (N)gal/min)	
Working pressure:	5-10 bar (73-145 psi)	
Connection types:	5 USB ports, 1 Ethernet port, automatic water filling kit	

Lisa Mini sterilizers were designed, certified and validated with the most stringent directives and standards:		
2017/745 Medical Device Regulation	IEC 61326-1 Electromagnetic compatibility	IEC 61770 Electric appliances connected to the water mains
2012/19/EU Waste Electrical and Electronic Equipment	IEC 61010-1 Safety requirements	EN 13060 Small steam sterilizers
2014/68/EU Pressure Equipment Directive	IEC 61010-2-040 Specific requirements for steam sterilizer	• The sterilizer can be validated according to EN ISO 17665-1

Manufacturer:

W&H Sterilization Srl.
Via Bolgara, 2
24060 Brusaporto (BG), Italy
t +39 035-66 63 000
f +39 035-50 96 988
office.sterilization@wh.com
wh.com

Pictures are for illustrative purposes only. Additional equipment and accessories shown are not included as standard.

20578 AEN Rev. 000 / 15.05.2024
Subject to alterations