

# MOBILITY SCOOTER ASSEMBLY



- Use the frame release lever to lift the front section as you align the frame hooks of the front section with the lower frame tube of the rear section



- Once the frame hooks are over the lower frame tube, lower the front section completely. This will engage the frame lock-up.



- Raise the tiller and fully tighten the tiller adjustment knob.



- Reinstall the battery pack



- Reinstall the seat and rotate it until it locks into place



[contact@ridealong.com](mailto:contact@ridealong.com)

214.813.3186

# MOBILITY SCOOTER DISASSEMBLY



- Remove the seat by lifting it straight up and off of the scooter



- Remove the battery pack. Use the battery pack handle to pull the battery pack up and away from the scooter.



- Loosen the tiller knob by turning counter clockwise. Lower the tiller and then tighten the tiller adjustment knob by turning it clockwise.



- Lift up the frame release lever until the frame hooks are no longer resting on the lower frame tube. Slowly separate the two sections of the scooter.



[contact@ridealong.com](mailto:contact@ridealong.com)

214.813.3186