

leep

RING 1

English

LEEP RING 1 PRODUCT INFORMATION (EN)

Thanks for choosing Leep Ring 1! Please read the user manual and safety instructions to get the most out of your new smart ring.

TECHNICAL SPECIFICATIONS

- OPERATING TEMPERATURE RANGE: -20 – 50°C / 4–122°F
- BATTERY TYPE: Rechargeable Lithium-Ion
- BATTERY LIFE: 10 days (battery life may vary due to usage)
- CHARGING TIME: 1.5 hours (ring)
- CHARGING TIME: 2.5 hours (case)
- CHARGING TEMPERATURE: min 0°C / 32°F - max 45°C / 113°F
- BATTERY CAPACITY: 21.5 mAh, 0.079Wh (ring)
- BATTERY CAPACITY: 500 mAh, 1.85Wh (case)
- INPUT: 5V/1A, OUTPUT: 5V/16mA
- BLUETOOTH VERSION: 5.4
- BLUETOOTH RANGE: ≤10 m
- RF INTERFACE FREQUENCY RANGE: 2402-2480 GHz
- MAX OUTPUT POWER: 2.5mW

GENERAL INFORMATION

Product: Wellness Ring

Brand: Leep

Model number: Leep Ring 1

Manufacturer: Leep Health Ltd., 71-75 Shelton Street, Covent Garden, London, United Kingdom, WC2H 9JQ

Website: www.leephealth.com

Email: support@leephealth.com

LEEP RING 1 USAGE (EN)

Leep Ring 1 is designed for your everyday activities. Leep Ring 1 is water resistant (IP68) up to 50m / 164ft.

We recommend cleaning the ring regularly and keep it dry, to ensure good functionality of the ring and to reduce potential skin irritation. You can use a damp cloth to wipe the ring. Please ensure that the ring is fully dry before using it again.

Leep Ring 1 charging case is not water resistant. Do not use or store the charging case in humid environments like a bathroom for longer periods.

The Leep Ring 1 may get scratched in regular use and may in direct contact with softer materials (metal, jewelry or other objects) cause scratches itself if it gets in close contact.

The light-emitting diodes (LED) in the product are in the visible color range. They are safe for the person wearing the product and other people around.

LEEP APP

The Leep App is available in iOS App Store and Google Play. The Leep Ring is compatible with iPhones running iOS 14 or later and Android 6 or later.

SECURITY

Encrypted Bluetooth communication is used between Leep Ring 1 and Leep App.

FACTORY RESET

Instructions can be found at leephealth.com/support. Always perform a factory reset before disposing the ring.

WARRANTY DISCLAIMER

leephealth.com/terms-of-use

SUPPORT AND FAQs


If you have questions about using your Leep Ring 1, our Leep App, or our online services, please do not hesitate to contact us.

You will find our most Frequently Asked Questions (FAQs) at leephealth.com/support

For support you can contact us at support@leephealth.com or via the Leep App.

You can access Leep privacy policies at leephealth.com/privacy-policy


LEEP RING 1 CHARGING INSTRUCTIONS (EN)

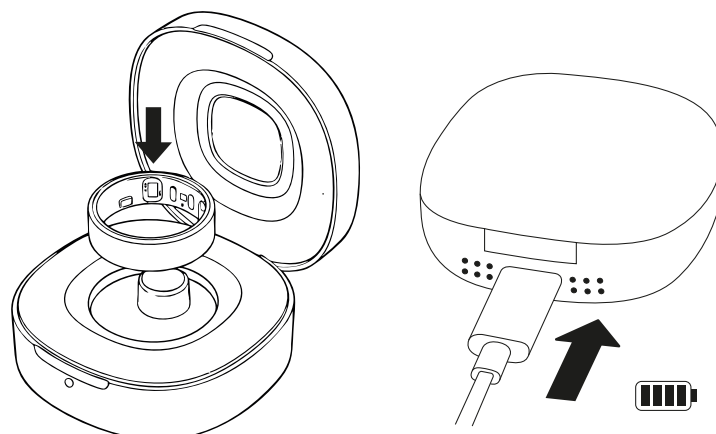

1. Connect the supplied USB Type-C Charging Cable or another certified USB Type-C power cable to the Charging Case.
2. Connect the USB Type-C Charging Cable to a 5V/1A external power source. Avoid charging the Charging Case with Fast Chargers as they may impact the battery life.
3. When power is connected and the charging case is fully charged, the green light on the front of the case stays on.
4. When the charging case has a low battery, the green light will flash rapidly. When the charging case is charging, the green light will flash slowly.
5. To charge the Leep Ring 1 place the ring in the Charging Case, ensuring the contacts on the engraved "L" side of the ring align with the charging contacts in the charging case.
6. When the ring is charging, the green light inside the ring will flash slowly and the light on the front of the charging case will stay on. You can charge the charging case with the lid open or closed.

Note: The charging case is not water resistant. For indoor use only.

HOW TO WEAR THE LEEP RING 1 (EN)

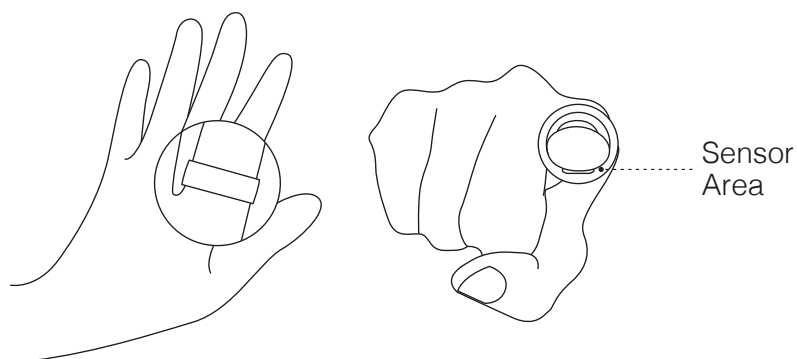

1. Choose a ring size that fits your finger to avoid wearing it too tightly or too loosely.
 2. For the best performance and accuracy, it is recommended to wear the ring on the index finger of the non-dominant hand. It can also be worn on any other finger as long as the ring sits securely and comfortably.
 3. When wearing the ring, ensure that the ring sensor (the side with the engraved "L") is facing the palm side of your finger for a more accurate measurement.
-

LEEP RING 1 TROUBLESHOOTING (EN)

If the ring does not pair with your smartphone:

1. Make sure the ring is charged.
2. Check your smartphone to ensure Bluetooth is turned on. This should be in your Bluetooth settings.
3. Restart the Leep App and try pairing again.

If the ring is not measuring any data:


Please contact our support team at support@leephealth.com

If the ring battery is draining too quickly:

1. Calibrate your battery by charging it for at least 3 hours.
2. Start using your ring again as you normally would.
3. If the problem persists, please contact our support team at support@leephealth.com

If you are experiencing an issue not mentioned here please contact our support team at support@leephealth.com and report the issue in detail.

LEEP RING 1 SAFETY (EN)

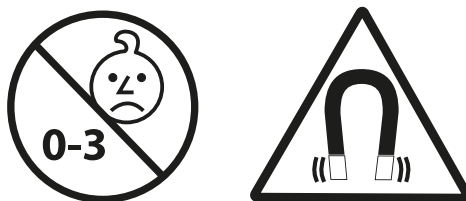

GENERAL SAFETY

Do not disassemble, modify, or expose the Leep Ring 1 to liquids, extreme heat, or open flames. Do not puncture, burn, or incinerate.

Avoid handling batteries, working on machinery that contains batteries, or situations where there is a risk of electric shock.

MEDICAL DEVICE WARNINGS

Leep Ring 1 is not a medical device. It's not intended to mitigate, prevent, treat, cure or diagnose any disease or condition. If you have any medical condition, you should always consult a licensed Physician. If you make any changes to your behavior based on data received from your Leep Ring 1 you agree that you do so at your own risk.

The Leep Ring 1 surface materials are safe for skin contact according to the ISO 10993 standard. If you experience irritation on your skin due to the Leep Ring 1, remove the ring. Please contact a physician if the skin irritation persists after not using Leep Ring 1.

Please be careful so that the Leep Ring 1 does not get caught on fixed structures or heavy objects.

If you experience any emergency or discomfort, or if you cannot remove your Leep Ring 1 yourself, we advise you to seek immediate medical attention.

This product contains magnetic components. If you have an implantable medical device, consult your physician before use. Keep a safe distance between the ring, charging case, and your medical implant.

CHILD SAFETY WARNINGS

This product contains small parts and magnets that may present a choking hazard. Not suitable for children under 3 years of age. Keep the ring and

charging case out of reach of young children. This product is not intended for individuals under the age of 18.

BATTERY WARNING

This product contains rechargeable Lithium-ion batteries. Do not open the device or attempt to replace the batteries. Mishandling may result in fire or explosion. Dispose of the product at certified electronic recycling facilities. Do not dispose of with household waste.

Do not expose Leep Ring 1 or the charging case to heat, such as in a vehicle or directly in the sun. Do not charge the Leep Ring 1 in such environments.

Avoid handling batteries, working on devices with machinery that contain batteries, or situations where there is a risk of electric shock.

LEEP RING 1 LEGAL (EN)

WARRANTY DISCLAIMER

THE SERVICES ARE PROVIDED TO YOU "AS IS WITH ALL FAULTS" AND "AS AVAILABLE" WITHOUT WARRANTY OF ANY KIND, AND LEEP AND ITS SUBSIDIARIES, AFFILIATES, OFFICERS, DIRECTORS, EMPLOYEES, REPRESENTATIVES, AGENTS, PARTNERS AND LICENSORS HEREBY DISCLAIM ALL WARRANTIES AND CONDITIONS WITH RESPECT TO THE SERVICES WHETHER EXPRESS OR IMPLIED AND EXPRESSLY DISCLAIMS ANY IMPLIED WARRANTIES OF TITLE, MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE, SATISFACTORY QUALITY, AND NON-INFRINGEMENT. LEEP DOES NOT WARRANT AGAINST INTERFERENCE WITH YOUR ENJOYMENT OF THE SERVICES, THE AVAILABILITY OF CONTENT, THAT THE FUNCTIONS CONTAINED IN THE SERVICES WILL MEET YOUR REQUIREMENTS, THAT THE SERVICES WILL BE FREE OF VIRUSES OR OTHER HARMFUL COMPONENTS, THAT THE OPERATION OF THE SERVICES WILL BE UNINTERRUPTED OR ERROR-FREE, THAT DEFECTS IN THE SERVICES WILL BE CORRECTED, OR THAT THE FUNCTIONS CONTAINED IN THE SERVICES WILL FUNCTION WITH OTHER MOBILE APPS OR HARDWARE, OR WITHIN A SYSTEM. NO ORAL OR WRITTEN INFORMATION OR ADVICE GIVEN BY LEEP OR A LEEP AUTHORIZED REPRESENTATIVE SHALL CREATE A WARRANTY. SOME JURISDICTIONS DO NOT ALLOW THE EXCLUSION OF IMPLIED WARRANTIES OR LIMITATIONS ON APPLICABLE STATUTORY RIGHTS OF A CONSUMER, SO THE ABOVE EXCLUSION MAY NOT APPLY.

NOT WITHSTANDING THE FOREGOING, LEEP WARRANTS TO THE ORIGINAL END USER PURCHASER THAT THE PRODUCT IS FREE FROM DEFECTS IN MATERIAL AND WORKMANSHIP FOR ONE (1) YEAR FROM THE DATE OF PURCHASE.

If your local law in effect at the time of purchase requires a warranty period longer than one (1) year, this warranty shall be extended to the extent required by such law. Within the warranty period, Leep shall repair or replace at no charge to you any components of the Product that fail the limited warranty provided. You shall be responsible for any related transportation charges. Replacement products may be new or refurbished at our discretion. This limited warranty does not apply to (i) normal wear and tear, including scratches and dents; (ii) consumable parts included in the Product, such as batteries, unless product damage has occurred due to a defect in materials or workmanship; (iii) damage resulting from your failure to use the Product in accordance with the instructions accompanying the Leep Product or available at the website; (iv) damage resulting from an accident, flood, fire, misuse, or

abuse; (v) damage resulting from service performed, or damage resulting from tampering or alterations to the Product, by anyone not authorized by Leep; or (vi) use of the Product with any other application or software than the Leep Mobile App.

Leep retains the exclusive right to repair or replace the Product, or offer a full refund, at its sole discretion. Such remedy shall be your sole and exclusive remedy for any breach of this limited warranty. Warranty repairs and replacements have a new warranty which is the longer of ninety (90) days or the balance of the original one (1) year warranty.

LIMITATION OF LIABILITY


IN NO EVENT WILL LEEP, ITS SUBSIDIARIES, AFFILIATES, OFFICERS, DIRECTORS, EMPLOYEES, REPRESENTATIVES, AGENTS, PARTNERS AND LICENSORS, BE LIABLE FOR DAMAGES OF ANY KIND, UNDER ANY LEGAL THEORY, ARISING OUT OF OR IN CONNECTION WITH YOUR USE, OR INABILITY TO USE, THE SERVICES OR ANY CONTENT ASSOCIATED WITH THE SERVICES, OR SUCH OTHER SITES OR ANY SERVICES OR ITEMS OBTAINED THROUGH THE SERVICES, INCLUDING ANY DIRECT, INDIRECT, SPECIAL, INCIDENTAL, CONSEQUENTIAL OR PUNITIVE DAMAGES, INCLUDING BUT NOT LIMITED TO, PERSONAL INJURY, PAIN AND SUFFERING, EMOTIONAL DISTRESS, LOSS OF REVENUE, LOSS OF PROFITS, LOSS OF BUSINESS OR ANTICIPATED SAVINGS, LOSS OF BUSINESS OPPORTUNITY, BUSINESS INTERRUPTION, LOSS OF USE, LOSS OF GOODWILL, LOSS OF DATA, AND WHETHER CAUSED BY TORT (INCLUDING NEGLIGENCE), BREACH OF CONTRACT OR OTHERWISE, EVEN IF FORESEEABLE. THESE EXCLUSIONS OR LIMITATIONS WILL APPLY REGARDLESS OF WHETHER OR NOT LEEP HAS BEEN WARNED OF THE POSSIBILITY OF SUCH DAMAGES. THE FOREGOING DOES NOT AFFECT ANY LIABILITY WHICH CANNOT BE EXCLUDED OR LIMITED UNDER APPLICABLE LAW. TO THE EXTENT LIABILITY CANNOT BE EXCLUDED OR LIMITED AS SET FORTH ABOVE, IN NO EVENT SHALL LEEP BE LIABLE FOR ANY CLAIM, WHETHER IN CONTRACT, TORT, OR UNDER ANY OTHER THEORY OF LIABILITY, IN EXCESS OF \$100.

TERMS OF USE

Your use of the Leep Products and Services is subject to our Terms of Use, available at leephealth.com/terms-of-use/, and Privacy Policy, available at leephealth.com/privacy-policy. The Warranty Disclaimer and the Limitation of Liability set forth above are from our Terms of Use and are subject to change if we change our Terms of Use.

INTELLECTUAL PROPERTY

Leep is a trademark of Leep Health Ltd. Leep Content, Leep Products, Leep Features and Services, and underlying technology are protected by copyright, trademark, and other laws of the United Kingdom and foreign countries. All rights reserved. Manufactured for Leep Health Ltd. 71-75 Shelton Street, Covent Garden, London, United Kingdom, WC2H 9JQ.

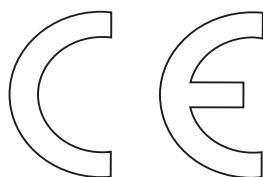LEEP RING 1 REGULATORY


EN: Conformity information

Leep Health Ltd. hereby declares that this radio equipment product is in compliance with Directive 2014/53/EU Radio Equipment Regulations 2017 and all other applicable EU regulations. The full text of the EU declaration of conformity is available at the following internet address:
leephealth.com/support/compliance




EN: Disposal and recycling

This product is compliant with Directive 2012/19/EU. Please dispose of used electrical and electronic equipment (WEEE) according to national laws and regulations with electrical waste. It should not be mixed with general household waste.

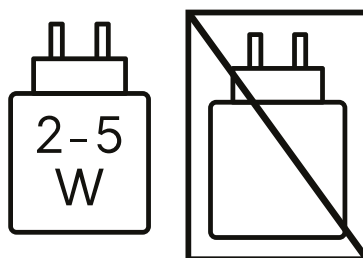

EN: Min 2 watts charging

The power delivered by the charger must be between min 2 Watts required by the radio equipment, and max 5 Watts in order to achieve the maximum charging speed.

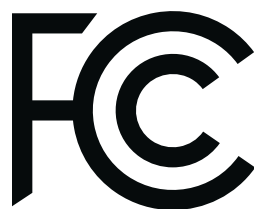

EN: FCC compliance

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference, and:
2. This device must accept any interference received, including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Changes or modifications not expressly approved by Leep for compliance could void the user's authority to operate the equipment.

FCC ID: 2BSX5-LR1

EN: RF exposure statement

This equipment complies with the FCC and ISED radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

EN: ISED Canada

This device complies with ISED Canada licence-exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.



Euverify Ltd

Unit 3D North Point House, North Point
Business Park, New Mallow Road, Cork,
T23 AT2P, Ireland
info@euverify.com



PACKAGING RECYCLING		Paper / Carta	Raccogli e differenzia. Verificare le disposizioni del tuo comune.
		Card / Carta	

