

OVERHEAD THUMBLOOP GOWN - FULL BACK

DONNING & DOFFING INSTRUCTIONS



PREPARING THE GOWN FOR DONNING

- Take hold of the gown from the neck area.
- Allow the remainder of the gown to unfold.



DONNING THE GOWN

- Slide your arms into the sleeves and hands into the thumbloop cuffs
- Pull the gown over your head.
- Pull the neckline of the gown up to maximize coverage.
- Tie the waist tie at the left side of the gown



DOFFING THE GOWN

- Break the belt using both hands.
- Cross one arm to the opposite shoulder, grab the gown and pull to break the neck closure.
- Roll the gown into a ball to contain gloves, fluids and contaminants and discard appropriately.

