

How to Use Your NADvance® Injection Pen

Follow these simple steps to safely prepare and administer your NAD+ treatment.

-  Easy to use
-  Precision dosing
-  Hygienic & Safe



Step-by-step Guide



1 Wash your hands thoroughly. Remove the cartridge from the fridge and allow it to rest at room temperature for 3 minutes. Gather all materials on a clean surface.




2 Unscrew the clear cartridge holder from the pen body. Insert the new cartridge, silver tip first into the holder.



3 Reattach the cartridge holder to the pen until secure. Attach a new sterile needle by twisting it on firmly. Remove the outer needle cap and keep it nearby (for safe disposal).



4 Hold the pen upright and dial 1 click to prime the device.

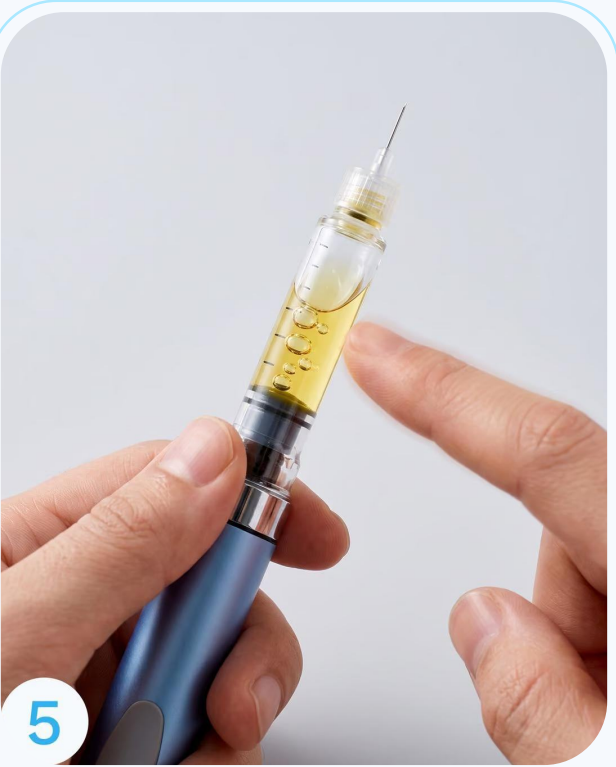

5 Tap the cartridge gently to move any air bubbles to the top.



6 Press the injection button to release a small drop of liquid. The pen is now ready.




7 Select an injection site: lower abdomen (at least 5 cm from the navel), outer thigh, or upper arm.



8 Clean the injection area with an alcohol swab. Pinch a fold of skin (about 2–3 cm).



9 Insert the needle straight (90° angle) into the pinched skin. Set the relevant dose. Hold the needle in place for 5 seconds to ensure full delivery.

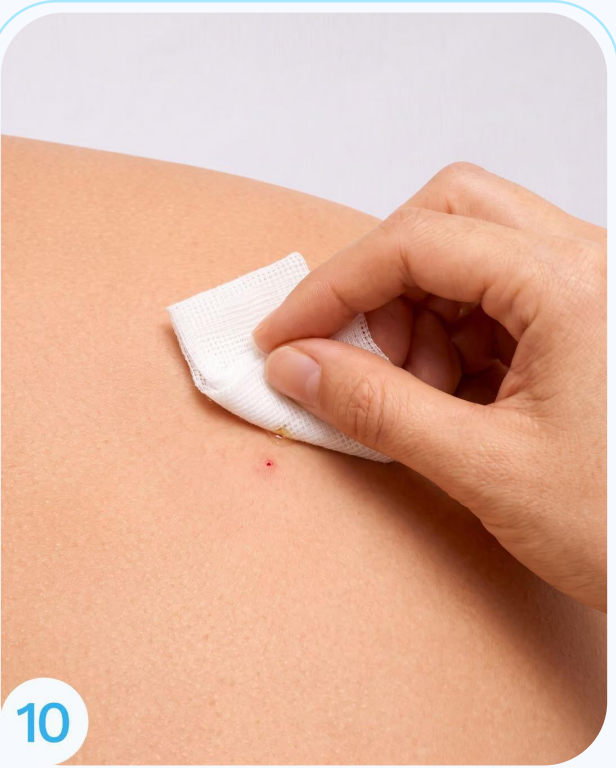

10 Withdraw the needle and apply gentle pressure with sterile gauze. Do not massage the site. Dispose of the used needle in a sharps container.

How to Use Your NADvance® Injection Pen

Follow these simple steps to safely prepare and administer your NAD+ treatment.

-  Easy to use
-  Precision dosing
-  Hygienic & Safe

Storage Guidelines

Proper storage keeps your Nadvance Pen safe, effective, and ready to support your wellness journey.



Store Between 2 °C to 8 °C

Refrigerate your Nadvance Pen and keep it between 2°C and 8°C to maintain its effectiveness.



Do Not Freeze Your Pen

Freezing can damage the pen and reduce its effectiveness.



Avoid Prolonged Room Temperature Exposure

Do not leave the pen at room temperature for extended periods. Always store it in the refrigerator.



Store correctly. Stay safe. Get the best results.

Proper storage ensures your Nadvance Pen works the way it should.



Suitability and Eligibility Criteria

Before we can fulfill your NADvance order, you have to certify the following eligibility criteria:



- ✓ I do not have a known allergy to NAD+.
- ✓ I have not been diagnosed with cancer within the past three years.
- ✓ I do not have any severe respiratory issues.
- ✓ I confirm that the self-injection pen is intended for my personal use only.
- ✓ I am not currently pregnant.