



Active Back

Pain can appear in different forms and in different region on your body. Worldwide, millions of people suffer from acute or chronic pain. Lower back pain is a common disorder involving bones, muscles and nerves. Lumbar back pain can have a serious impact on your quality of life.

PAINflex® Active Back provides a flexible bandage with six concentric magnets in, that will help to alleviate different symptoms of acute and chronic pain. This product is the alternative to pain killers, anti-inflammatories, and other oral preparations! Active Back support is made of breathable elastic and allows more flexibility. It is designed to be used possibly to relief muscle aches, pain and relieve tension in your body by improving blood circulation.

Indications?

On the one hand, the **PAINflex® Active Back** should be worn when you suffer from acute or chronic back pain. Furthermore, on the other hand, the support is ideal for preventing back pain when working or lifting heavy things.

How does it work?


PAINflex® Active Back consists of 3 x 2 concentric magnets. Each of the six included magnets is a patented 3000+ gauss deep penetration magnet consisting of circles of alternating polarity which are created to influence nature's own healing processes. The concentric magnet, with its unique configuration of both constant and gradient magnetic fields, causes an increase in the local blood flow (Hall Effect) accelerating the body's natural healing processes as well as blocking pain. The magnets in the **PAINflex® Active Back** are in unique configuration of both constant and gradient magnetic fields for enhanced circulation, reduced swelling and relief from chronic back pain. There are different effects:

Magnet → Dilation of blood vessels → increase circulation of blood flow and oxygen → **Pain alleviation**
Magnet → Nerve stimulation → create heat → **Pain blockage**


How to use?

There are common **characteristic pain spots** on the body, which are indicated by **red stars** in the right figure. **PAINflex® Active Back** is especially designed relief pain arising in lumbar back region (red box).

Due to its high flexibility the **PAINflex® Active Back** is easy to use. Wrap the elastic strap around your hips and pull it lightly before fastening it with the simple Velcro strap mechanism. The back support is pleasant to wear and the breathable and elastic material allows you to move around carefree.

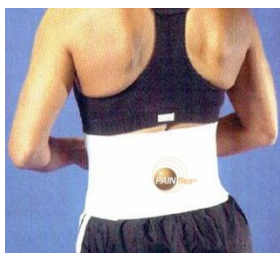

Due to the circulation-enhancing effect of the concentric magnets inside of the **PAINflex® Active Back** bandage, already after a few hours you can feel a natural arising pleasant warmth in your lumbar back. It can be worn 24/7 if desired. We suggest you wear it all-day for the first few days to relieve pain.

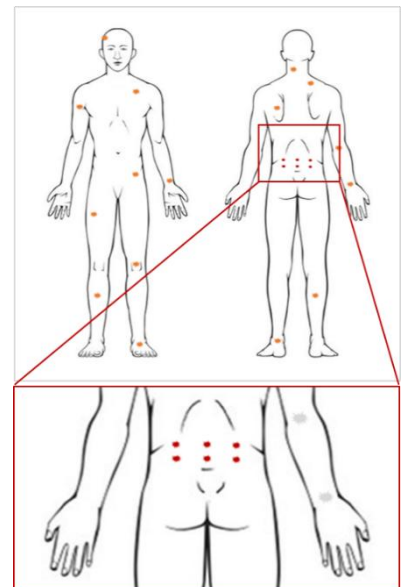

Contraindications/Warning

PAINflex® Active Back should never be placed directly over an open wound. It should not be used by individuals using pacemakers. If you want to use it during pregnancy, consult your physician first. **PAINflex® Active Back** support do not have any known unpleasant side effects. Hand wash only **PAINflex® Active Back** at 30°C. The product is not suitable for the dryer and do not iron the bandage.



Registered at the German Patent and Trademark Office under 399 851 31 –
European Union trade mark 001 885 706

International registration No. 1418181 (PAINFLEX) Office of Ukraine
International registration No. 1418181 (PAINFLEX) Office of Russian Federation



Warning pacemaker
Not near the pacemaker
Distance 10 mm

