# OCCLUSIONVD

EN | ENGLISH

## Instruction for use

### Important notes

- Read these instructions for use carefully before each use and keep them easily accessible for the user or the relevant technical personnel.
- Carefully read the warnings indicated by this symbol. Improper use of the products may result in serious injury to the patient, users or third parties.
- May only be supplied to dentists and dental laboratories or on their behalf. Keep away from children! Only for use by dentists!
- The instructions for use are valid for edelweiss OCCLUSIONVD with the following article numbers: 1315X, 1316X, 1317X, 1318X, 1319X, 132XX

### Product description

edelweiss OCCLUSIONVD are prefabricated, polymerized, radiopaque, highly filled nano-hybrid composite enamel shells. The occlusal and anatomical shape of the thin tooth shell is used for direct veneering of single or multiple posterior teeth. The edelweiss OCCLUSIONVD are cemented in place with an adhesive resin composite cement, resulting in a stable unit of the entire restoration. The composite is first filled into the tooth shell and cemented in with the tooth shell. The shade of the composite shines through the tooth shell and the restored tooth appears in the matching shade to the other teeth.

### Intended purpose

edelweiss OCCLUSIONVD form the anatomical basis for individual or complete reconstructions and for raising the vertical dimension in the posterior region.

### Indications

- Caries
- Old restorations
- All filling classes I-II
- Misaligned teeth
- Tooth fractures
- Tooth discoloration
- Veneering of old crowns
- Anatomical malformation
- Abrasion
- Erosion
- Cosmetic corrections
- Lifting of vertical dimension in the posterior region

### Contraindications

If known allergies to any of the components.

### Side effects

No systemic side effects are known. Contact allergies with products of similar composition have been reported in isolated cases. In such cases discontinue use and contact a physician.

**Analysis of the clinical literature has revealed that the following possible side-effects and risks may result from the use of this product:**

### Biological complications

- Secondary caries
- Endodontic complications
- Periodontal pathology
- Bleeding
- Post-operative sensitivity

### Technical complications

- Loss of retention
- Surface degradation
- Surface roughness
- Discoloration
- Low wear resistance
- Material fracture
- Chipping and marginal gap formation
- Debonding

### Intended users

The products are only intended to be used by dentists and dental laboratories or on their behalf.

### Patient population

There are no restrictions in the patient population except for the mentioned contraindications.

### Clinical benefit

The intended clinical benefit for edelweiss OCCLUSIONVD results from its intended purpose. The edelweiss OCCLUSIONVD are used for individual or complete reconstructions and for raising the vertical dimensions in the posterior region.

### Composition

- Barium glass
- Bis-GMA-based matrix made of dental resins
- Pigments
- Additives
- Catalyst

### Application

**Use of Rubber Dam is recommended whenever feasible.**

#### 1. Cleaning the teeth

Clean the tooth to be treated as well as the mesial and distal adjacent teeth using fluoride-free cleaning paste. Roughen/clean interdental areas using abrasive strips.

#### 2. Selection of the tooth shape

The tooth shape is selected using the supplied sizing guide (edelweiss OCCLUSIONVD Sizing Guide). The sizing guide is placed over the teeth to be restored, whereby the contours visible on the transparent film are used to select the best matching edelweiss OCCLUSIONVD.

#### 3. Selection of the enamel and dentin shades

Shade selection is performed before drying using the edelweiss Shade Guide in bright, natural daylight or with a light source that reproduces natural daylight conditions.

- 3.1. Selection of shade
  - Determine the dentin shade in the buccal and occlusal areas using the dentin shade stick, as this is where the shade is most pronounced. Insert the dentin stick into the Enamel VENEER tray.
- 3.2. Selection of shade brightness
  - Select shade of composite, in case of severely damaged or discolored teeth, a dentin material is recommended to achieve homogeneous coloring.

#### 4. Preparation of the tooth

Depending on the indication, minimally-invasive as possible.

- 4.1. Matrix/interdental wedges
  - Apply a thin, transparent matrix band to protect and separate the adjacent teeth. If necessary, fix the matrix interproximally with interdental wedges.

#### 5. Pretreatment of edelweiss OCCLUSIONVD

- 5.1. Adjust the fit of the edelweiss OCCLUSIONVD.
- 5.2. To improve adhesion, the edelweiss OCCLUSIONVD is sandblasted on the inner fitting surface using 25 µm or 50 µm aluminum oxide or roughened using a diamond bur.
- 5.3. A commercially available resin adhesive bond or enamel bond (e.g. "edelweiss VENEER BOND", "Parkell Add&Bond Adhesive Composite Primer", "Bisco All-Bond", "Clearfil SE Bond", etc.) can be used to bond the

restoration. The bond is applied to the inner surface of edelweiss OCCLUSIONVD and cured using light. The instructions for use of the manufacturer of the bond must be observed.

#### 6. Pretreatment of prepared tooth surface

- 6.1. Isolating and cleaning the preparation: During adhesive luting/cementing protocol thorough isolation of the operating field - preferably with a rubber dam or alternatively with cotton rolls and a saliva ejector.
- 6.2. Clean the tooth surface using a polishing brush and an oil- and fluoride-free cleaning paste and rinse with water spray. Then lightly dry with water- and oil-free air. Avoid over-drying.
- 6.3. Apply 37 % phosphoric acid gel to the prepared enamel and then flow the etchant onto the prepared dentin. The instructions for use of the manufacturer of the etchant must be observed.
- 6.4. Rinse thoroughly with water and gently dry with compressed air.
- 6.5. Starting with the enamel, thoroughly coat the tooth surfaces to be treated with Dentin Bonding Agent/ Adhesive. The adhesive must be gently scrubbed into the tooth surface for at least 20 seconds and light cured as per manufacturer's instructions.

#### 7. Cementation of edelweiss OCCLUSIONVD

- It is recommended to use a light cured resin-based composite for cementation of the edelweiss OCCLUSIONVDs. Follow instructions for use from the manufacturer regards pre-treatment of the tooth.
- 7.1. The light cured resin-based composite is applied to the edelweiss OCCLUSIONVDs and positioned onto the prepared tooth.
  - 7.2. If necessary, build up the cavity as well as the lingual and buccal areas with several layers of composite.
  - 7.3. Take roughly 1 cm of the composite strip in the fingertips and roll it into a ball. Use the finger tips to press the ball into the inside of the edelweiss OCCLUSIONVD.
  - 7.4. Use the spatula instrument to evenly distribute the composite into the edelweiss OCCLUSIONVD.
  - 7.5. Adapt excessive composite and remove excess material. The previously placed interproximal matrices prevent adhesion between the teeth.
  - 7.6. Light-cure with a curing light for 2 seconds and remove all excess composite material.
  - 7.7. Final light-cure for 20 seconds on all surfaces and margins with a curing unit with light intensity of 1000 mW/cm².

#### 8. Finishing and polishing

After having adhesively cemented the restoration, adjust occlusion/articulation with suitable finishing instruments. Work the interproximal areas with finger-strips then polish with polish-strips. Polish the cervical areas with silicone-polishing cup. Final polishing is done with silicone polishers and cotton/linen buffs.

#### 9. Indirect application on cast model

By lining the pre-fabricated occlusal surfaces on existing cast model occlusal surfaces with composite, one can simply create a new functioning biomechanical, balanced and smoothly working masticular apparatus. As a consequence, the occlusions of the model are transferred to the patient's occlusion, fixed with bonding and cemented with composite. The maxilla and the mandible models are articulated in habitual bite position. A bite registry material (silicone) is used to make an occlusal impression which accurately portrays the clearance between maxilla and mandible in the articulator. Using the fixed silicone bite register, direct application of the pre-fabricated occlusion (in the patient's mouth) can be performed. The silicone bite register is in each case placed unilaterally (left or right) on the patient's occlusion. In other words, when the left quadrant is restored with occlusions, the right quadrant fixes the height and position of bite elevation and thus the final position of the composite-lined and bonded occlusions on the left when biting down using the silicone bite registry (replaces the supporting pin when closing the articulator). Once the left quadrant has been restored, the right quadrant is treated. Now one can optionally prepare a further silicone register from the quadrant already restored with occlusions (left) in combination with the silicone register (right) already prepared with the articulator, which additionally guides and fixes the already attained occlusal relationship between the maxilla and the mandible (left) during biting. Experienced users can dispense with this silicone register (for example, on the left) as the patient now already bites on the newly gained occlusion or bite elevation during biting or the final positioning of the occlusions on the left respectively prior to curing by the treating clinician. After fixating all occlusions, reocclusion is performed until the desired contact points have been attained.

#### Reusability

The edelweiss OCCLUSIONVD are marketed as "single use". The edelweiss OCCLUSIONVD are intended for single use only. Single-use products must not be reused, since according to their design they no longer function as intended after the first use.

#### Disposal

Please dispose of the devices properly at the end of the product life. The national regulations and disposal guideline must be observed.

#### Important

To ensure optimal results, it is recommended to use in combination with all edelweiss components.

#### Obligation to report

All serious incidents that occurred in relation to the device must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

#### Summary report on safety and clinical performance

A summary report on safety and clinical performance in accordance with Regulation (EU) 2017/745, Art. 32, is available at the following web address: <https://ec.europa.eu/tools/eudamed>

Symbols	Description
	Symbol for "Manufacturer"
	Symbol for "Date of manufacture"
	Symbol for "Medical device"
	Symbol for "Article number"
	Symbol for "Batch code"
	Symbol for "Consult instructions for use or consult electronic instructions for use"
	Symbol for "CE marking of conformity" with identification number of the notified body
	Symbol for "Non-sterile"
	Symbol for "Do not reuse"
	Symbol for "Attention"
	Symbol for "Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner"

Date of issue of this instruction for use  
2026-04

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