# VENEER

EN | ENGLISH

## Instruction for use

### Important notes

- Read these instructions for use carefully before each use and keep them easily accessible for the user or the relevant technical personnel.
- Carefully read the warnings indicated by this symbol. Improper use of the products may result in serious injury to the patient, users or third parties.
- May only be supplied to dentists and dental laboratories or on their behalf. Keep away from children! Only for use by dentists!
- The instructions for use are valid for edelweiss VENEER with the following article numbers: 101XX, 130XX, 1310X, 1311X, 141XX, 170XX

### Product description

edelweiss VENEER are prefabricated, polymerized, radiopaque, highly filled nano-hybrid composite enamel shells. The anatomical shape of the thin tooth shell is used for direct veneering of single or multiple anterior teeth. The edelweiss VENEER are cemented in place with an adhesive resin composite cement, resulting in a stable unit of the entire restoration. The composite is first filled into the tooth shell and cemented in with the tooth shell. The shade of the composite shines through the tooth shell and the restored tooth appears in the matching shade to the other teeth.

### Intended purpose

edelweiss VENEER form the anatomical basis for individual or complete reconstructions in the anterior region.

### Indications

- Caries
- Old restorations
- Lengthening of the incisal edge
- Misaligned teeth
- Tooth fractures
- Tooth discoloration
- Incorrect chromaticity
- Veneering of old crowns
- Anatomical deformities
- Diastemas
- Attrition
- Abrasion
- Erosion
- Cosmetic corrections

### Contraindications

If known allergies to any of the components.

### Side effects

No systemic side effects are known. Contact allergies with products of similar composition have been reported in isolated cases. In such cases discontinue use and contact a physician.

**Analysis of the clinical literature has revealed that the following possible side-effects and risks may result from the use of this product:**

### Biological complications

- Secondary caries
- Endodontic complications
- Periodontal pathology
- Bleeding
- Post-operative sensitivity

### Technical complications

- Loss of retention
- Surface degradation
- Surface roughness
- Discoloration
- Low wear resistance
- Material fracture
- Chipping and marginal gap formation
- Debonding

### Intended users

The products are only intended to be used by dentists and dental laboratories or on their behalf.

### Patient population

There are no restrictions in the patient population except for the mentioned contraindications.

### Clinical benefit

The intended clinical benefit for edelweiss VENEER results from its intended purpose. The edelweiss VENEER are used for individual or complete reconstructions in the anterior region.

### Composition

- Barium glass
- Bis-GMA-based matrix made of dental resins
- Pigments
- Additives
- Catalyst

### Application

**Use of Rubber Dam is recommended whenever feasible.**

#### 1. Cleaning the teeth

Clean the tooth to be treated as well as the mesial and distal adjacent teeth using fluoride-free cleaning paste. Roughen/clean interdental areas using abrasive strips.

#### 2. Selection of the tooth shape

Selection of the tooth shape is performed using the sizing template provided (edelweiss VENEER Sizing Guide). The transparent sizing template is placed over the teeth to be restored in order to provide a visible profile when selecting the best fitting edelweiss VENEER. Necessary corrections can also be marked on the sizing template. Adjustments to the shape of the edelweiss VENEER can be best performed using a rough surfaced disc at a slow speed without water.

#### 3. Selection of the enamel and dentin shades

Shade selection is performed before drying using the edelweiss Shade Guide in bright, natural daylight or with a light source that reproduces natural daylight conditions.

##### 3.1. Selection of shade

Determine the dentin shade in the cervical area using the dentine shade stick, as the shade is most pronounced here. Insert the dentine stick into the Enamel VENEER tray.

##### 3.2. Selection of shade intensity and opacity

- Cervical area of the dentin shade/dentin composite
- Incisal area of the enamel shade/enamel composite

##### 3.3. Selection of shade brightness

#### 4. Preparation of the tooth

Depending on the indication, minimally-invasive as possible.

##### 4.1. Matrix/interdental wedges

Apply a thin, transparent matrix band to protect and separate the adjacent teeth. If necessary, fix the matrix interproximally with interdental wedges.

#### 5. Pretreatment of edelweiss VENEER

5.1. Adjust the fit of the edelweiss VENEER.

5.2. To improve adhesion, the edelweiss VENEER is sandblasted on the inner fitting surface using 25 µm or 50 µm aluminum oxide or roughened using a diamond bur.

5.3. A commercially available resin adhesive bond or enamel bond (e.g. "edelweiss VENEER BOND", "Parkell Add&Bond Adhesive Composite Primer", "Bisco All-Bond", "Clearfil SE Bond", etc.) can be used to bond the restoration. The bond is applied to the inner surface of edelweiss VENEER and cured using light. The instructions for use of the manufacturer of the bond must be observed.

#### 6. Pretreatment of prepared tooth surface

6.1. Isolating and cleaning the preparation: During adhesive luting/cementing protocol thorough isolation of the operating field – preferably with a rubber dam or alternatively with cotton rolls and a saliva ejector.

6.2. Clean the tooth surface using a polishing brush and an oil- and fluoride-free cleaning paste and rinse with water spray. Then lightly dry with water- and oil-free air. Avoid over-drying.

6.3. Apply 37 % phosphoric acid gel to the prepared enamel and then flow the etchant onto the prepared dentin. The instructions for use of the manufacturer of the etchant must be observed.

6.4. Rinse thoroughly with water and gently dry with compressed air.

6.5. Starting with the enamel, thoroughly coat the tooth surfaces to be treated with Dentin Bonding Agent/ Adhesive. The adhesive must be gently scrubbed into the tooth surface for at least 20 seconds and light cured as per manufacturer's instructions.

#### 7. Cementation of edelweiss VENEER

It is recommended to use a light cured resin-based composite for cementation of the edelweiss VENEER. Follow instructions for use from the manufacturer regards pre-treatment of the tooth.

7.1. The light cured resin-based composite is applied to the edelweiss VENEER and positioned onto the prepared tooth.

7.2. Select the composite shade. For substrates that are severely broken down or discolored a dentin shade is recommended to provide a homogenous color. For shape corrections, an enamel composite can be used (Enamel).

7.3. Take roughly 1 cm of the composite strip in the fingertips and roll it into a ball. Use the fingertips to press the ball into the inside of the edelweiss VENEER.

7.4. Use the spatula instrument to evenly distribute the composite into the edelweiss VENEER.

7.5. Press the central edelweiss VENEERS in desired positions.

7.6. For placement of multiple edelweiss VENEER: With a complete anterior restoration it is recommended to begin with the central edelweiss VENEER.

7.7. Adapt excessive composite and remove excess material.

The previously placed interproximal matrices prevent adhesion between the teeth.

7.8. Light-cure with a curing light for 2 seconds and remove all excess composite material.

7.9. Final light-cure for 20 seconds on all surfaces and margins with a curing unit with light intensity of 1000 mW/cm².

#### 8. Finishing and polishing

After having adhesively cemented the restoration, adjust occlusion/articulation with suitable finishing instruments.

Work the interproximal areas with finger-strips then polish with polish-strips.

Polish the cervical areas with silicone-polishing cup.

Final polishing is done with silicone polishers and cotton/linen buffs.

#### Reusability

The edelweiss VENEER are marketed as "single use". The edelweiss VENEER are intended for single use only. Single-use products must not be reused, since according to their design they no longer function as intended after the first use.

#### Disposal

Please dispose of the devices properly at the end of the product life. The national regulations and disposal guideline must be observed.

#### Important

To ensure optimal results, it is recommended to use in combination with all edelweiss components.

#### Obligation to report

All serious incidents that occurred in relation to the device must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

#### Summary report on safety and clinical performance

A summary report on safety and clinical performance in accordance with Regulation (EU) 2017/745, Art. 32, is available at the following web address: <https://ec.europa.eu/tools/eudamed>

Symbols	Description
	Symbol for "Manufacturer"
	Symbol for "Date of manufacture"
	Symbol for "Medical device"
	Symbol for "Article number"
	Symbol for "Batch code"
	Symbol for "Consult instructions for use or consult electronic instructions for use"
	Symbol for "CE marking of conformity" with identification number of the notified body
	Symbol for "Non-sterile"
	Symbol for "Do not reuse"
	Symbol for "Attention"
	Symbol for "Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner"

Date of issue of this instruction for use  
2026-04

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