

How to apply myPatch®sl for adult patients

STEP 1 Prepare the skin

Remove any chest hair then clean the area with the prep pads.

Allow skin to dry.

STEP 2 Prepare the device


- Remove the electrode Patch from the pouch (keep pouch)
- Peel the protective cover off the front of the myPatch®sl electrode
- Press the myPatch®sl recorder onto the four metal snaps on the electrode patch.
- Hold the power button down – a green light will flash for 30 secs while the device starts up

STEP 3 Apply myPatch®sl

Peel the plastic cover off the back off the myPatch®sl electrode and place on the body as shown. Smooth out all edges of the electrode, ensuring there are no creases or folds.


STEP 4 Record time & date

You must now mark the time & date of your fitting on your symptom diary.

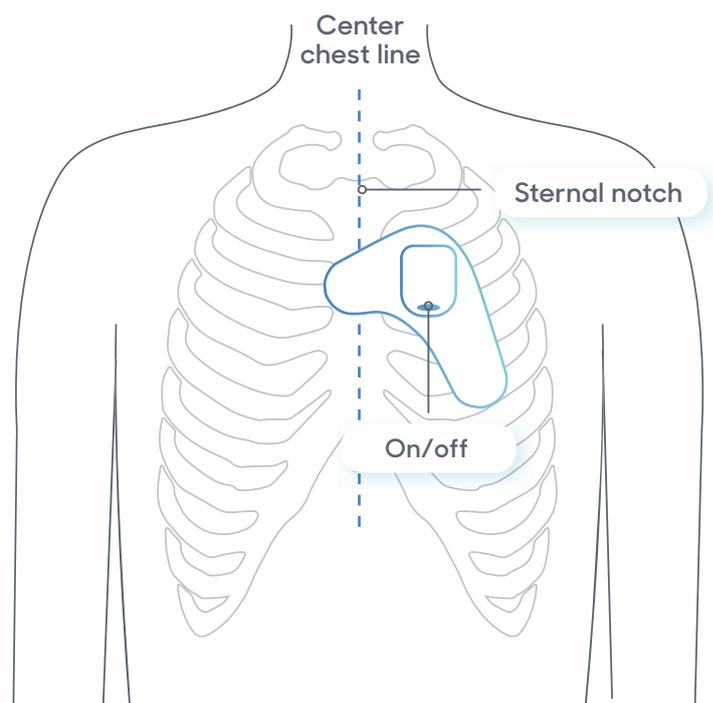

Watch "How to apply adult myPatch®sl" video

Use the QR code

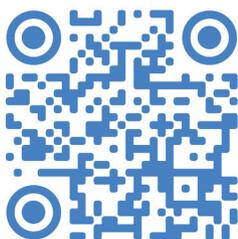

Watch easy-to-follow video instructions by scanning the QR code (above) with your iPhone camera and clicking the link or using your preferred Android QR code reader.

Website

Visit cardioscan.co/mypatch-help

Recording events and symptoms

Patients feeling symptoms such as dizziness or heart flutters, should double tap myPatch®sl to mark the event. A blue light comes on briefly to indicate the event has been captured.

Watch the video guide online
cardioscan.co/mypatch-help