

How to apply myPatch®sl for T-electrode

STEP 1 Prepare the skin

Remove any chest hair then clean the area with the prep pads. Allow skin to dry.

STEP 2 Prepare the device

- Remove the electrode Patch from the pouch
- Press the myPatch®sl recorder onto the four metal snaps on the electrode patch.
- Hold the power button down – a green light will flash for 30 secs while the device starts up

STEP 3 Apply myPatch®sl

Peel the plastic cover off the back of the myPatch®sl T- electrode and locate the bottom of the sternum with one finger.

Place the bottom of the T-electrode above your finger before moving on to the top two points, placing the T-electrode so it sits in the centre of the chest. Pat firmly onto the skin to ensure there are no creases or folds.


STEP 4 Record time & date

You must now mark the time & date of your fitting on your symptom diary.

Note – Recording starts when flashing stops. No light will be active during the recording.


Recording events and symptoms

Patients feeling symptoms such as dizziness or heart flutters, should double tap myPatch®sl to mark the event. A blue light comes on briefly to indicate the event has been captured.

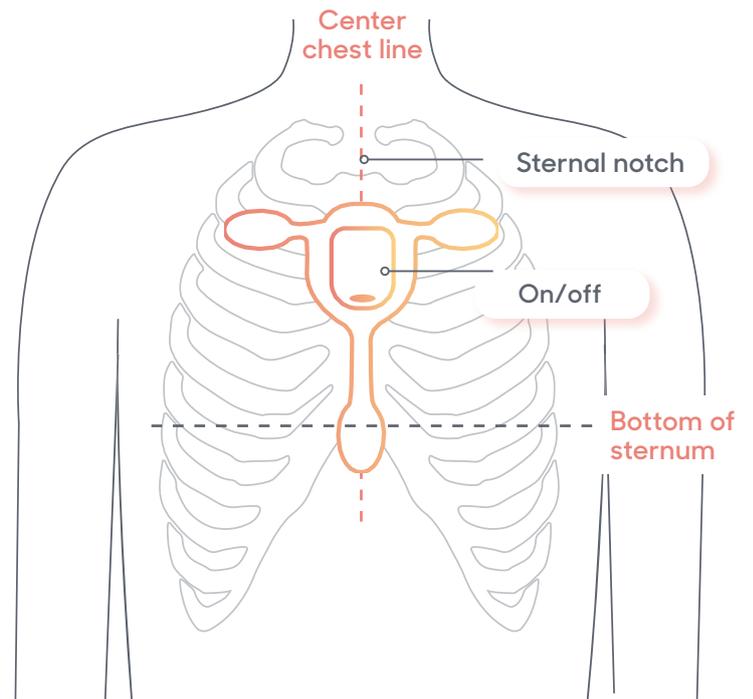

Watch "Apply T- electrode adult myPatch®sl" video

Use the QR code

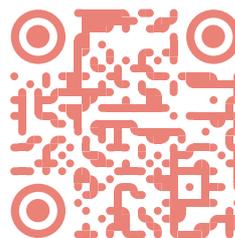

Watch easy-to-follow video instructions by scanning the QR code (above) with your iPhone camera and clicking the link or using your preferred Android QR code reader.

Website

Visit cardioscan.co/myPatch-help

Contact us

For more resources, guides, FAQs or to watch instructional videos, visit:

cardioscan.co

CardioScan Australia

Suite 301, 293 Camberwell Rd,
Camberwell, VIC 3124 Australia

T: (+61) 9832 2222

E: info@cardioscan.com.au

CardioScan Singapore

80 Bendemeer Rd, #07-03,
Singapore 339949

T: (+65) 6735 9552

E: info@cardioscan.sg

CardioScan Hong Kong

Rm 1301, 13/F, 102 Austin Road,
Jordan, Kowloon, Hong Kong

T: (+852) 3996 8969

E: info@cardioscanhk.com

CardioScan Thailand

Rm 1103, 11/F, 51 Major Tower,
Rama 9 - Ramkhamhaeng Rd.,
Hua Mak, Bang Kapi, Bangkok 10240

T: (+66) 92 491 2999

E: info@cardioscan.co.th

CardioScan Malaysia

No 88-2A Jalan PJU 1/3B, Sunwaymas
Commercial Centre, 47301 Petaling Jaya,
Selangor Darul Ehsan

T: +61 9832 2222

E: contact@cardioscan.com.my

CardioScan UK

7 & 8 Crescent Stables, 139 Upper Richmond Road,
Lodon SW15 2TN

T: (+44) 1992 351 033

E: info@cardioscan.co.uk

Cardiac Monitoring Service (USA)

1300 Quail Street, Suite 209,
Newport Beach, CA 92660, USA

T: (+1) 800 267 5248

E: info@cardiacmonitoringservice.com