Otolift
Modul-Air Smart

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Contents

1	Safety	3
1.1	Introduction	3
1.2	Symbols and labels	3
1.2.1	Safety warnings	3
1.2.2	Note	3
1.3	Intended use	3
1.4	Reasonably foreseeable misuse	3
1.5	Safety components	4
1.6	Safety precautions	5
1.7	Labels	5
1.8	Emergency situations	5
2	Description of the stairlift	6
2.1	Design and function	6
2.1.1	Stairlift components	6
2.1.2	Display interface	7
3	Use	8
4	Move the stairlift with the remote control	15
5	In case of a power outage or malfunction of the stairlift	16
6	Optional: Calling for assistance	17
7	Display notifications and alarms	18
7.1	Current status	18
7.2	Alarms on the display	20
8	Maintenance	23
8.1	Cleaning	23
8.2	Maintenance schedule	23
8.3	Replacing the remote control battery	23
8.4	Replacing the key fob battery	23
9	Disposal	23
10	Technical data	24
10.1	Type plate	24
11	EC declaration of conformity for machinery	25
Appendix 1: Schematics		26

Introduction

About this document

This manual contains technical descriptions and instructions for the following product:

Category of product	Stairlift
Product name	Otolift Modul - Air Smart
Serial number	

Write down the serial number of your product in the table above for future reference.

This manual contains all instructions and safety information for use of the product.

This manual is intended for:

- the end - user who uses the product;
- the product owner who cleans the product.

Original instructions

This manual has been translated into multiple languages. The original manual is written in UK English. All other language versions are translations of the original manual.

Copyright

The content of this manual is protected by copyright and other intellectual property laws. The content of this manual may only be copied, modified, reproduced, translated with explicit written permission from the manufacturer. This manual may only be published, transmitted, displayed or made available to a third party with explicit written permission from the manufacturer.

Disclaimer of liability

Royal Otolift Stairlifts cannot be held responsible for personal injury, damage to the product or property damage caused by incorrect use, foreseeable misuse or failure to follow the instructions in this manual. This also applies to unauthorized modifications of the product and the use of non - approved spare parts, tools or accessories.

Royal Otolift Stairlifts reserves the right to modify this manual without notification beforehand.

1 Safety

1.1 Introduction

Please make sure you have fully read and understood the instructions in this manual before you start using the stairlift. If you fail to follow the instructions from this manual you can put persons, surroundings, and the stairlift at risk. Store this manual in an accessible place near the stairlift for future reference.

Always comply with the information, such as labels and the nameplate, attached directly to the stairlift and keep the information in a legible condition. If the labels have become unreadable or damaged, you must replace them.

Always comply with any applicable laws and regulations that have not been accounted for in this manual.

1.2 Symbols and labels

1.2.1 Safety warnings

This manual contains safety warnings that may result in injury when ignored. Each safety warning is indicated with a signal word. The signal word corresponds with the level of risk of the described hazardous situation, see the table below.

Signal word	Risk of injury	Result when instruction is ignored or not followed correctly
 WARNING!	Medium	Could result in death or serious injury
 CAUTION!	Low	Could result in minor or moderate injury

Safety warnings at the start of a section, apply to the entire section.

Safety warnings that refer to a particular sentence or procedure step are embedded in the running text.

1.2.2 Note

Messages that are not hazard - related are indicated with the signal word **Note**. These messages do not have a safety alert symbol.

Signal word	Function
Note	Indicates extra information that is not directly hazard or damage related.

1.3 Intended use

The Otolift Modul - Air Smart stairlift is designed for various staircases and situations, and is intended to transport one person comfortably throughout the journey over the staircase. The stairlift is only intended to be used on the staircase for which Otolift has originally configured and installed the stairlift.

The Otolift Modul - Air Smart may only be installed or dismantled by Otolift trained personal.

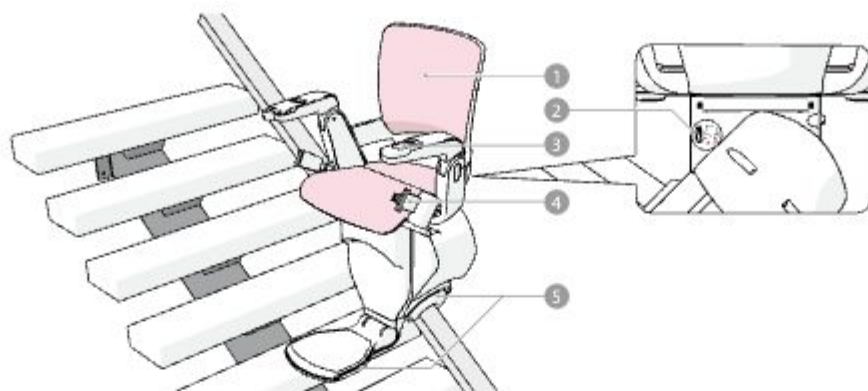
1.4 Reasonably foreseeable misuse

The following is considered unintended use and will void all warranty / liability by Otolift.

- Use by under - aged or mentally disadvantaged people without supervision.
- Transporting animals or goods, unless the stairlift is specifically adapted for this purpose by Otolift.
- Transporting more than one person at a time.
- Transporting a person heavier than the maximum load capacity, as stated in the technical specifications in this manual.
- Use without using the seatbelt.
- Use that varies from or exceeds the given operating conditions.
- Failure to comply with the instructions in this manual.
- Failure to eliminate faults, malfunctions or defects of the product that impose safety risks.
- Unauthorised removal or modification of parts or safety devices of the product.
- Use of spare parts or accessories that have not been approved by the manufacturer.

1.5 Safety components

The stairlift is equipped with several safety components that ensure safe working with the product.

1 Swivel seat and folding footplate

On narrow staircases it may be necessary to drive with the chair rotated to avoid obstacles for the head / knees or the footplate.

2 Emergency swivel release

In case of a stairlift malfunction, or any other emergency situation, use the emergency swivel release on the backside of the frontframe to manually swivel the stairlift toward the stairs for safe dismounting.

3 Armrest on/off switch

The on / off switch in the armrest ensures that a person can get on or off the stairlift safely at the endpoints, without the possibility of someone else controlling the stairlift using the remote control units. The on / off switch also functions as the stop button when an emergency stop is needed.

4 Seatbelt with sensor

To prevent falling hazards, the stairlift is standard equipped with a seatbelt with seatbelt sensor. This prevents the stairlift from moving when you try to use the stairlift without the seatbelt fastened.

5 Obstruction sensors

The stairlift is equipped with obstruction sensors in the footplate and around the rail, to ensure that the stairlift stops when something is obstructing the stairlift in the travel direction. When triggered, the stairlift can only be moved in the opposite direction so the obstruction can be removed.

1.6 Safety precautions

Despite the safe design and construction of the stairlift and the prescribed protective measures, the product possesses residual risks. This manual provides safety messages to indicate these risks. The formatting and appearance of safety messages that are dedicated to a particular section or sentence is explained in the Preface. Overall safety messages are grouped in the overview below.

⚠ WARNING!

- The stairlift should be supervised when used around children;
- Children should not be allowed to use the stairlift. Do not allow children to play on or around the stairlift;
- Do not put your hands on the rails to avoid pinching hazard;
- Always wear the safety belt when using the stairlift to avoid falling off the stairlift while in use;
- Always keep all arms and legs on the stairlift to avoid pinching hazard;
- Do not use this product if the chair, footrest, railing or seatbelt show any signs of damage;
- Do not use this product if the housing is broken, cracked, open, or show any other indication of damage;
- Switch off the stairlift and immediately contact customer support for service if at any time you think the stairlift is unsafe. Do not use the stairlift until the problem is identified and corrected;
- Do not remove, relocate or repair any parts of the stairlift.

⚠ CAUTION!

- Ensure that the power supply cable of the stairlift is positioned so it will not be stepped on, tripped over, or otherwise be subjected to damage or stress;
- There are no user serviceable parts inside. Do not attempt to repair or service the stairlift yourself.

1.7 Labels

This label indicates a pinching hazard! Do not put your hands on the rails to avoid pinching hazard.

This symbol indicates the location of the emergency swivel release. Use this lever to swivel the seat towards the stairs in case of a power outage or malfunction of the stairlift to safely dismount the stairlift.

This symbol indicates that this product may not be disposed with household waste. Recycle this product in an environmentally friendly manner according to local regulations.

1.8 Emergency situations

The stairlift may not be used for fire - prevention or evacuation purposes. In case of fire, do not use the stairlift.

2 Description of the stairlift

2.1 Design and function

The stairlift is moved along the rail by means of rack and pinion and is powered by an electric motor with rechargeable batteries.

The batteries are charged with an integrated charger in the stairlift. The charger is powered through charge points on the rail, connected to a transformer that is plugged into an electrical outlet.

Optionally, extra charge points can be implemented on safe mounting / dismounting positions or on specific "parking only" positions. For example, if the stairlift covers multiple floors, an additional charge point can be placed halfway along the rail on the first floor.

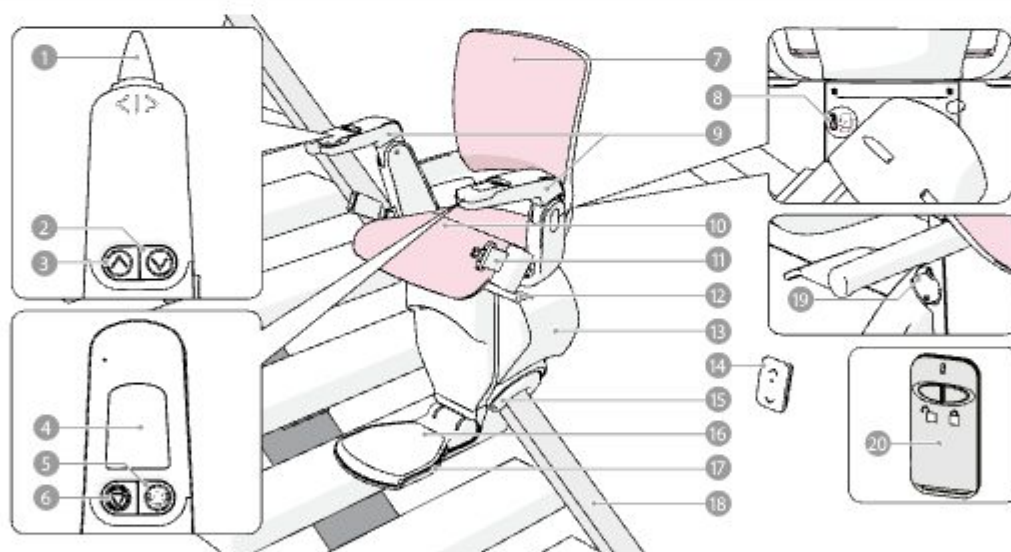
The stairlift can be controlled in two ways:

- The joystick control on the end of the armrest;
- Remote control units mounted near both endpoints of the rail, in line of sight with the stairlift. (In some cases there will not be a remote control unit at every endpoint)

A display is implemented in the armrest and provides you with useful information such as battery level, the current status of the stairlift and more. In case of an obstruction or error, the display will notify you with clear audio and visuals. For more information on the display and interface, see chapter 2.1.2.

The stairlift is equipped with various safety components to maximise your safety during use. See chapter 1.5. **Safety components** for more information.

2.1.1 Stairlift components


- | | |
|--|-------------------------------------|
| 1 Joystick | 11 Safety belt |
| 2 Footrest up button | 12 Manual swivel release (optional) |
| 3 Footrest down button | 13 Main cover |
| 4 Display | 14 Remote control |
| 5 Emergency call button (optional) | 15 Safety protection cover (rail) |
| 6 On/off switch / stop button | 16 Foldable footrest |
| 7 Backrest | 17 Safety protection Footplate |
| 8 Emergency swivel release for powered swivel. | 18 Rail |
| 9 Foldable armrests | 19 Keyswitch |
| 10 Foldable seat | 20 Keyfob |

				
1	GPRS signal strength	Weak - Strong		
				
2	Notification area	See chapter 4 for more information on notifications		
3	Power indicator	Enabled	Disabled	
				
4	Battery level	Low - High	Charging	
				
5	Wi-Fi signal strength	Weak - Strong		
				
6	Stairlift status	Locked	Maintenance	
				
7	Stairlift status	Ready	Notice	Warning
				

3 Use

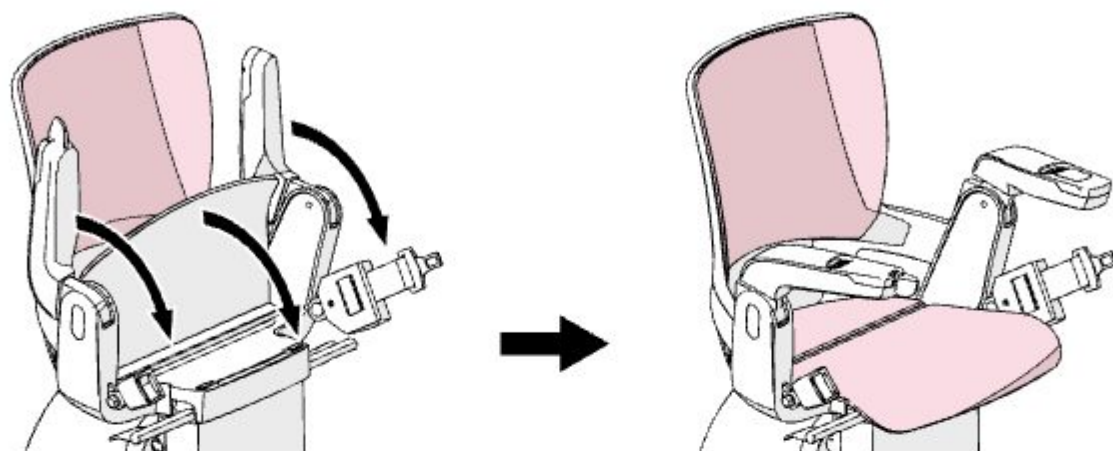
- 1 Fold down the seat and armrests.


- 2 Turn off the stairlift with the on/off button.

CAUTION!

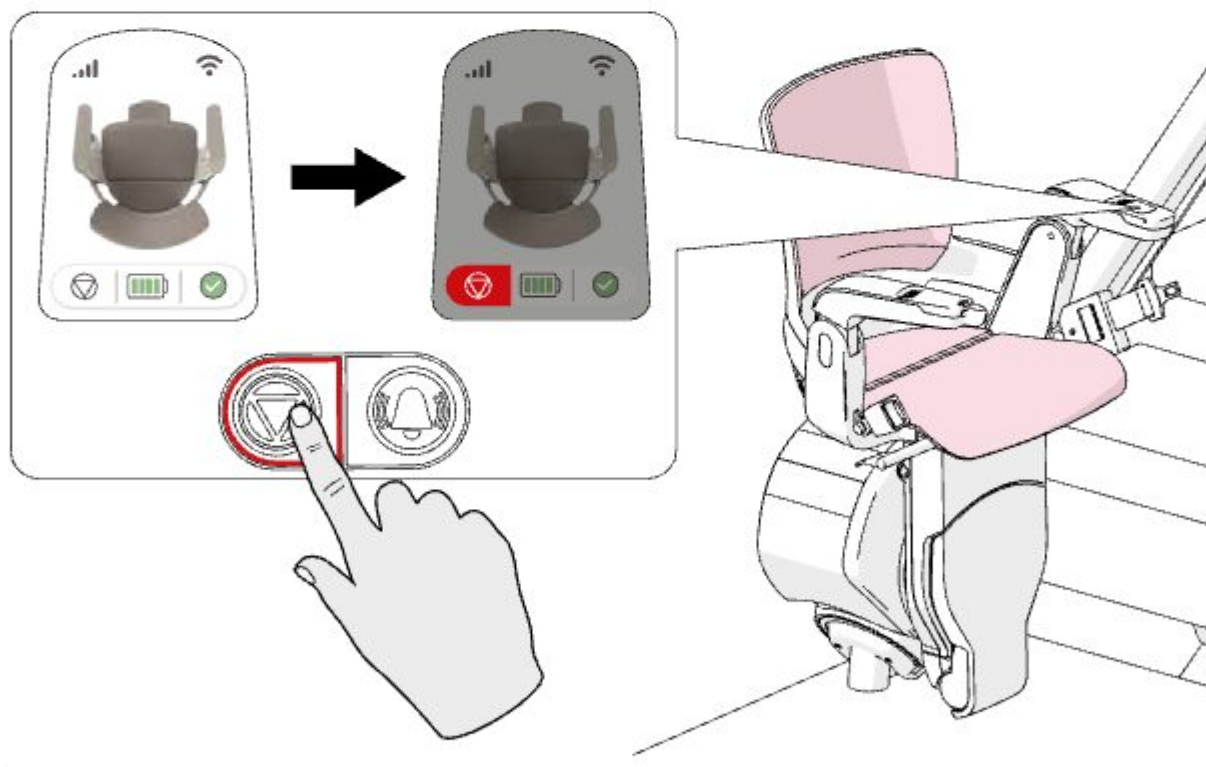If you leave the stairlift turned on, somebody could move the stairlift with a remote control unit while you are mounting the stairlift and cause you to lose your balance and fall.

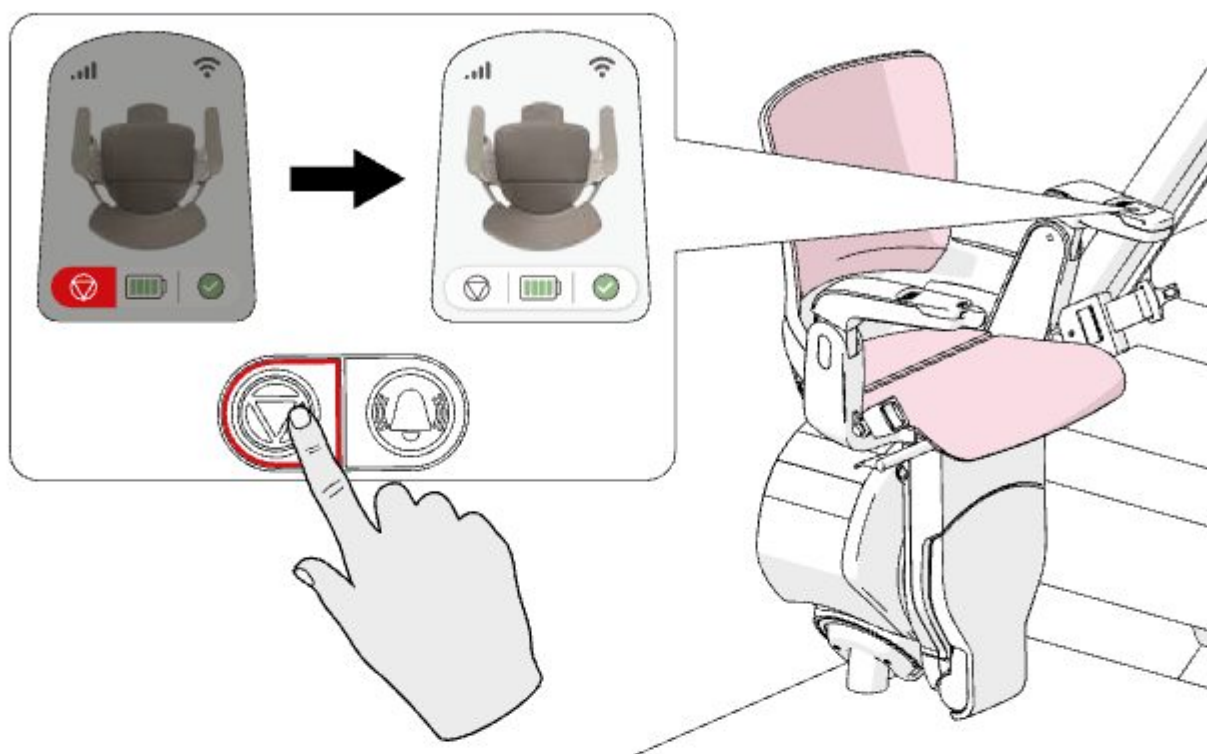
- 3 Sit down on the seat.
- 4 Turn on the stairlift with the on/off button.


- 5 Press and hold the 'footrest down' button to fold down the footrest and place your feet on the footrest.

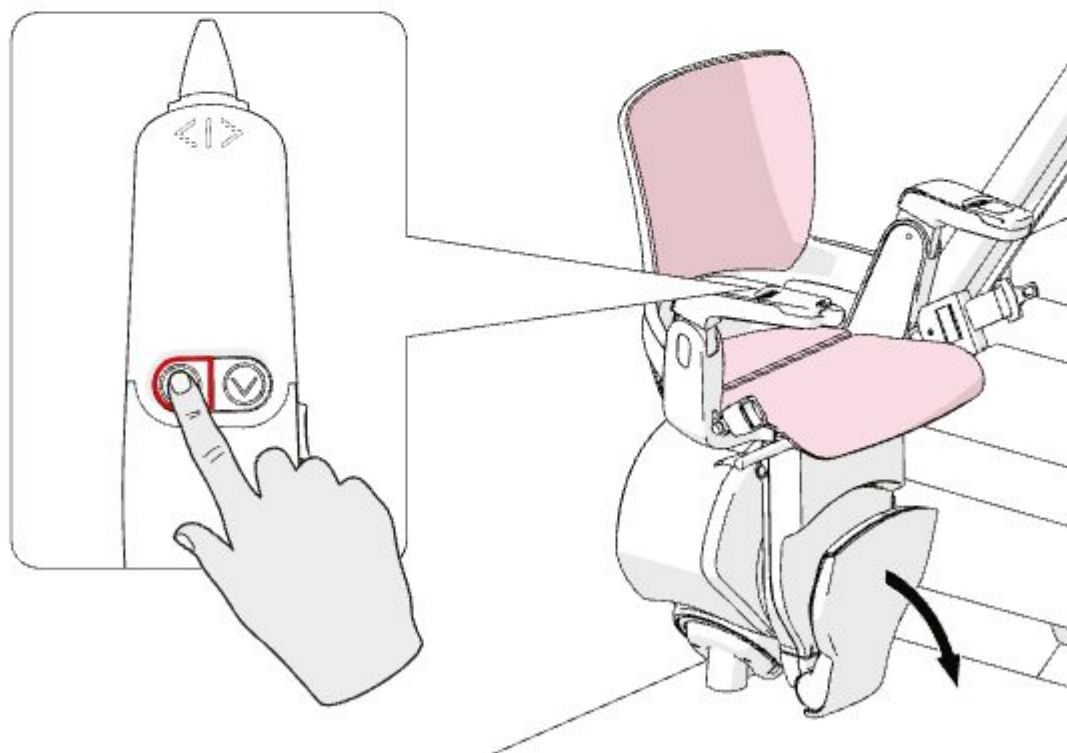Note: The footrest folds up / down electrically. Do not try to manually fold up / down the footrest. This may cause damages to the stairlift.

- 6 Fasten the seatbelt.

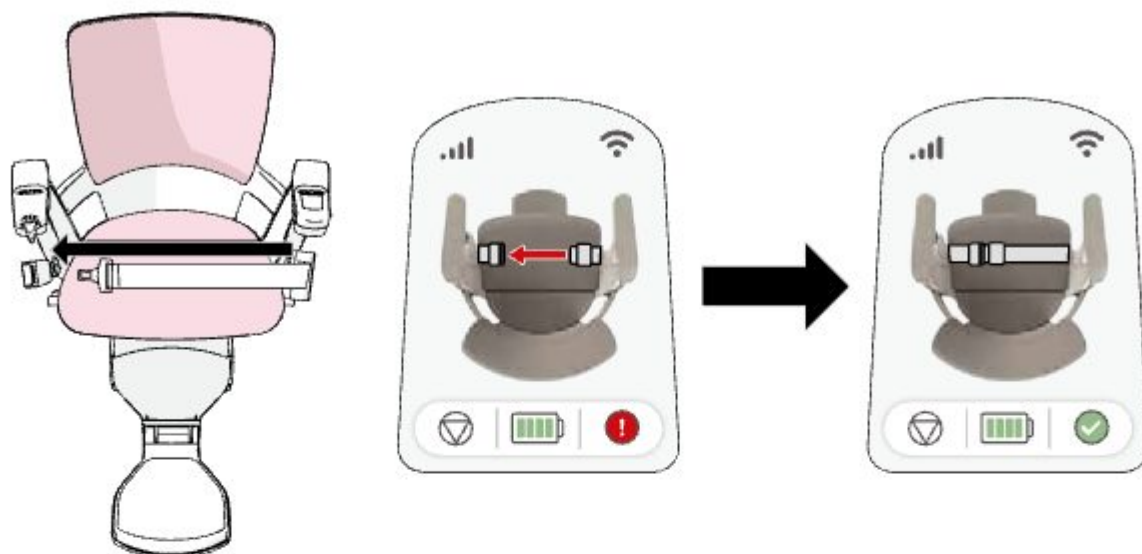Note: The stairlift will not move if you do not fasten the seatbelt.


- 7 Verify if the stairs are clear before travelling.

- 8 Tilt and hold the joystick in the direction you wish to travel. The stairlift moves over the rail as long as you hold the joystick.


Note: When you release the joystick, the stairlift immediately stops moving.

Note: If your stairlift uses rotated drive, the seat will first rotate to the drive position before starting to travel.

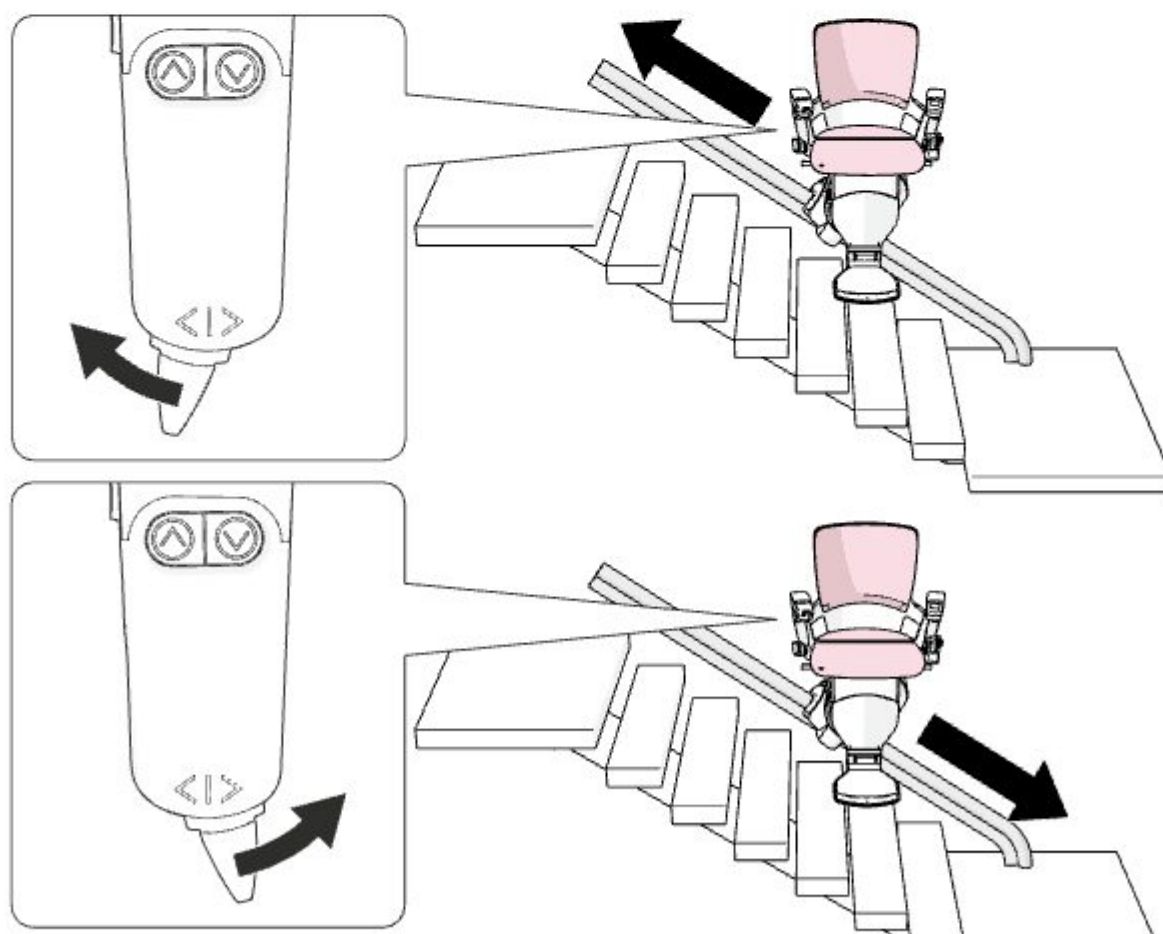Note: The footrest will not always rotate the same way as the swivel seat does.

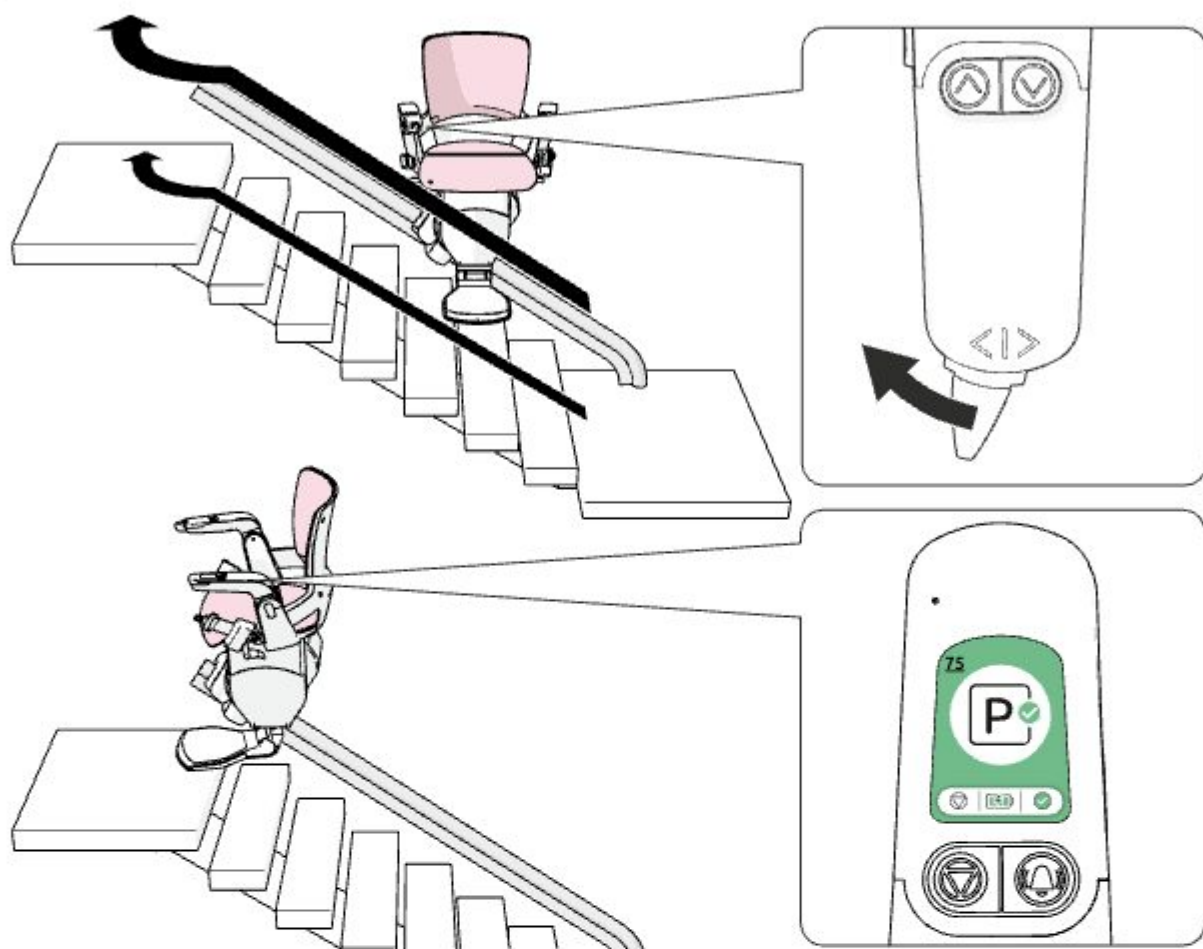- 9 Hold the joystick until you reach the end of the rail, the stairlift has swiveled to the dismounting position and the display shows the parked and charging notification.


Note: If your stairlift is not equipped with electrically powered swivel, wait until the display shows the parked and charging notification and use the manual swivel release to swivel the seat and footrest to the dismounting position.


- 10 Turn off the stairlift with the on/off button.

⚠ CAUTION!

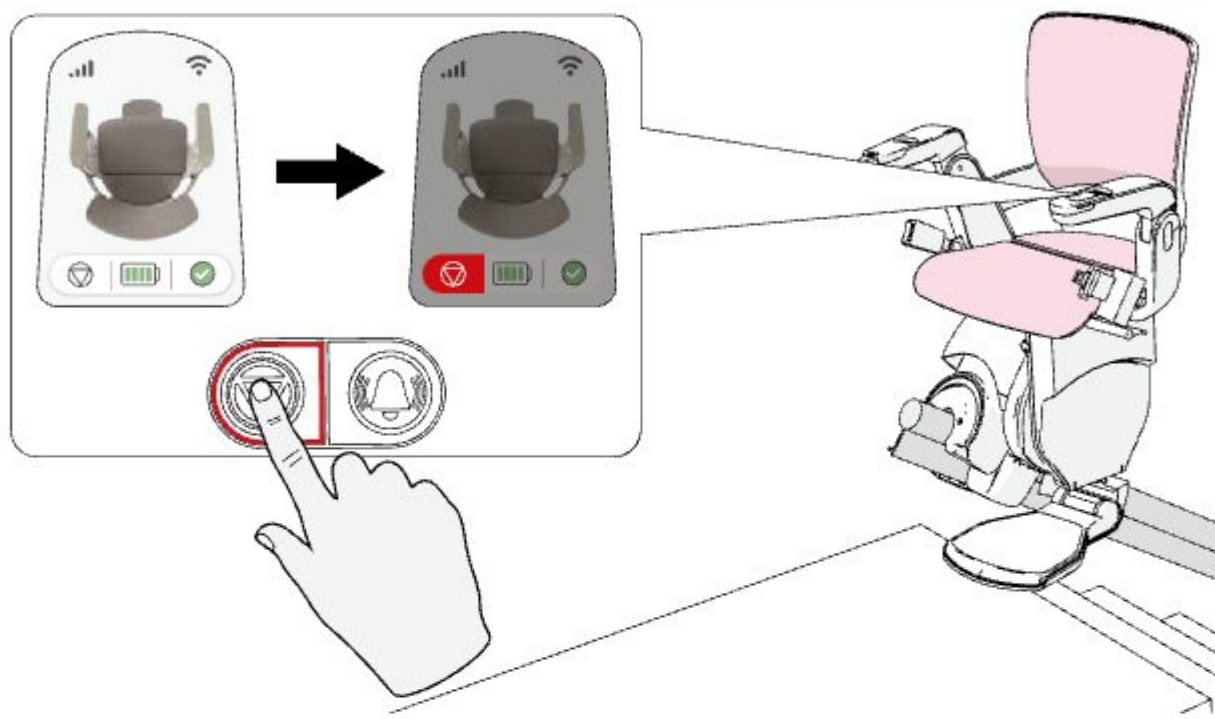
If you leave the stairlift enabled, somebody could move the stairlift with a remote control unit while you are dismantling the stairlift and cause you to lose your balance and fall.

- 11 Release the seatbelt.

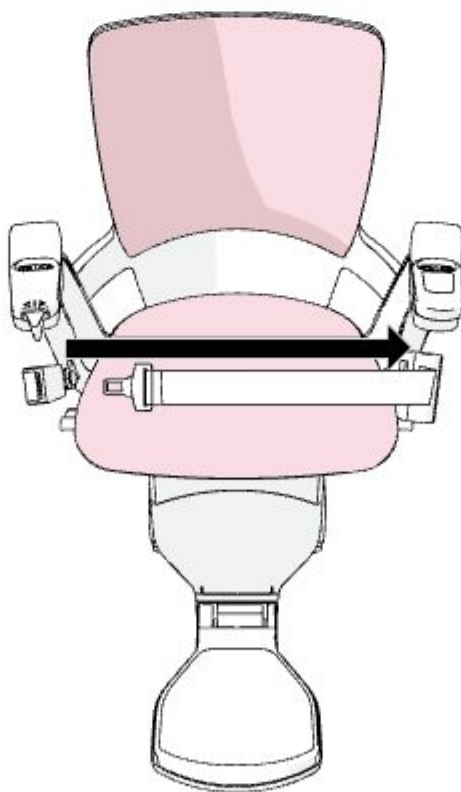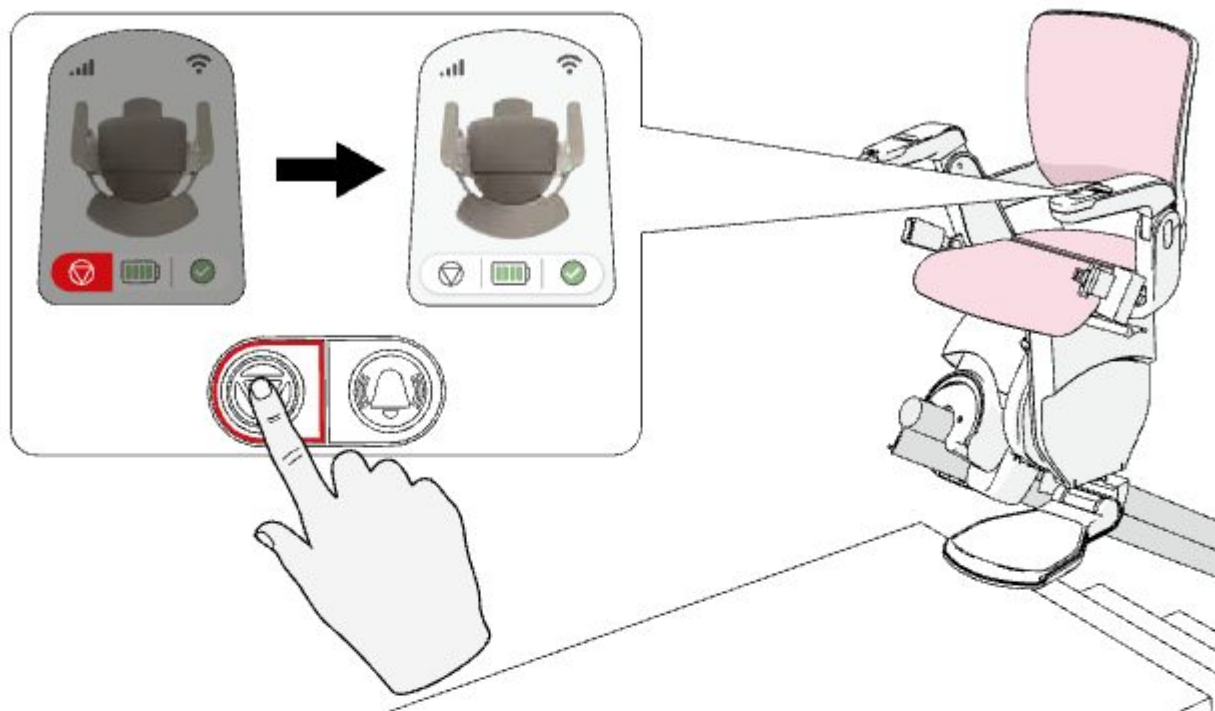
- 12 Carefully stand up and dismount the stairlift.
- 13 Turn on the stairlift with the on/off button.

- 14 Hold the 'footrest up' button to fold up the footrest.


Note: If your stairlift is equipped with an electrically powered swivel, it will first rotate back to the neutral position before folding up the footrest.

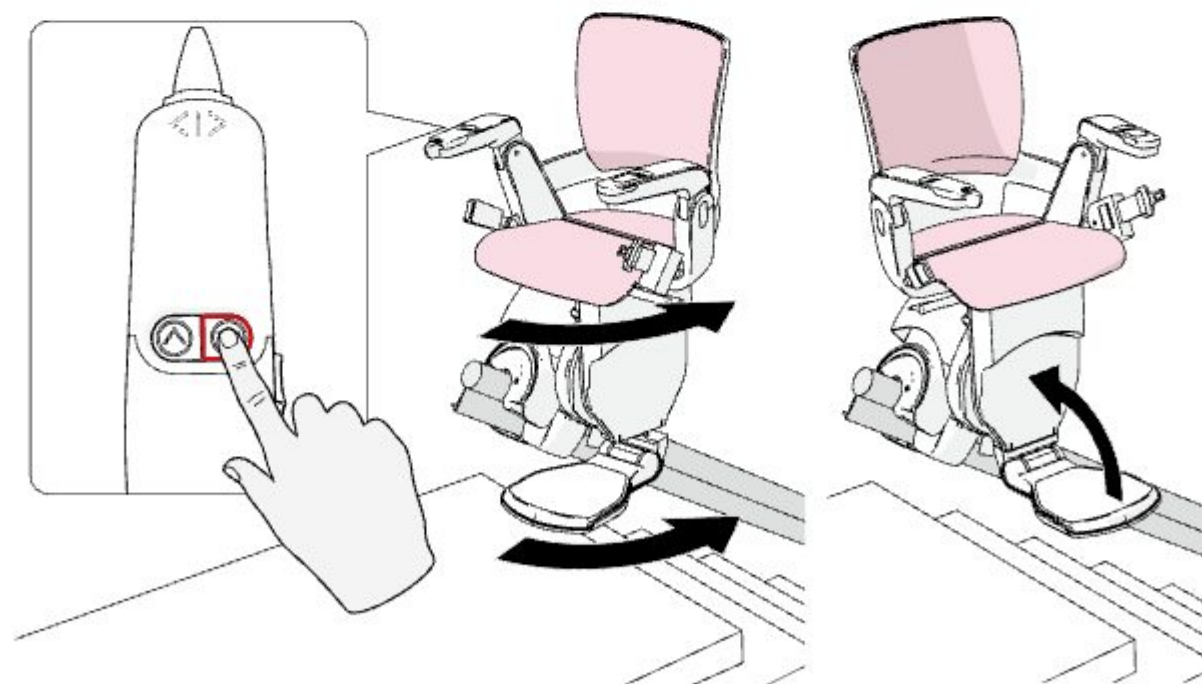Note: If your stairlift is equipped with a manual swivel seat, use the swivel seat release handle to turn the seat back to the neutral position.


15 Fold up the seat and armrests.

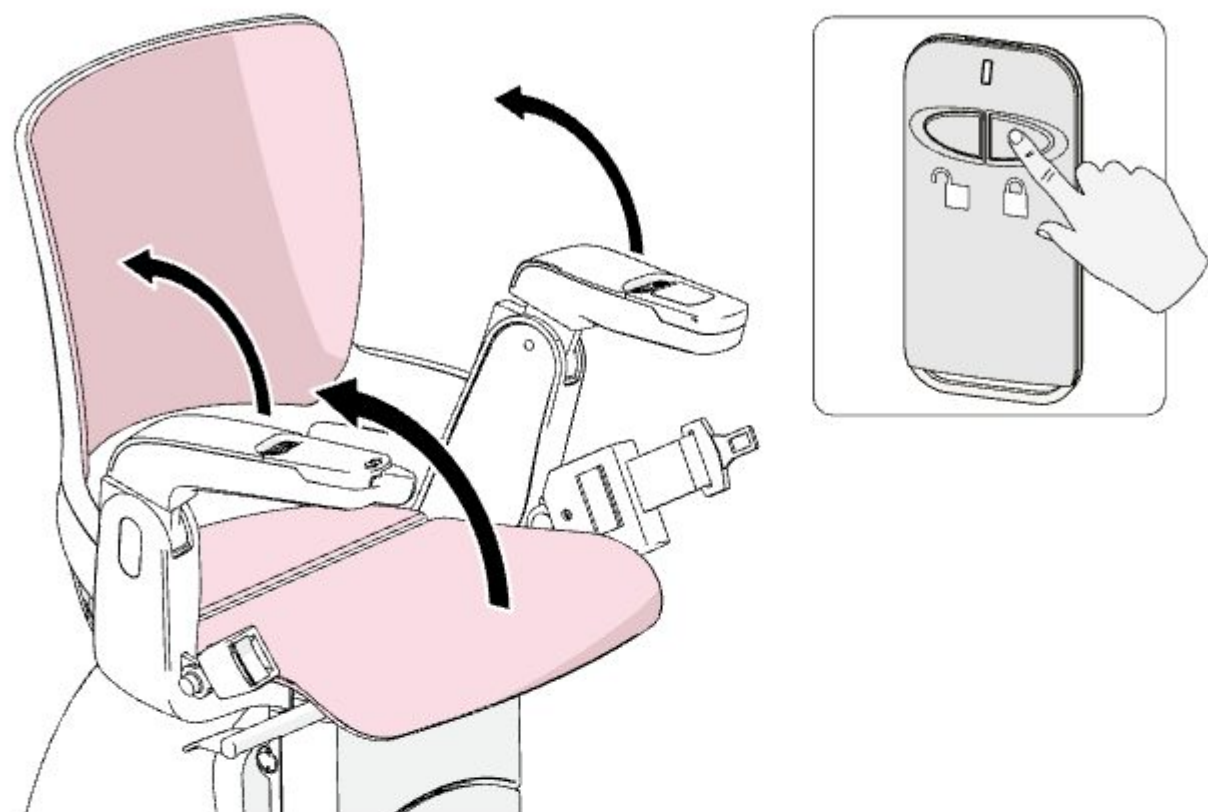Optional: If you want to ensure that the stairlift can not be used without your supervision, you can use the included key fob to remotely lock and unlock the stairlift.

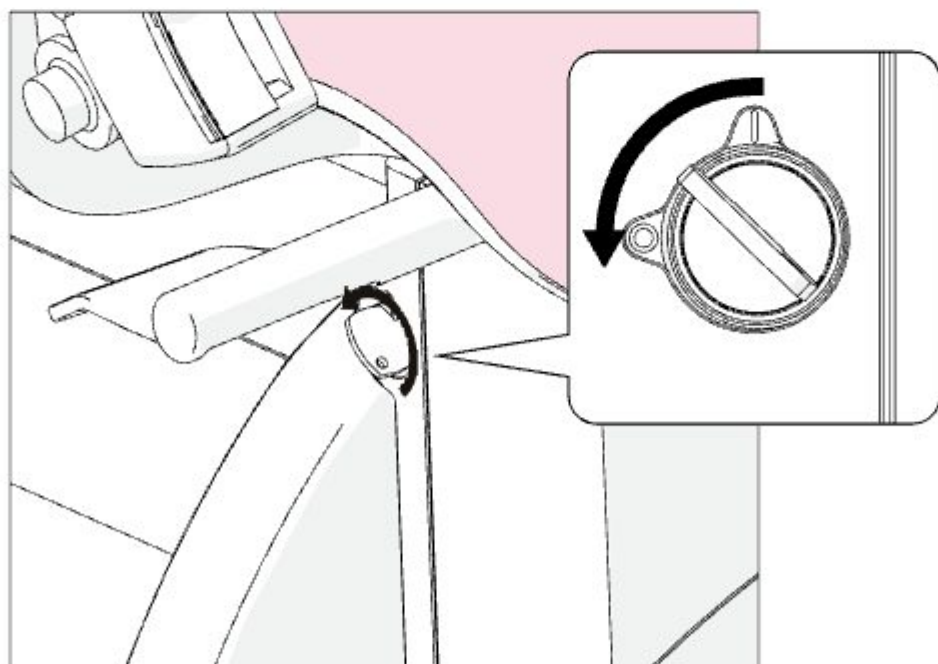Optional: If the stairlift is not in use for longer than a week, switch off the stairlift with the key switch.

Note: Switch the stairlift off and on to reset the stairlift.

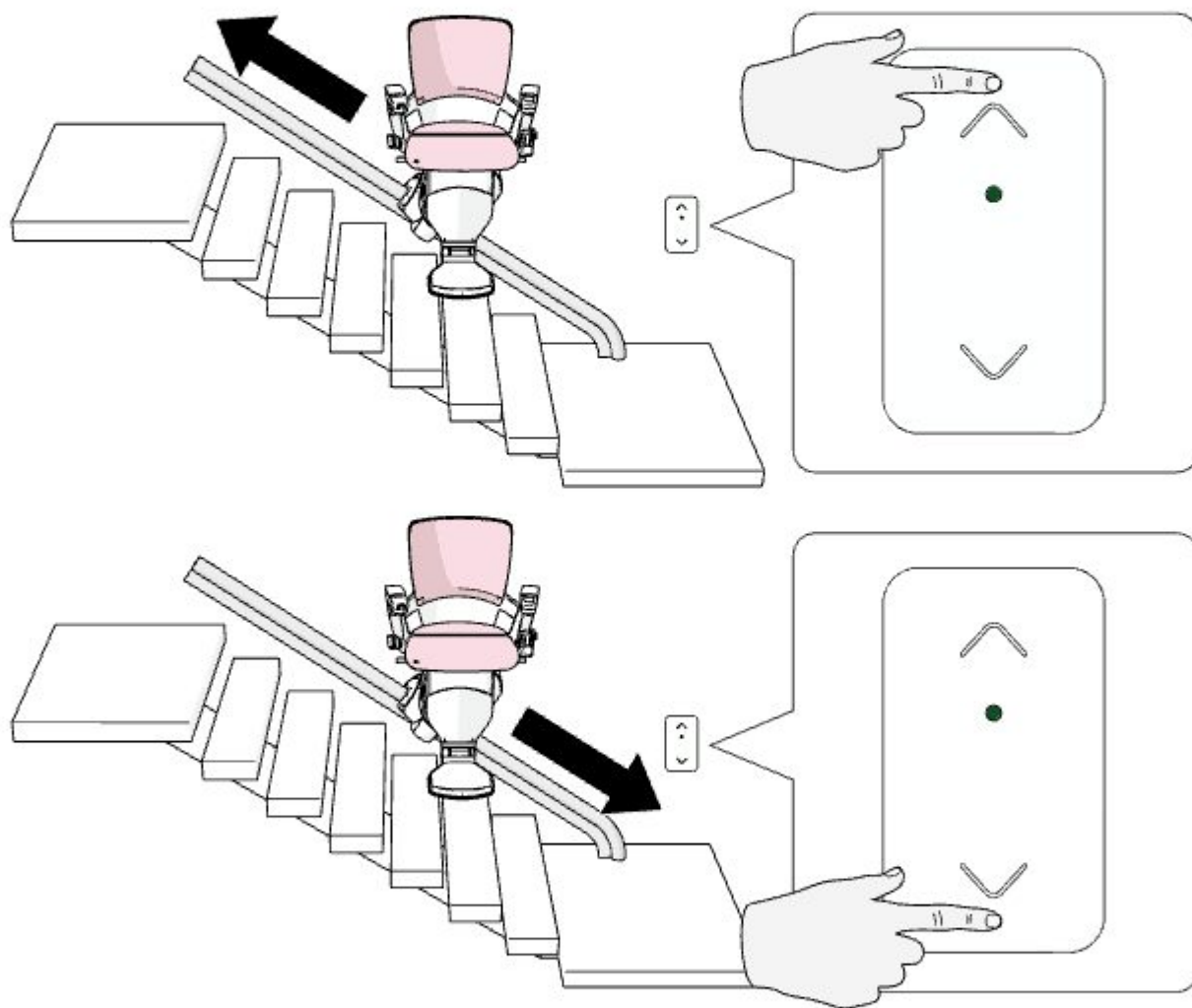
4 Move the stairlift with the remote control

You can also use the remote control to move the stairlift up or down.

- 1 Verify if the stairs are clear before moving the stairlift.
- 2 Press and hold the remote control in the direction you wish to move the stairlift.


Note: If your stairlift has the rotated drive option and the armrests are folded down, the seat will rotate to the drive position before starting to travel. If the armrests are folded up, the stairlift will move in the folded state.

Note: If you have an extra charging point installed on the rail, you can use the remote control to move the stairlift to the charging point. Hold the remote control up or down button until the stairlift automatically stops at the charge point and you hear the notification sound indicating that the stair lift has reached the charge point and is charging.

5 In case of a power outage or malfunction of the stairlift

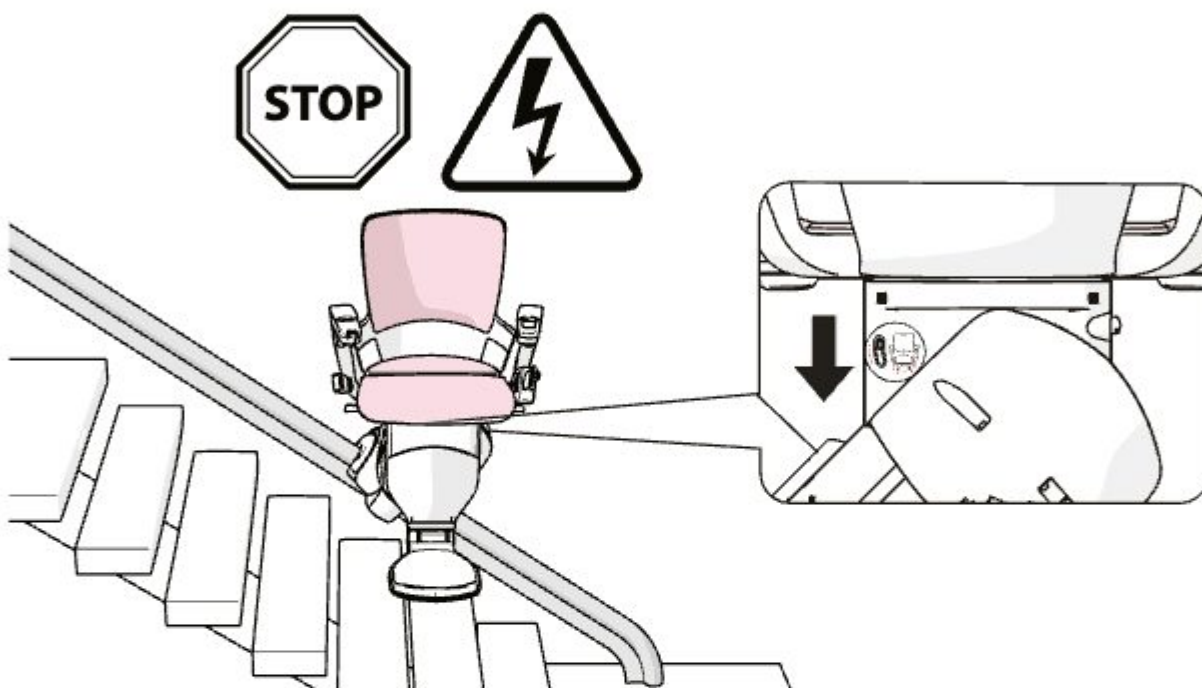
- If your stairlift is equipped with an electrically powered swivel seat, use the emergency swivel release to swivel the seat towards the stairs to safely dismount the stairlift.
- If your stairlift is equipped with a manual swivel seat, use the manual swivel release handle to swivel the seat towards the stairs to safely dismount the stairlift.

6 Optional: Calling for assistance

- 1 Press and hold the yellow call button for ~2 seconds until you hear a beep. The display now shows a list of pre-programmed contacts.
- 2 Press the yellow call button to cycle through the contacts.

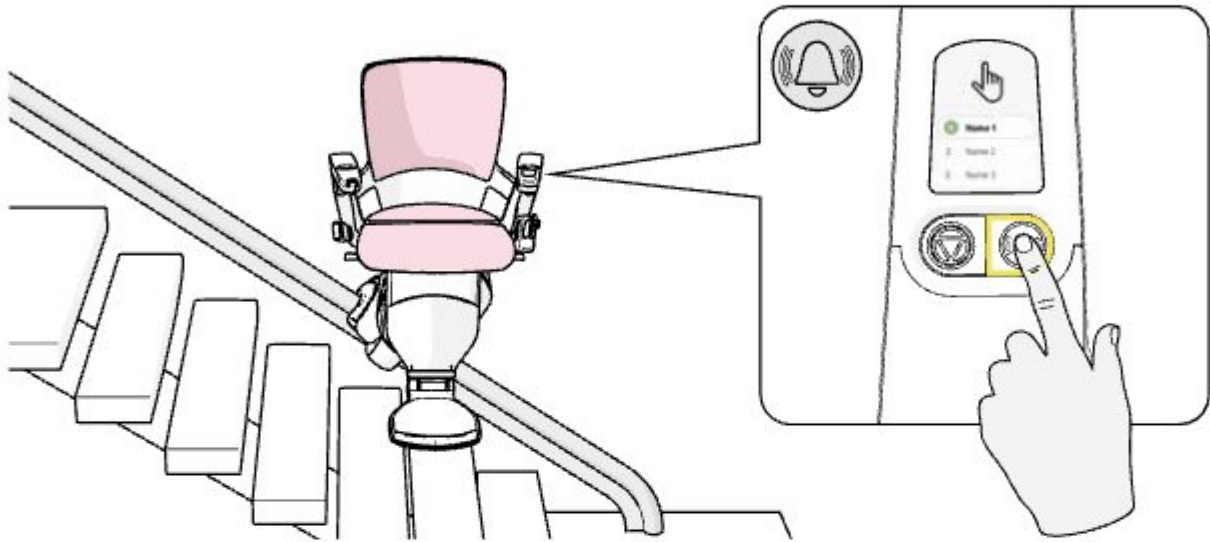After 3 seconds, the selected contact will automatically be dialled.

- 3 Press the yellow call button to end the current call.

Note: When initiating a new call right after ending one, the next contact in the list will be highlighted.

Note: The dial function is still operational in case of a display malfunction.

7 Display notifications and alarms

The display in the armrest informs you of the current status of the stairlift in the form of a notification. The stairlift can also emit audio signals. Check the overview in this chapter for explanations of the display notifications.

7.1 Current status

General

The stairlift is ready for use.

The stairlift is locked for use by the key fob unit.

The stairlift is moving to the left.

The stairlift is moving to the right.

The footrest is moving up.

The footrest is moving down.

The stairlift seat is rotating.

The seatbelt is fastened.

The stairlift is in use.

Connections

The stairlift has no wi - fi connection.

The stairlift has good wi - fi connection.

The stairlift has no DECT signal.

The stairlift has good DECT signal.

Faults

The stairlift is disabled.

The seatbelt is not fastened.

The armrests are not (fully) folded down.
Push the armrest down.

The footrest is not (fully) folded down.
Press and hold down the 'footrest down' button.

The seat is not (fully) folded down.
Sit down on the seat.

The seat is swivelled outside of the starting position.
Swivel the seat back to the starting position.

Faults

The stairlift is due for periodic maintenance.

Obstruction faults

The footrest is obstructed from travelling in the marked direction.

Travel the other way and remove the obstruction to continue.

The stairlift is obstructed from travelling in the marked direction.

Travel the other way and remove the obstruction to continue.

7.2 Alarms on the display

Remote locking

The stairlift is locked for use by the key fob unit.

The stairlift is locked for use by the keyswitch.

Parking

The stairlift is in the parking position and is charging.

The stairlift is in the parking position, but is not charging. Check if the power cable is properly connected to a working power outlet.

The stairlift is not in a parking position and is not charging. Move the stairlift further to the parking position to avoid draining the battery.

Battery status

The stairlift is fully charged.

The stairlift battery is low. Move the stairlift to a charging position and allow the stairlift to charge before using the stairlift again.

The battery of the stairlift is drained. Try to move the stairlift to a charging position and allow the stairlift to charge before using the stairlift again. If the stairlift is unmovable using the methods described in this manual, please contact customer service to schedule a mechanic.

Obstructions/faults

The stairlift is obstructed from travelling in the marked direction while the joystick is pushed in that direction.

- Fault 20: Left side
- Fault 21: Right side

Travel the other way and remove the obstruction to continue.

The stairlift is obstructed from travelling in the marked direction while the joystick is pushed in that direction.

- Fault 25: Left side
- Fault 26: Right side

Travel the other way and remove the obstruction to continue.

The footrest is obstructed from travelling in the marked direction while the stairlift is travelling.

- Fault 30: Left side
- Fault 31: Right side

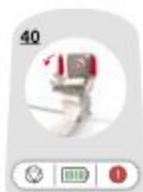
Travel the other way and remove the obstruction to continue.

The footrest is obstructed from folding.

- Fault 35: The footrest is not folding up
- Fault 36: The footrest is not folding down

Remove the obstruction and press the 'footrest up' or 'footrest down' button to (un)fold the footrest.

The armrests are not (fully) folded down while the stairlift is travelling.

- Fault 40: Left side
- Fault 41: Right side

Push the armrest down.

Obstructions/faults

Electrically powered swivel seat has not reached the turning point.
Fold the footrest up and down to reset.

Electrically powered swivel seat is blocked while swivelling.
Fold the footrest up and down to reset.

The seat is not (fully) folded down.
Sit down on the seat.

The seatbelt is not fastened while the stairlift is travelling.
Fasten the seatbelt and operate the stairlift again.

The stairlift is defective and emits a beeping alarm. Operate the stairlift again with the joystick or remote. Turn the key switch off and on to reset the stairlift. Avoid additional service costs by following the steps above. If this does not solve the problem, please contact customer service to schedule a mechanic.

Other

The stairlift is due for periodic maintenance.
Please contact customer service to schedule a mechanic.

The battery is due for replacement.
Please contact customer service to schedule a mechanic.

The stairlift has no wi-fi connection.

8 Maintenance

8.1 Cleaning

It is recommended to clean your stairlift at least once every week. Use a mild cleaner and a soft brush or cloth to clean the seat. Do not use degreasers or chlorine - based, silicone based or aggressive cleaners. Use a brush to clean the teeth of the rail and lotion wipes to keep the rail tubes clean. Avoid excessive use of water.

8.2 Maintenance schedule

The stairlift requires periodic preventive maintenance to ensure long - term, trouble - free operation. It is recommended that your stairlift be checked and serviced every 12 months. The following components need to be checked at least once every two years: balance sensor, OSG (catch) and the plastic wheels. Inspection must be performed by an Otolift authorised service technician.

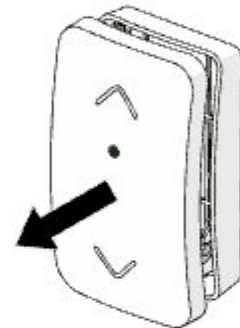
8.3 Replacing the remote control battery

The wall - mounted remote control is powered by a single type AA battery. The light on the front of the remote control quickly blinks green when the battery is low.

You can easily replace the battery:

- 1 Carefully pull the front cover from the back housing.
- 2 Remove the empty battery.
- 3 Place a full type AA battery in the battery holder.

Note: Make sure to match the polarity of the battery with the markings on the remote control.

8.4 Replacing the key fob battery

The included key fob control is powered by a single type C2032 battery cell.

If the stairlift no longer locks or unlocks with the key fob, the battery of the key fob might be low.

- 1 Remove the battery cover from the back of the key fob housing.
- 2 Remove the empty battery.
- 3 Place a full type C2035 battery cell in the battery holder.

Note: Make sure to match the polarity of the battery with the markings on the key fob.

9 Disposal

The stairlift contains two batteries, each 12 V (7 Ah), to provide the stairlift with power. These batteries may only be replaced by authorised maintenance technicians. The remote controls are equipped with a penlight AA battery. You may replace these batteries yourself. Do not dispose of old batteries with domestic refuse, but hand them in when you buy the new batteries.


When an Otolift stairlift is no longer required the supplying dealer may take it back. However this is purely at their discretion. They are under no obligation to do so. Contact them in the first instance for advice. If this is not possible, find out from your local government administration what the possibilities are for environmentally friendly disposal of the materials.

10 Technical data

General	
Maximum load capacity	125 kg
Battery capacity	7.0 Ah
Fire resistance	EN81 - 40 - according to 5.1.4. Fire resistance
Sound emission	<50 dB(A)
Electrical system	
Power voltage	24 V; 1.4 A
Mains	230V; 33.6W; 0.15 A / 115V; 33.6W; 0.29 A / 240V; 33.6W; 0.14 A (Selected by region, check power supply)
Ambient conditions	
Ambient temperature	-15 – 50°C
Relative humidity	Max 90% RH (non - condensing)

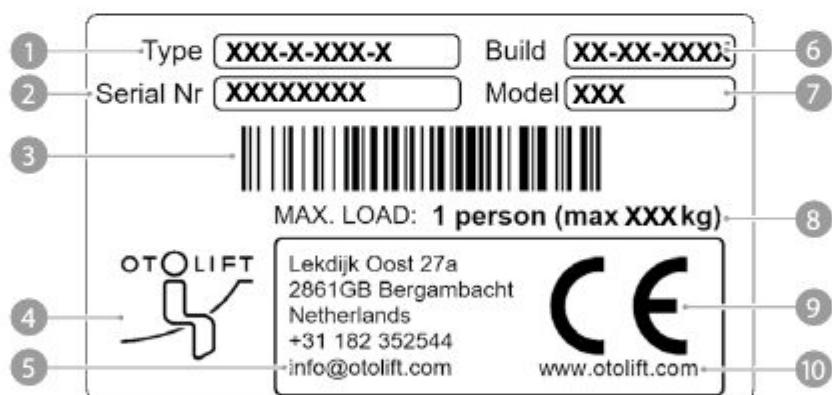10.1 Type plate

- | | |
|--------------------------------|----------------------------|
| ① Type code | ⑥ Date of production |
| ② Serial number | ⑦ Model name |
| ③ Barcode | ⑧ Maximum rated load |
| ④ Manufacturer logo | ⑨ Marking of CE compliance |
| ⑤ Manufacturer contact details | ⑩ Manufacturer website |

11 EC declaration of conformity for machinery

We,

Otolift Trapliften B.V.
Lekdijk Oost 27a
2861 GB Bergambacht
The Netherlands

Declare, under our own responsibility, that the Otolift stairlift to which this declaration refers is in full accordance with directive:

2006/42/EC

Standards applied:

EN 81-40:2020, NEN 14121-2:2012, EN-IEC 61508-1/7:2010, 2014/30/EU

Certification Institute:

Liftinstituut B.V. (Identification number Notified Body 0400)
Buikslotermeerplein 381
1025 XE, Amsterdam
The Netherlands

Certificate no.:

NL23-400-1001-101-06 rev

Person authorised to compile the technical file:

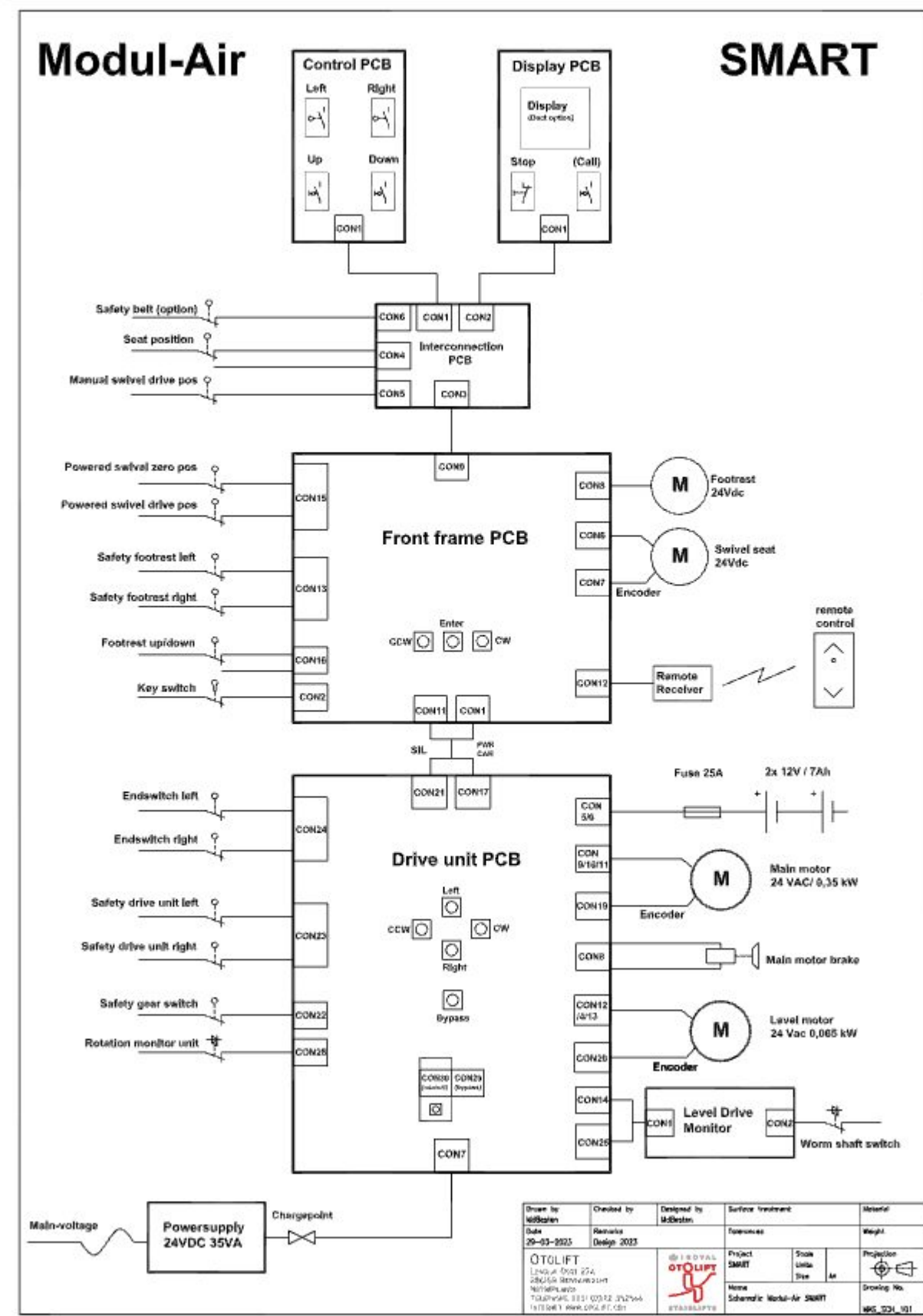
M.A. Stam
Senior Product Engineer
Lekdijk Oost 27a
2861 GB Bergambacht

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2860 AB Bergambacht