



Hersteller:

MMN Medizin-Mechanik-Nord GmbH
Russeer Weg 54a
D-24111 Kiel
Deutschland
Tel. +49(0)431-6902814
Fax: +49(0)431-6902815
Mobil: +49(0)170-4446009
E-Mail: info@medizin-mechanik.de




Anhang zu Gebrauchsanweisung Version 2.0_DE_EN

Montage Selbsthaltender Schraubendreher mit Quick Fix


1. Verriegeln



2. Implantat verriegeln

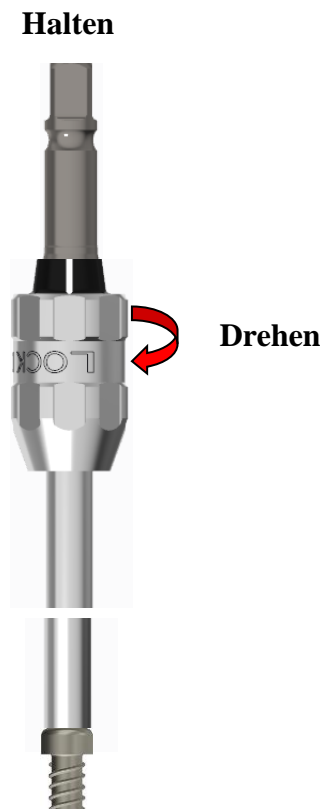

3. Entriegeln

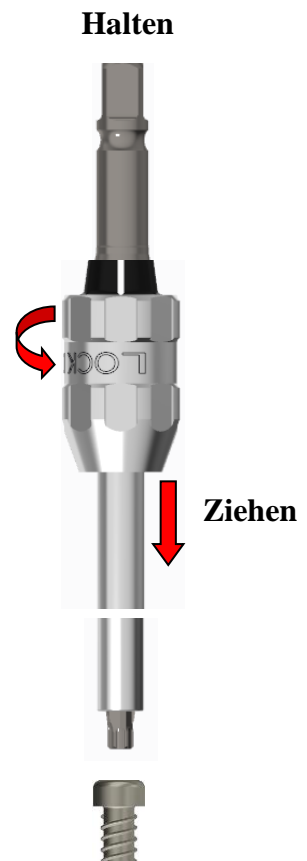

OP-Schwester

- Quick Fix mit Schraubendreher

vormontiert

Operateur

- Schraube aufsetzen
- Quick Fix drehen
- Schraube verriegelt

bereit für die OP

Operateur

- Quick Fix drehen
- Quick Fix abziehen

bereit zur Reinigung



Manufacturer:

MMN Medizin-Mechanik-Nord GmbH
Russeer Weg 54a
D-24111 Kiel
Germany
Tel. +49(0)431-6902814
Fax: +49(0)431-6902815
Mobil: +49(0)170-4446009
E-mail: info@medizin-mechanik.de




Appendix to Instructions for Use Version 2.0_DE_EN

Mounting Selfretaining Screwdriver with Quick Fix


1. Lock



2. Lock implant

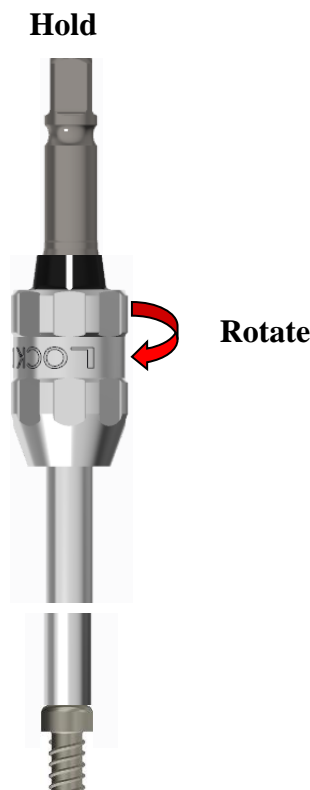

3. Unlock

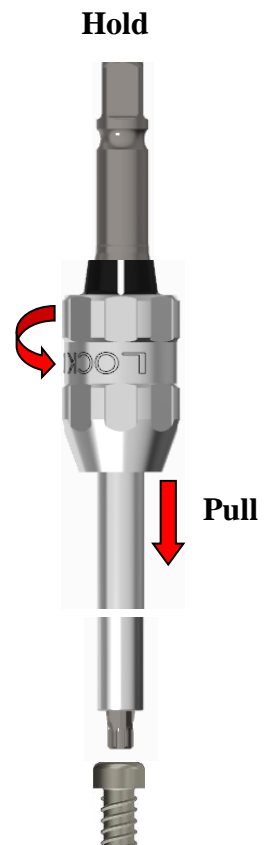

OR-Nurse

- Quick Fix with Screwdriver

pre - assembled

Surgeon

- Put on screw
- Rotate Quick Fix
- Screw locked

ready for the surgery

Surgeon

- Rotate Quick Fix
- Pull off Quick Fix

ready for cleaning