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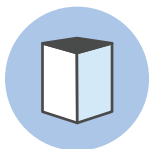
1. Let's get organized.

Get familiar with the kit before we get down to business.

The **Urine Test** should contain these items:

One collapsible urine
collection cup

One pipette

One collection tube
(no preservative liquid)

Write your name, date of birth, and collection date in MM/DD/YYYY format on the collection tube in the designated area. Now you're ready to start self testing!

Note that you should try not to urinate for an hour prior to collection (this makes sure you have enough volume for collection).

2. Now onto the sample collection.

It might look intimidating, but we have faith in you. You got this!

Wash your hands

First things first: wash your hands with warm water and soap.

Open the urine collection cup and let it stand

Take the flattened urine collection cup by its two edges. Push them inward so that it forms a diamond. Stand the cup upright.

Urinate into cup

Take the urine collection cup, holding the sides. Make sure that the bottom is facing down. Urinate directly into the collection cup, and fill it up 3/4 of the way.

Transfer urine into tube

Take the pipette and use it to transfer your urine sample from the cup into the collection tube (**without preservative liquid**). Fill the tube 2/3 of the way. Once you're done with this, you can dispose of the urine cup.

Seal and package sample

Place the lid onto the tube and ensure that it's closed evenly and tightly. Put the tube in the large biohazard bag, and make sure the seal is closed completely. Congrats, you're done! Keep the sample out of extreme temperatures (above 100F or below 32F) before you ship.

3. Pack it up!

Go over the checklist below to ensure your sample is ready to ship.

- ☐ Used all equipment for the Urine Test to collect sample
- ☐ Wrote your name, date of birth, and collection date on the tube
- ☐ Placed sealed tube in the large biohazard bag
 - *You can put more than one tube in this bag
- ☐ Put bagged sample back into the kit box

To return your kit, simply close the box, place the box inside the return envelope, and drop off at your local mail carrier.

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