Blood Spot Card

Blood Test

Instructions

Instrucciones

para análisis

de sangre

1. Let's get organized.

Get familiar with the kit before we get down to business.

The **Blood Test** should contain these items:

Blood collection cards
(up to two)

Alcohol pads

Sterile gauze pad

Single use lancets

Adhesive bandages

Small specimen bag
with desiccant pack

Take the blood collection card(s) and write your **name, date of birth, and collection date in MM/DD/YYYY format** where prompted.


2. Now onto the sample collection.

It might look intimidating, but we have faith in you. You got this!


Warm up, prep your test, and wash your hands

Get your blood flowing by drinking plenty of water beforehand! Do a set of jumping jacks, or swing your arms by your side. Run your hands under warm water. Open the flap of the blood card to expose the blood collection paper, being careful not to touch the collection paper itself. Then give your hands a good wash with soap and warm water.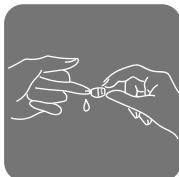


Clean your fingertip and prick with lancet

Clean your fingertip with the alcohol pad. Try using the middle or ring finger of your non-dominant hand. Then, take a lancet and twist the cap off. Press the small tip into the side of your finger near the tip, until the needle ejects with a click. Keep your hand below your heart to help the blood flow.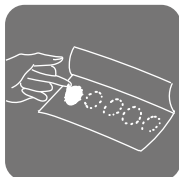

Fill collection paper circles

Apply 3-6 drops of blood to each circle on the collection paper so the blood soaks all the way through the paper. Start with one, and once the circle is full, move onto the next circle. Make sure all of the circles are filled completely, but don't let the blood spots run into each other. Don't add additional blood to circles once it has dried--this will invalidate the test!

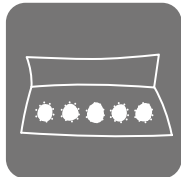Check paper, repeat for second card if applicable

Check the back side of the blood collection paper to make sure that the blood has saturated all the way through and filled each circle. Repeat the above steps for the second collection card if applicable. Then, let the paper for both cards air dry on a flat surface for one hour, or until completely dry.

Package samples

Close the blood collection cards after they dry, and place them into the small biohazard bag with the desiccant pack. Congrats, you're done! Keep the sample out of extreme temperatures (above 100F or below 32F) before you ship.

3. Pack it up!

Go over the checklist below to ensure your sample is ready to ship.

- ☐ Signed your name, date of birth, and collection date on the card
- ☐ Filled all collection circles completely on the collection card
- ☐ Placed the card in the small biohazard bag with the desiccant pack
*You can put more than one card in this bag
- ☐ Put bagged sample back into the kit box

To return your kit, simply close the box, place the box inside the return envelope, and drop off at your local mail carrier.

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