Vaginal, Rectal, and Oral Sample Collection Instructions

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Before You Get Started


Check the label on the collection tubes before you start. Follow the instructions for the test type indicated on the label. For example, read the "Rectal Test" section of this booklet if your collection tube indicates a rectal test kit.

Vaginal Test

1. Let's get organized.

Get familiar with the kit before we get down to business.

The **Vaginal Test** should contain these items:

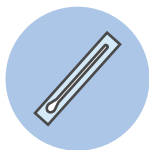One swab

One collection tube
(preservative liquid
included)

Write your name, date of birth, and collection date in MM/DD/YYYY format on the collection tube in the designated area. Now you're ready to start self testing!

Note that if you're on your period or have had sex within the last 24 hours, you should wait until your period ends and/or it's been more than 24 hours since you had sex to collect your sample.

2. Now onto the sample collection.

It might look intimidating, but we have faith in you. You got this!

Wash your hands

Remember to wash your hands! Now remove the swab from its wrapper, and be careful not to touch the tip anywhere before insertion.

Insert swab into vagina

Using your non-dominant hand, spread your vaginal lips so that the opening to your vagina is easily accessible. Hold the swab about 1 inch from the end with your dominant hand, and insert the swab into your vagina about 1-2 inches (3-5 cm).

Swirl the swab

Gently circle the swab for 20 seconds with pressure on the walls of your vagina, until the swab is moist.

Remove and seal

Remove the swab and immediately insert into the collection tube, using extra caution not to allow the tip to touch anything. Break the swab at the score line by bending it against the collection tube. Place the lid onto the tube and ensure that it's closed evenly and tightly.

Package sample


Put the tube in the large biohazard bag, and make sure the seal is closed completely. Congrats, you're done! Keep the sample out of extreme temperatures (above 100F or below 32F) before you ship.

Rectal Test

1. Let's get organized.

Get familiar with the kit before we get down to business.

The **Rectal Test** should contain these items:

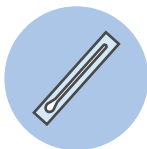One swab

One collection tube
(preservative liquid
included)

Write your name, date of birth, and collection date in MM/DD/YYYY format on the collection tube in the designated area. Now you're ready to start self testing!

2. Now onto the sample collection.

It might look intimidating, but we have faith in you. You got this!


Wash your hands

Give your hands a good wash. Then unwrap the swab from its wrapper and be careful not to touch the tip anywhere before insertion into your anus.

Insert swab into anus

Use your non-dominant hand to gently push your buttock so you can easily insert the swab. Insert about two inches in.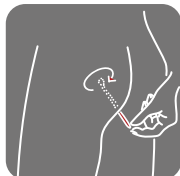

Swirl the swab

Gently move the swab in a circular motion against the lining of your rectum for 10-15 seconds.

Remove and seal

Remove the swab and immediately insert into the collection tube, using extra caution not to allow the tip to touch anything. Break the swab at the score line by bending it against the collection tube. Place the lid onto the tube and ensure that it's closed evenly and tightly.

Seal and package sample


Put the tube in the large biohazard bag, and make sure the seal is closed completely. Congrats, you're done! Keep the sample out of extreme temperatures (above 100F or below 32F) before you ship.

Oral Test

1. Let's get organized.

Get familiar with the kit before we get down to business.

The **Oral Test** should contain these items:

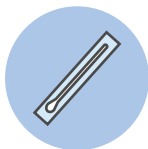One swab

One collection tube
(preservative liquid
included)

Write your name, date of birth, and collection date in MM/DD/YYYY format on the collection tube in the designated area. Now you're ready to start self testing!

Note that you should not eat, drink, or smoke for 30 minutes prior to collection of the oral sample.

2. Now onto the sample collection.

It might look intimidating, but we have faith in you. You got this!

Wash your hands

Remember to wash your hands! Now remove the swab from its wrapper, and be careful not to touch the tip anywhere before insertion.

Open wide and swab throat

Using a spoon or a tongue depressor, push down your tongue so it's out of the way (try saying "AHHHH" if you're struggling at first). Carefully insert the swab into your mouth, being careful not to touch your tongue or the sides of your mouth. Then lightly swab the back of your throat.

Remove and seal

Slowly remove the swab from your mouth, being mindful not to touch anything on the way out, and insert it into the collection tube. Break the swab at the score line by bending it against the collection tube. Place the lid onto the tube and ensure that it's closed evenly and tightly.

Oral Test - English

Package sample

Put the tube in the large biohazard bag, and make sure the seal is closed completely. Congrats, you're done! Keep the sample out of extreme temperatures (above 100F or below 32F) before you ship.

Pack it up!

Go over the checklist below to ensure your sample is ready to ship.

- ☐ Used all equipment for the three tests to collect samples
- ☐ Wrote your name, date of birth, and collection date on the tubes
- ☐ Placed sealed tubes in the large biohazard bag
 - *You can put more than one tube in this bag
- ☐ Put bagged samples back into the kit box

To return your kit, simply close the box, place the box inside the return envelope, and drop off at your local mail carrier.

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