

Flexible.
Comfortable.
Minimal.

The ResMed AirFit™ F40



Live life in high Res

AirFit™
F40

Help patients live a brighter life with better sleep

The ResMed AirFit™ F40 builds on what you and your patients love about ResMed masks. Featuring a fully flexible frame with a soft, total silicone AdaptiSeal™ cushion and fewer adjustment points, the AirFit F40 is designed for easy fitting and comfort. Its ultra-compact under-the-nose design and adaptive seal means patients can enjoy sleeping in any position.

98%
Fit range¹

New AdaptiSeal™ full face cushion

Soft silicone adapts to the face, designed to maintain seal.

Quick-release elbow

Makes it easy to connect and disconnect to therapy.

Short tube

Reduces drag and moves with the patient.



Headgear adjustment points 1 2 3 4

Two upper and two lower adjustment points simplifies fitting and use.

Flexible mask frame

Plastic frame covered by a soft nylon fabric padding. The frame sits below the eyes, designed for comfort and to reduce facial marks.

Magnetic clips*

Conveniently and intuitively guide the headgear into the frame in a snap.

QuietAir™ diffused venting

Integrated into the cushion to provide a more peaceful bedroom environment.



AirFit™ F40

Complete mask system

Product name	Product code
AirFit F40 System – SW/STD APAC	64645
AirFit F40 System – M/STD APAC	64646
AirFit F40 System – L/STD APAC	64647

Starter Pack

Product name	Product code
AirFit F40 Starter Pack – STD STPK APAC	64650

Frame and Elbow System

Product name	Product code
AirFit F40 ELBOW SYS QUIET UDI	64638
AirFit F40 Frame	64641

Cushion with QuietAir vent ring

Product name	Product code
AirFit F40 Cushion with QuietAir vent Small Wide	64627
AirFit F40 Cushion with QuietAir vent Medium	64628
AirFit F40 Cushion with QuietAir vent Large	64629

Headgear

Product name	Product code
AirFit F40 Headgear SML	64633
AirFit F40 Headgear STD	64634
AirFit F40 Headgear LGE	64635

*This mask contains magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete labeling information, including magnet contraindications and warnings.



Small in size. Big on features.

How to fit the AirFit F40




Unclip the lower straps away from the frame and place cushion under the nose to fit against the face.

With the ResMed logo facing up, pull headgear over head.



Bring the lower straps behind the ears and reattach to the frame with the magnetic clips*.

Feed longer hair through the back of the frame, if needed.

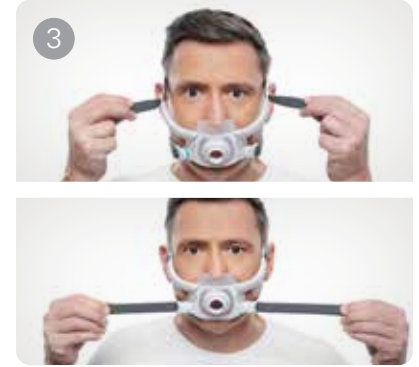

Adjust the upper straps and then the lower straps for a snug fit.

Experience the ResMed AirFit F40 today.

The minimalist ultra-compact full face mask is designed to deliver a reliable and adaptable seal. The soft, total silicone cushion moulds to the face and moves with patients as they sleep.

The ResMed AirFit F40. A minimalist mask, that adapts to your patient's natural sleep.

For more information, visit ResMed.com/AirFitF40 or contact your ResMed sales representative.



*This mask contains magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete labeling information, including magnet contraindications and warnings.

¹ 97.7% Fit range based on data from two ResMed external clinical studies of experienced PAP patients. The first study included users with ≥ 6 months' therapy who used ResMed's AirFit F40 and AirFit F20 full face masks in randomized order for up to seven consecutive nights each at home. Study conducted in Australia, June 6 – 27, 2023, n=28 (AirFit F40) and 29 (AirFit F20). Facial measurements of 29 participants were taken. The second study included current users with ≥ 12 months' PAP therapy who met 90-day CMS compliance for PAP therapy in the three months prior to the study. Participants used ResMed's AirFit F40 and Fisher & Paykel's Evora Full in a randomized order with their own for up to seven consecutive nights each at home. Study conducted in the U.S., October 4 – November 20, 2023, n=57. Facial measurements of 57 participants were taken.

ALWAYS READ THE LABEL AND FOLLOW THE INSTRUCTIONS FOR USE. You should speak to your doctor about your symptoms and whether a CPAP device is suitable for you.



ResMed Pty Ltd
1 Elizabeth Macarthur Drive
Bella Vista NSW 2153 Australia

AirFit, AdaptiSeal and QuietAir are trademarks of the ResMed family of companies. For patent and other intellectual property information, see ResMed.com/ip. © 2024 ResMed Pty Ltd. 10114189/2 2024-08