SHIELD RP201 FFP2 NR Particle Filtering Half Mask € € 0598

EN 149:2001+A1:2009 FFP2 NR

User Instructions

IMPORTANT: Keep these user Instructions for reference.

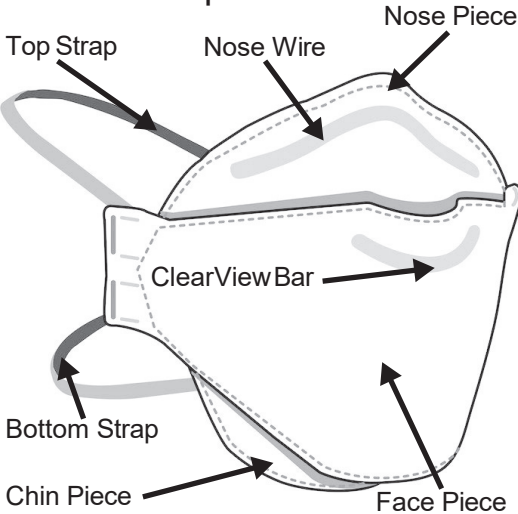
Donning Video

248083505-201 R2

This mask meets the requirements of EN 149:2001 + A1:2009, filtering face piece mask for use against particles. It should be used to protect the wearer from solid and non-volatile liquid particles only.

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Back Side (Top)

How to Wear:

a. Gently cover the front of the mask with both hands, avoiding any disturbance to its position.

b. Exhale forcefully. A positive pressure indicates a secure seal with no leakage.

c. If there is leakage, adjust the straps or nose piece as previously described.

d. If a proper seal cannot be achieved, remove the mask immediately and do not enter the contaminated area.

Remove the mask without touching the facepiece by first lifting the bottom strap over your head and then the top strap. Avoid any contact with the mask. Dispose of the mask appropriately

WARNINGS AND LIMITATIONS

- Failure to follow all instructions and limitations on the use of this product could reduce the effectiveness of the mask and result in sickness or death.
- This mask does not eliminate the risk of contracting any disease or infection. Pathogens could enter your body through your skin or eyes unless you are wearing other PPE in addition to this mask.
- This product does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. At least 19.5%.
- Do not use when concentrations of contaminants are immediately dangerous to health or life.
- This mask is not suitable for explosive environments.
- Evacuate area immediately if breathing becomes difficult, you experience dizziness or other respiratory distress occurs.
- This mask shall not be used in aerosol concentrations that exceed 10 times the occupational exposure limit.
- Mustaches, beards or other facial hair may reduce the effectiveness of this mask. Make sure that nothing interferes with the seal when you are wearing the mask.
- Do NOT alter, modify or clean this mask in any way.
- Do NOT wash the mask or get it wet. If mask gets wet, remove and discard.
- If mask becomes damaged, soiled or breathing becomes difficult; dispose of the mask and use a new one.
- Check the product is within stated shelf life before initial use.
- Mask is for single use only; do not use for more than one shift.
- Efficiency of the filter may deteriorate in the presence of substances, such as oil mists, commonly encountered in the workplace.

STORAGE AND TRANSPORTATION:

Store in accordance with manufacturer's instructions, see packaging

MARKING:

NR = Nonreusable

End of Shelf-life

Temperature Range

Maximum relative humidity

APPROVALS:

This product is type approved and audited annually by: SGS Fimko Oy, Takomotie 8, FIN-00380 Helsinki, Finland. This respirator meets the requirements of European Regulation EU 2016/425 and all applicable local regulations. The European/local legislation and notified body can be determined by reviewing the Certificate and Declaration of Conformity at <https://lwshhealthandsafety.com/certifications/>

Contact Information: LWS Health & Safety, Benelux B.V. de Kazematten 14d 6681-CS Bommel, The Netherlands
Tel: +31 (0)481-715832
www.lwshhealthandsafety.nl
Manufacturer: LWS Health & Safety 300 West Antelope Road White City, OR 97503, USA
Tel: +1 541-770-5905
www.lwshhealthandsafety.com

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