



Use

1. Wheel the Lotus/Lotus compact/Lotus XL to the client and turn away the seat parts.
2. If necessary adjust the legs of the Lotus/Lotus compact/Lotus XL and wheel it as closely as possible to the client.
3. Apply the brake to the Lotus/Lotus compact/Lotus XL.
4. Allow the client to stand up and turn the seat part back.
5. The client can now sit down.

Stand up carrying out the same procedure (back to front)

Safety recommendations

- Read instructions for use carefully (find this at: www.lopital.com) before use.
- Maximum load 200 kg/440 lbs (Lotus/Lotus compact) or 227 kg/500 lbs (Lotus XL).
- Make sure that the Lotus/Lotus compact/Lotus XL is on the brake during client transfers from and to the device.



Wheel blocking




1. Wheel blocking
2. Unlocking

Base spreader




The Lotus/Lotus compact/Lotus XL is fitted with a mechanical adjustable leg base. Stepping on the pedal will move the legs inwards or outwards.

Rotatable seat parts

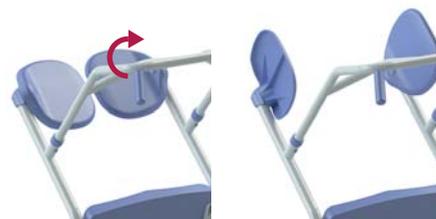

The Lotus/Lotus compact/Lotus XL is fitted with 2 comfortable seat parts that can be turned away.

The Lotus/Lotus compact has an entry of 44 cm /17.3 inch when the seat parts are turned away. The Lotus XL has an entry of 52 cm/20.5 inch.

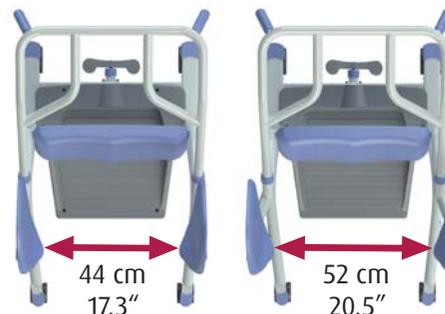

Adjustable legsupport

By loosening the nuts that secure the lower leg support, the lower leg support can be adjusted up or down.



Cleaning and maintenance

The Lotus/Lotus compact/Lotus XL must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the PUR components can be disinfected with a solution of 70% ethanol or propanol. The complete maintenance schedule and cleaning protocol can be found in the instructions for use. (www.lopital.com)

According to the regulation MDR (EU) 2017/745 for medical devices, the Lotus/Lotus compact/Lotus XL must be maintained in line with the manufacturer's instructions. Those instructions can be found in the maintenance protocol, see website Lopital (www.lopital.com).

In the event of a breakdown, contact your Lopital distributor.

Instructions for use

For the full Instructions for use go to www.lopital.com



Important!

This quick user manual may not be considered a replacement for the instruction for use!