



### Safety recommendations

- Read instructions for use carefully (find this at: [www.lopital.com](http://www.lopital.com)) before use.
- Maximum load 250 kg / 550 lbs.
- Make sure that the chair is on the brake during client transfers from and to the device.
- Place both feet on the foot supports.




### Use

1. Place the chair in the right position and turn the foot rests away.
2. Apply the brakes to the chair!
3. If necessary, fold the armrests up.
4. Support the client when they are taking a seat.
5. Shut the armrests and let the client's feet rest on the footplates.


Stand up carrying out the same procedure (back to front)

### Wheel blocking

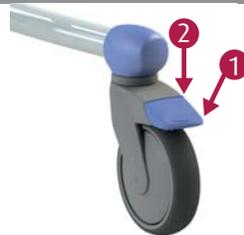

1. Wheel blocking
2. Unlocking

### Directional / braked wheel

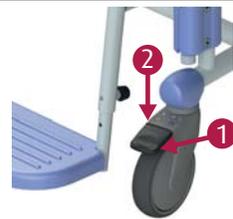

1. To block the wheel direction(1x)  
Wheel blocking(2x)
2. Unlocking

### Removable seat

If required, you can remove the Tango XL seat from the frame. That makes it much easier to clean it and no dirt will remain between the frame and the seat.



### Foot rest



Both foot plates must be folded away for the client to take a seat in the chair. To do so, fold up the foot plates or turn out both supports. Make sure that the client has both feet on the foot plates when moving.

### Armrest

1. Folding up



### Toilet function

- *In combination with toilet:* The Tango XL can be moved over almost every toilet.
- *With bedpan:* The Tango XL can also be used as a toilet chair. Simply slide the stainless steel / plastic bedpan under the chair using the guides.

### Cleaning and maintenance

The Tango XL must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the PUR components can be disinfected with a solution of 70% ethanol or propanol. The complete maintenance schedule and cleaning protocol can be found in the instructions for use. ([www.lopital.com](http://www.lopital.com))

According to the regulation MDR (EU) 2017/745 for medical devices, a chair must be maintained in line with the manufacturer's instructions. Those instructions can be found in the maintenance protocol, see website Lopital ([www.lopital.com](http://www.lopital.com)).

In the event of a breakdown, contact your Lopital distributor.

### Instructions for use

For the full Instructions for use go to [www.lopital.com](http://www.lopital.com)



### Important!

This quick user manual may not be considered a replacement for the instruction for use!