

Safety recommendations

- Read instructions for use carefully (find this at: www.lopital.com) before use.
- Maximum load 135 kg / 300 lbs.
- Make sure that the chair is on the brake during client transfers from and to the device.
- Place both feet on the foot supports.
- Front closure is closed during up / down movements and arms should rest on armrest.
- Only relocate and move the chair in the lower position.



Operation

Up



Down



Tilt forwards




Tilt backwards



Activate



Lock button

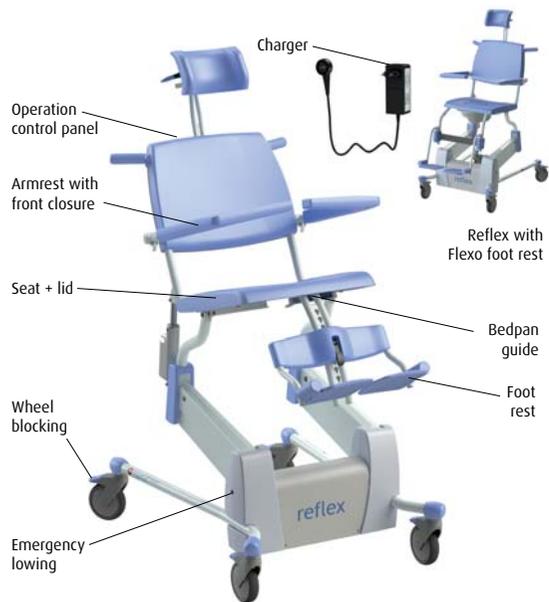


Gebruik

1. Activate the Reflex by pressing the two 'up' buttons simultaneously for 1 second.
Note: After use, the Reflex will go into a sleep setting after 60 minutes.
2. Move the chair into the right position; adjust the head and foot rests too (or fold them up).
3. Apply the brakes to the chair!
4. Open the armrests and fold these up if necessary.
5. Support the client when they are taking a seat.
6. Shut the armrests and let the client's feet rest on the footplates.


Stand up carrying out the same procedure (back to front). All functions on the Reflex are halted as a result of using the lock button. The Reflex must be reset. You do this by pressing both 'up' buttons for 4 seconds.

Wheel blocking

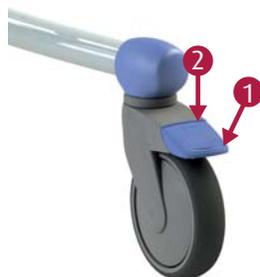

1. Wheel blocking
2. Unlocking

Foot rest

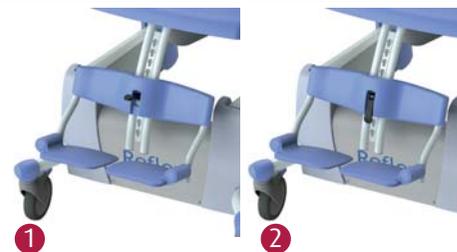


1. Unlocking
2. Remove complete foot rest
3. Choose desired position
4. Hook in and lock

Make sure that the client has both feet on the foot plates when moving.

Armrest with front closure

1. Unlocking
2. Rotating
3. Folding up

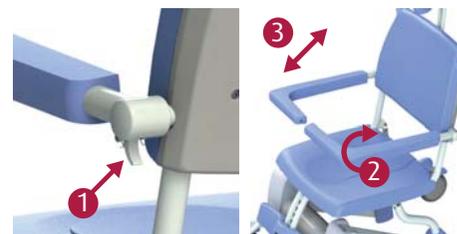

Charging batteries

After using the Reflex, connect it to the charger to maintain the batteries.



Off
Green
Blinking red
Red

Sleep setting
Chair ready to use
Battery is almost empty
Battery is empty

The Reflex will not work during charging.

Important!

This quick user manual may not be considered a replacement for the instruction for use!

Toiletfunction

- Remove the lid on the seat before the client sits down in the chair.
- *In combination with toilet:* The Reflex can be moved over almost every toilet. Move the chair upwards or downwards to achieve the right position.
- *With bedpan:* The Reflex can also be used as a toilet chair. Simply slide the stainless steel / plastic bedpan under the chair using the guides.

Emergency lowering

If the Reflex breaks down, you can lower the chair using the emergency lowering handle.



Cleaning and maintenance

The Reflex must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the PUR components can be disinfected with a solution of 70% ethanol or propanol. The complete maintenance schedule and cleaning protocol can be found in the instructions for use. (www.lopital.com)

According to the regulation MDR (EU) 2017/745 for medical devices, a chair must be maintained in line with the manufacturer's instructions. Those instructions can be found in the maintenance protocol, see website Lopital (www.lopital.com).

In the event of a breakdown, contact your Lopital distributor.

Instructions for use
For the full Instructions for use go to www.lopital.com

