


Dummy Straw Instructions



Often mistaken as unnecessary, regular shocking of dental unit waterlines is a vital component of every effective waterline maintenance protocol. It allows for low-level antimicrobial products like straws/cartridges to perform at their best. Follow these instructions to utilize the dummy straw provided by ProEdge Dental Water Labs.

- 1.** Remove your water bottle. Unscrew your cartridge/straw from the quick-connect fitting.

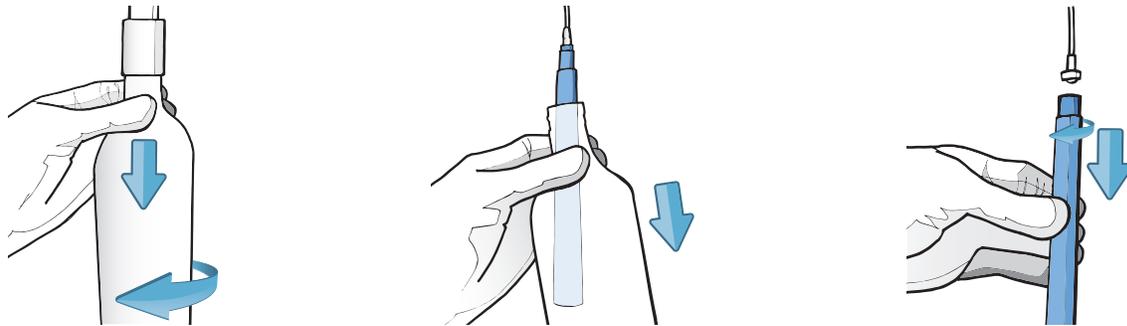


- 2.** Attach dummy straw by simply twisting the male quick-connect fitting into the female fitting (about 1/2 turn is all it takes). Trim the length with scissors if necessary.



Dummy straws are reusable!

Each dummy straw should be kept and stored for your next shock.

- 3.** Complete shocking procedure. After shocking, reattach your cartridge/straw and make a note of the install date.

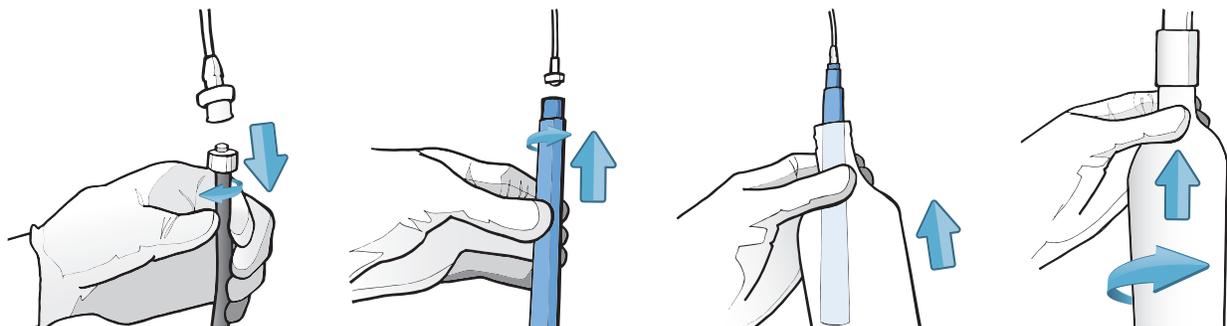

A match made in dental heaven.

