

BluTube Install Guide for Standard Bottles

What's in the Box:



BluTube® cartridge



Gray dummy straw



6-month indicator



Clear female luer lock




White male luer lock

What you will need:

- Pair of scissors
- Measuring tape

Step 1.

Practicing aseptic technique, remove dark blue vinyl shipping caps from both ends of BluTube cartridge. Attach clear female luer lock to BluTube cartridge.

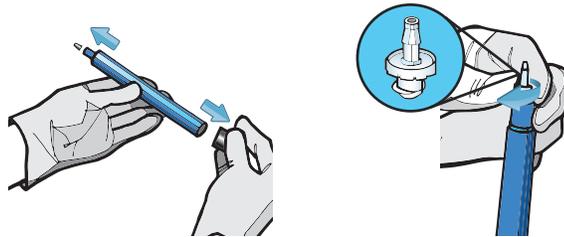

Step 2.

Remove water bottle from dental unit. Cut the uptake tube according to the table below and attach to the barb on the manifold.




Bottle Size	Uptake Tube Length
750mL	2 1/4"
2L	7 1/4"




Step 3.

Connect BluTube to the uptake tube using the attached luer lock. Reattach water bottle to unit. Set air pressure to 45psi.

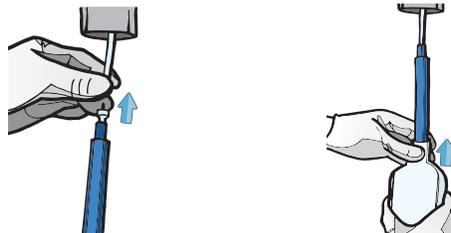

Step 4.

Flush cold tap water through BluTube cartridge for 3-5 minutes. It is normal for a slight tint during initial flush. Adhere the supplied 6-month indicator to the outside of the bottle. Activate 6-month replacement indicator by pressing firmly on the bubble. At six months, the entire window bar will be filled with red, letting you know it's time to replace the BluTube cartridge.

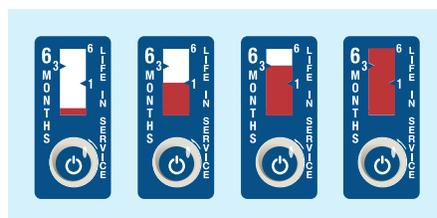

Pro tip! Some offices will place the 6-month indicator on their maintenance calendar to serve as the reminder that it's time to change your BluTube.

BluTube Install Guide for A-dec Bottles

What's in the Box:



BluTube® cartridge



Gray dummy straw



6-month indicator



Clear female luer lock



White male luer lock

What you will need:

- Pair of scissors
- Measuring tape
- 1/8" allen wrench


Step 1.

Practicing aseptic technique, remove A-dec bottle by pushing up while twisting ¼ turn to the left. Then take a 1/8 inch allen wrench and remove both screws from collar and set aside. Separate collar and lift uptake tube from bottle.



Step 2.


Take BluTube® cartridge and remove dark blue vinyl shipping caps from both ends of cartridge. Twist female luer lock into male end of BluTube cartridge.

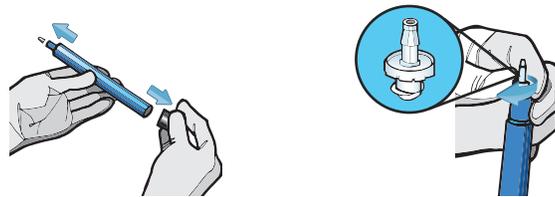

Step 3.


Cut the uptake tube according to the table below. Using luer lock, connect BluTube to the cut end of uptake tube. Place the uptake tube and BluTube into A-dec bottle.

Bottle Size	Uptake Tube Length
A-dec	3/4"

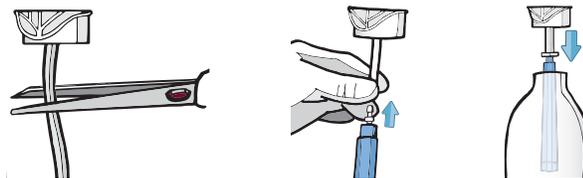


Step 4.

Locate notches on inside of collar. Match notches to indentations on bottle and uptake tube assembly (they connect at the lowest section of the manifold). Attach one side of the collar at a time. If it feels loose, try twisting collar around until notches drop into indentations in bottle and uptake tube assembly. Use the allen wrench and previously removed screws to secure collar back together. Ensure the seal is tight and there are no visible gaps. Lift and turn A-dec bottle to the right to reconnect. Set to 45psi.

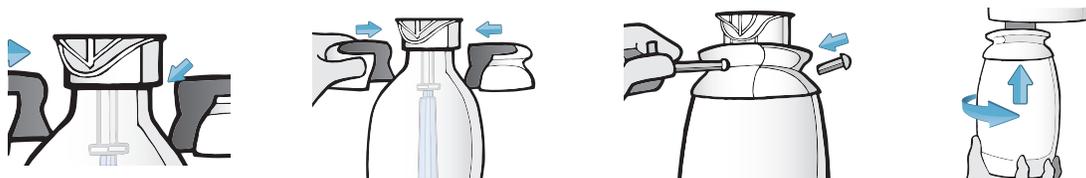

Step 5.

Flush cold tap water through BluTube cartridge for 3-5 minutes. It is normal for a slight tint during initial flush. Adhere the supplied 6-month indicator to the outside of the bottle. Activate 6-month replacement indicator by pressing firmly on the bubble. Store other BluTube cartridge in a cool, dry place until ready for use.

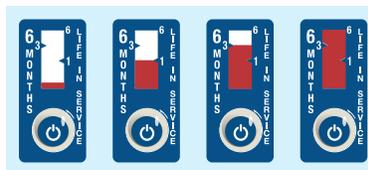

Pro tip! Some offices will place the 6-month indicator on their maintenance calendar to serve as the reminder that it's time to change your BluTube.