

# BluTab® Quick Start Guide

Waterline Maintenance Tablets

BluTab® Waterline Maintenance Tablets are formulated specifically for dental unit waterlines. BluTab is designed to be continuously present in your waterlines and helps keep lines clean.

## Read First:

Before starting a treatment program with BluTab®, shock lines with diluted bleach or a shock product recommended by your equipment manufacturer to remove potential biofilm buildup. Repeat shock procedure quarterly or more frequently when water test results require. For resources on how to shock: [proedgedental.com/shock](http://proedgedental.com/shock)

## Directions for Use:

# 1

Place a tablet or tablets (see table below) in your empty dental unit water bottle each time you refill with water.

# 2

Fill bottle with water and reconnect to the dental unit. Tablet(s) will dissolve in a few minutes.

# 3

BluTab may be left in the waterlines overnight and for extended periods of non-use. BluTab has been shown to be effective for up to 28 days. Use BluTab treated water until water bottle is empty; or within 28 days. It is not necessary to purge lines at the end of each day.

| Water Volume | 750 mL Tablets | 2L Tablets |
|--------------|----------------|------------|
| 700-750 mL   | 1              | -          |
| 1500 mL      | 2              | -          |
| 2L           | 3              | 1          |

### Pro Tip: Premix

Many offices premix 1 gallon of BluTab/water solution to fill their water bottles. This is also a great way to consistently use the same dilution of BluTab to water every time. Simply add six 750mL tablets or two 2L tablets to one gallon of water. Use premix solution within 28 days.

## Don't Forget!

Water testing ensures proper compliance standards are met and procedures are performed correctly. We recommend testing your water quality quarterly to ensure your waterline maintenance program is effective. If water quality results approach or exceed the EPA/CDC standard of 500 CFU/mL, repeat your shock protocol and retest.